

Rotary Style SE Valve Installation

(Model #'s 03-21 & 03-27- Non-autoclavable)


Step 1- Shut off vacuum.

Step 2- Cut tubing immediately below existing valve and discard.

Step 3- Attach the valve to the tubing by inserting the barbed base into your tubing. Dipping the end of your tubing in hot water will make it easier to slip over the barb.

Note- After each disinfection- completely disassemble by pulling knob out (as shown at right). Clean & thoroughly rinse with water and lubricate all moving parts with a silicone based lubricant.

- DO NOT USE BLEACH ON THESE PRODUCTS.
- DISINFECT ONLY - NOT HEAT STERILIZABLE.

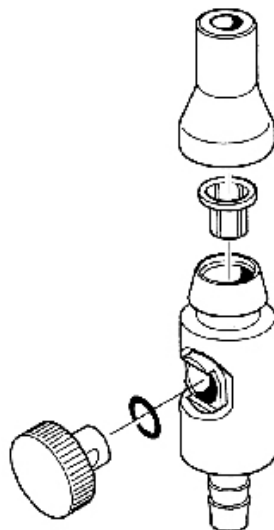

AMERICANdental
accessories

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

www.amerdental.com