
2 Hole Foot Control Installation

(Model #'s 04-01 & 04-20 — 04-01 shown)

- Step 1- Turn the unit off.
- Step 2- Note where each line of your existing foot control is connected & disconnect.
- Step 3- The "ribbed" tubing connects to the source of air in the junction box. The "rib" runs the length of the tubing, like a seam. The "non-ribbed" tubing connects to unit for drive air.
- **Step 4-** Trim the flash from the end of the tubing and slip sleeve clamps onto tubing with the internal bevel facing the end of the tubing.
- Step 5- Slip the tubing over the barbs making certain that sleeve tools are already over the tubing & pushed back.
- **Step 6-** Lock tubing by using pliers or sleeve tool to slide sleeve clamps down over tubing & barbs.
- Step 7- Turn unit on and test for leaks and proper function. If the foot control "hisses" (audible air leak) constantly when the unit is on the two tubes have been reversed. Disconnect, switch and re-attach.
- NoteTake care not to break barbs when installing tubing and clamps. Inserting tubing ends in hot water will soften the tubing making it easier to slide over barbed fittings.

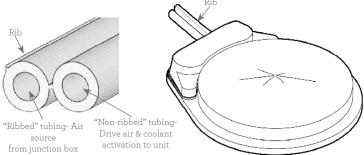



accessories

7310 Oxford Street • St. Louis Park, MN • 55426
Phone (800) 331-7993 • Fax (888) 729-1016

www.amerdental.com