


Two-way Routing Valve Installation

(Model #05-08)

- Step 1-** Shut unit off and depressurize the unit by depressing the buttons on the air/water syringe.
- Step 2-** Cut tubing as close as possible to old toggle using a wire cutter and remove existing toggle noting which tube is connected to the inlet and outlet as shown below.
- Step 3-** Install appropriate sized barbs with washers and tighten securely with a 1/4" open ended wrench.
- Step 4-** Apply liquid soap to tubing or dip tubing ends in hot water and slip sleeve clamps over tubing (approx. 3/8" from end) ensuring the interior bevel of the sleeve clamp faces the barb.
- Step 5-** Slip tubing over barbs and lock tubing by using a pliers or sleeve tool to slide the sleeve clamps over tubing and barb.
- Step 6-** Mount to unit using provided nut. Turn unit on and test for leaks and proper function.

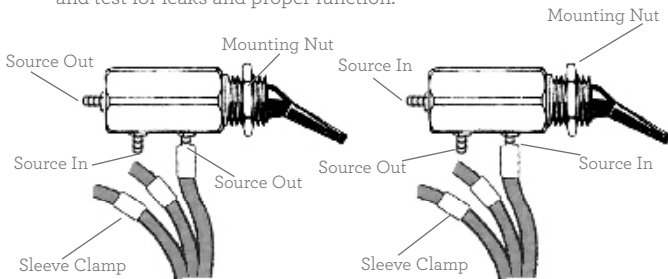

1 source in & 2 sources
out- normally for
water use (bottles)

2 sources in & 1 source
out- normally for air use
(handpiece selection)



AMERICANdental
accessories

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

www.amerdental.com