


## A-dec Style Toggle Valve Installation

(Model #05-10- momentary valve for air/water use, pre-1982 cuspidors)

- Step 1-** Shut unit off and depressurize the unit by depressing the buttons on the air/water syringe.
- Step 2-** Cut tubing and remove existing toggle, noting the tubing sources below.
- Step 3-** Install appropriate sized barb with washer.
- Step 4-** Apply liquid soap to tubing and slip sleeve clamp over tubing (approx. 3/8" from end) insuring the interior beval faces the barb.
- Step 5-** Slip tubing over barb and lock tubing by sliding sleeve clamp over tubing portion on the barb with a pliers or sleeve tool.
- Step 6-** Mount to unit using provided nut. Turn unit on and test for leaks and proper function.
- Note:** Take care not to break barb when installing tubing and clamp. Inserting tubing end in hot water will aid this process.

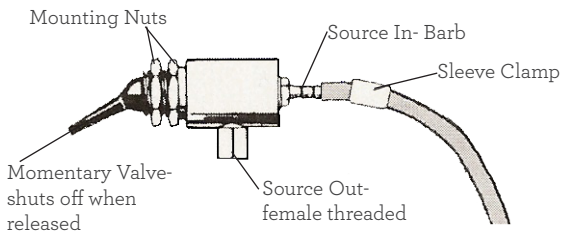

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

[www.amerdental.com](http://www.amerdental.com)