


# A-dec Style Signal Air Selector Valve Installation

(Models #05-18- for Century II cuspidors)

- Step 1-** Shut unit off and depressurize the existing valve by depressing the buttons on the air/water syringe.
- Step 2-** Cut tubings and remove existing valve noting the tubing sources below (2 configurations).
- Step 3-** Apply liquid soap to tubings and slip sleeve clamps over tubings (approx. 3/8" from end).
- Step 4-** Slip tubings over barbs and lock tubings by sliding sleeve clamps over tubing portion on the barb with a pliers or sleeve tool.
- Step 5-** Mount to unit and test for leaks and proper function.
- Note:** Take care not to break barbs when installing tubings and clamps. Inserting tubing ends in hot water will aid this process.

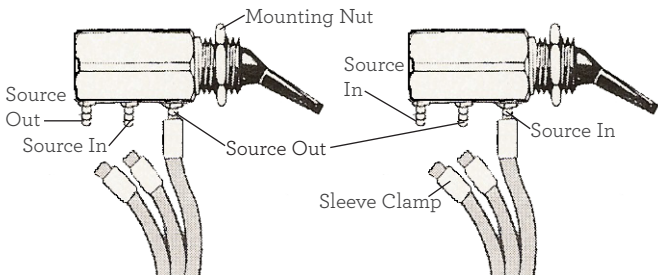

1 source in & 2 sources  
out- normally for water use

2 sources in & 1 source  
out- normally for air use



**AMERICAN**dental  
accessories

7310 Oxford Street • St. Louis Park, MN • 55426

Phone (800) 331-7993 • Fax (888) 729-1016

[www.amerdental.com](http://www.amerdental.com)