

Manifold Routing Block

(Item #49-83)

FUNCTION:

Routes up to 4 air or water inlet lines to 2 or 3 outlet lines each. Most often used to route main air and water and drive air in to various secondary valves, handpieces or syringe.

INSTALLATION:

1. Make certain to attach the tubings exactly as shown in the illustration.
2. Secure the tubing to the barbs using the provided sleeve clamps. If an inlet(s) is unused, the outlet(s) can be left open. If all outlets for a connected inlet are not used, the extra outlets should be plugged using a (#14-64) barb port plug.
3. Contact American Dental technical support if you have any questions concerning the installation of the routing block.

NOTE:

The routing block does not provide any on/off or flow control functions. It simply splits multiple inlets into even more outlets, which will ultimately feed to secondary valves that will perform the on/off and or flow control (e.g. handpiece control block).

