# INSTRUCTIONS FOR USE

# Cinch®

## Vinyl Polysiloxane Impression Materials

REF S453S, Various

For Safety Data Sheet (SDS) go to [www.parkell.com](http://www.parkell.com)

EN

Store material at room temperature (2° - 25° C / 36° - 77° F). Avoid excessive heat, light or moisture.

### DESCRIPTION AND INDICATIONS

Parkell's Cinch® DENTAL IMPRESSION MATERIALS consist of original Cinch® (4 minute set time) and QUICK Cinch® (90 second set time). All Cinch materials are addition-reaction silicones, also known as A-Silicones, polyvinyl siloxanes, PVS, vinyl polysiloxanes or VPS. Cinch products are available in light, medium and heavy viscosities. They are hydrophilic, and are capable of producing accurate C&B impressions with excellent tear strength. All Cinch impression materials are available in dual-barrel split cartridges that are dispensed using a high-efficiency, DS-50 (1:1, 2:1) impression gun. Some Cinch products are also available in tubes or high capacity Super-Cartridges, as listed in the included chart. The light-bodied materials are low viscosity crown & bridge washes that are meant to be used inside their correspondingly timed medium or heavy-bodied tray materials.

### IMPORTANT—For best results, we recommend that:

- Cinch Light should be used with Cinch Medium (Cinch Platinum), or Cinch Heavy (Cinch Automix Putty), each of which sets in 4 minutes, and
- QUICK Cinch Light should be used with QUICK Cinch Medium (Cinch-90) or QUICK Cinch Heavy, each of which sets in 90 seconds. Since mouth temperature accelerates the working and setting times of QUICK Cinch, make sure to insert the tray material into the mouth within 30-45 seconds of syringing the wash material onto the teeth.

This matching of setting times will permit the entire impression to set in the same amount of time. This eliminates the possible error of removing the impression from the mouth too soon, when the outside is set, but the inside is not.

- Excessive or intense squeezing force on cartridge-based restorative materials may cause gun breakage or cartridge cracking. Slow and steady force is preferred for dispensing these materials.
- Stated work times and set times are estimates, and may vary based upon storage condition, temperature, humidity, age of product, when the cartridge was first opened, among other factors.

### STORAGE AND GENERAL USE INSTRUCTIONS FOR THE CINCH IMPRESSION MATERIAL PRODUCTS

1. All Cinch products should be stored between 2° C (36° F) and 25° C (77° F). Exposure to excess heat will shorten shelf life. For best results, use at normal room temperature. To extend work/set times, the materials may be slightly chilled. However, cold reduces flowability.
2. Avoid contact between Cinch products and all latex or sulfur-containing products, including latex rubber gloves or rubber dams, and hemostatic agents with sulfates (eg. Aluminum Sulfate, Ferric Subsulfate). If teeth or gingiva are contaminated with these products, scrub with a 5% hydrogen peroxide solution on gauze, and follow with a copious water rinse before impressioning. Chloride hemostatic agents are acceptable substitutes.
3. Vinyl polysiloxanes like Cinch are not compatible with polysulfides or polyethers.
4. Acrylic or methacrylate residues from bonding agents and restorative resins can inhibit the setting of Cinch products. Remove the outer uncured layer of such products before impressioning. Avoid making temporaries inside final impressions.
5. The set materials may be disinfected using glutaraldehyde or most other standard dental surface disinfectant products.

6. Allergic reactions may occur in susceptible individuals. Flush affected tissues with copious water and consult a physician.
7. All Cinch products will work with equivalent products from other manufacturers.

## **SUGGESTED IMPRESSION TECHNIQUES FOR CINCH IMPRESSION MATERIALS**

The Cinch and QUICK Cinch materials are used together using several suggested methods, including:

- a. Single Step / Dual Body Method (Wet-Wet), where unset medium body or heavy body Cinch or QUICK Cinch is placed over unset light body Cinch or QUICK Cinch that has been syringed over the teeth. The materials set simultaneously
- b. Double Step / Dual Body Method (Dry-Wet or Tray-Wash Method), where a prefab or custom tray filled with medium body or heavy body Cinch or QUICK Cinch is used to make a primary impression on the teeth and allowed to fully set. After the prepping is complete, the initial impression material is relieved, cleaned, dried and a second impression is made in the first one, using the light body Cinch or QUICK Cinch to accurately reproduce detail.

**NOTE: THE PRIMARY (TRAY) IMPRESSION SHOULD NOT BE USED AS A MATRIX FOR A BIS-ACRYL TEMPORARY. THE UNCURED BIS-ACRYL SLICK LAYER WILL CONTAMINATE THE BOND BETWEEN THE IMPRESSION MATERIALS, CAUSING SEPARATION (DELAMINATION) TO OCCUR BETWEEN THEM.**

## **IMPRESSION PROCEDURE FOR CINCH USING DS-50 IMPRESSION GUNS**

1. Insert Cinch wash and tray viscosity cartridges of the same setting time into two separate impression guns. Twist off and discard the sealing caps. Bleed about  $\frac{1}{4}$ " of material from the orifices, ensuring that it is visible at both holes. Wipe the ends clean, avoiding cross-contamination.
2. Attach a pink-hubbed mixing tip to the light-bodied wash material cartridge and lock tight one-quarter turn. Affix a white intraoral tip (<sup>REF</sup> S439IOT) to the mixer and verify that the hole at the end is round and open.
3. Attach a pink or teal-colored mixing tip to the medium or heavy-bodied tray material cartridge, lock tight, and load the tray.
4. If a putty-like consistency is desired in the tray material, use Cinch Automix Putty and wait 90 seconds before inserting the tray into the mouth. For normal heavy body properties, either heavy-bodied Cinch can be used without the wait.
5. Quickly remove the retraction cord, and rinse and dry the tooth.

6. Bleed a small amount of light-bodied material thru the intra-oral tip outside of the mouth, and immediately syringe it into the gingival sulcus and over all the desired teeth, keeping the tip immersed in impression material to avoid air entrapment.
7. Place the loaded impression tray over the teeth, and remove the set impression with a quick, firm motion. Use the chart above to determine the working and setting times for the Cinch or QUICK Cinch product you are using.
8. Leave mixing tips on cartridges as sealing caps. Disinfect the cartridges and store them horizontally until the next use.
9. With Parkell's Quick-Pour formula, Cinch or QUICK Cinch impressions may be poured in any dental stone 15 minutes after removal from the mouth. Properly stored impressions may be poured up to 30 days later.

## **IMPRESSION PROCEDURE FOR CINCH PLATINUM IN TUBES**

1. Dispense equal lengths of base and catalyst (1:1) onto a mixing pad. Altering the ratio will impair final set.
2. Mix by hand for 30-45 seconds with clean spatula until color is uniform and streak-free, with no air bubbles.
3. Recap tubes immediately, making sure not to interchange the color-coded caps.
4. Load tray and syringe and proceed to follow standard impression procedure.

## **HOW TO USE CINCH IN THE 380 ML SUPER-CARTRIDGE**

1. With the Super-Cartridge seated on a flat surface and the nozzles aimed up, hold the cartridge with one hand and bend the tab on the cap upward.
2. While holding the cartridge with one hand, remove the cap.
3. Turn the retraction knob on the machine clockwise to completely retract the pistons.
4. Insert the cartridge into the mixer (Pentamix, Pentamix II or Mixstar) according to the mixer instructions. While holding the knob to keep the mixer pistons in the most-retracted position, insert the nozzle-end of the cartridge first, then seat the barrels.
5. Turn the retraction knob counter-clockwise so the plungers engage the cartridge plugs.
6. Bleed the cartridge before each use according to the mixer instructions to assure that the material is flowing properly. **Note: When cleaning off the expressed impression material, wipe the nozzles vertically (not horizontally) to avoid cross contamination of the nozzles.**

7. Install a mixing tip: Orient the tip so the large orifice on the tip will be inserted into the large orifice on the cartridge. Take care that the hex recess on the tip is aligned with the mixer's drive shaft. (Do not substitute mixing tips. A tip not specifically made for this cartridge may appear to fit, yet leak badly when used.)
8. Lock the mixing tip in position by slipping the yellow locking cap over the mixing tip and turning clockwise.
9. Dispense the Cinch material.
10. Immediately after filling the tray, relieve pressure on the plungers by turning the sideknob clockwise to disengage the plungers.
11. Leave the used mixing tip on the cartridge until ready for the next impression.
12. To change tips immediately before next use: Remove yellow ring by turning counter-clockwise. Remove mixing tip. Bleed cartridge (Step 6). Affix new tip (Step 7). Lock in place with yellow ring (Step 8).
13. To change cartridges, remove the yellow locking ring from the old cartridge by turning counter-clockwise. Turn the machine's retraction knob as far clockwise as possible to completely disengage the plungers. Remove the cartridge.

## **DISINFECTION OF CONTAINERS**

Cartridges and caps should be cleaned of debris with a water-damp paper towel and disinfected with a paper towel saturated with an EPA-registered low-level (HIV/HBV claim) to intermediate-level (tuberculocidal claim) hospital disinfectant. Afterwards, the chemical residue should be wiped off with a water-damp paper towel and air-dried. Utilize the surface disinfecting protocol of the disinfectant manufacturer. Consult [www.CDC.gov](http://www.CDC.gov) for the most recent version of the "Guidelines for Infection Control in Dental Health-Care Settings".

**Tips for Dental Practitioners:** the following information should help dental practitioners reduce the risk of cross-contamination between patients when using multiple-use dental dispensers: apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient; use new, uncontaminated gloves when handling multiple-use dental dispensers; utilize dental assistants to dispense material for the dentist; avoid contact of the reusable parts (e.g., the body of the multiple-use dental dispenser) with the patient's mouth; do not reuse the multiple-use dental dispenser if it becomes contaminated; do not reprocess a contaminated multiple-use dental dispenser by using chemical wipes or disinfectants; do not immerse multiple-use dental dispensers in a high level chemical disinfectant, as this may damage the dispenser and the material contained in the device; do not sterilize multiple-use dental dispensers, as this may damage the material contained in the device.

## **WARRANTY AND TERMS OF USE:**

For full Warranty and Terms of Use information, please see [www.parkell.com](http://www.parkell.com).

Safety Data Sheets (SDS) are available at [www.parkell.com](http://www.parkell.com). Parkell's Quality System is certified to ISO 13485.

## PROPERTIES OF CINCH AND QUICK CINCH MATERIALS

Product Viscosity	Cinch Light Body	Cinch Medium Body	Cinch Heavy Body
Product Name & Setting Time	Quick Cinch Light	Cinch 90	Cinch Platin-
Working time (@ room temp.)	60 seconds	60 seconds	3 minutes
Setting time (@ mouth temp.)	90 seconds	90 seconds	4 minutes
Final Durometer	45 (Shore A)	45 (Shore A)	72 (Shore A)
Stock Numbers for four (4) cartridges/50ml ea. (200ml ttl.)	<a href="#">REF</a> S461S (mixing tip is Pink <a href="#">REF</a> S445)	<a href="#">REF</a> S460S (mixing tip is Pink <a href="#">REF</a> S445)	<a href="#">REF</a> S454S (mixing tip is Pink <a href="#">REF</a> S445)
Stock Number for one (1) Super-Cartridge (380ml ttl.)		<a href="#">REF</a> S464 (mixing tip is <a href="#">REF</a> S472)	<a href="#">REF</a> S466 (mixing tip is <a href="#">REF</a> S472)
Stock Number for two pairs of tubes (320ml ttl.)			<a href="#">REF</a> S435
Intra-oral tip	<a href="#">REF</a> S439IOT	<a href="#">REF</a> S439IOT	<a href="#">REF</a> S439IOT

Each Cinch Impression Material Kit contains the proper mixing tips or tube squeezers for that product. VPS Adhesive ([REF](#) S446), the DS-50 Mixing Gun ([REF](#) S343), replacement mixing tips and intra-oral tips are available separately from Parkell or many dental dealers.

**Opbevar materialet ved stuetemperatur (2-25° C/36-77° F). Undgå overdreven varme, lys eller fugt.**

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## EXPLANATION OF SYMBOLS USED

	Follow instructions for use
	Store product above 2° C and below 25° C (36° - 77° F).
	Keep away from sunlight
	Manufacturer
	Use-by date (expiration)
	Batch code
	Catalogue / stock number
	Package contents
	Do not use if package is damaged.
	Single use only
	50 ml Split Cartridge / Paste
	380 ml Super-Cartridge / Paste
	Mixing Tip
	Intraoral Tip
	380 ml Super-Cartridge Tip
	380 ml Super-Cartridge Collar/Locking Cap

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Directa AB, Finvids väg 8 SE-194 47 Upplands Väsby, Sweden