

MAKE BETTER MODELS IN LESS TIME WITH **wonderfill**.
 — THE EZ IN & OUT TONGUE AND VOID FILLER!

FOR MODEL HEADACHES REACH FOR THE “PINK STUFF!”

Save up to 20 minutes!

- 🕒 Water-soluble & easy clean-up
- 🕒 No need to mix alginate filler
- 🕒 Eliminates grinding out stone from lower impressions
- 🕒 Holds up under heat when making heat-shrink retainers
- 🕒 Fills in bubbles, air pockets, missing teeth, & undercuts
- 🕒 Enhances the fit of appliances & bleaching trays
- 🕒 Excellent in creating a temporary dam when making partial molds

Just Compare —
 approximate time to
 make a model if using:

- Wonderfill
5 minutes
- Alginate
10 minutes
- Plaster/Stone
25 minutes



\$46.95

Net Wt. 1.13 kg
 Item #344



Wonderfill Instructions - For Tongue Filler

1. Form a domed bridge in the tongue area.
2. With Wonderfill in the tongue area, pour plaster/stone.
3. After plaster/stone fully sets, pop out non-sticky Wonderfill.
4. Using only water & a brush, remove the slight residue that might remain.



For Bleaching Tray

1. Use Wonderfill to block out area to be bleached.
2. Directly apply Wonderfreeze to vacuum forming material.
3. Remove Wonderfill. Ready to use!