Cleaning crystals Instructions for use

ENGLISH

Device description

Invisalign Cleaning Crystals are intended to clean Invisalign System aligners and retainers after the patient has removed them from their mouth. Invisalign Cleaning Crystals may also be used by Doctors who would prefer a quick and effective method for cleaning aligners/ retainers in their office. Invisalign Cleaning Crystals are provided in individual packets that are mixed in water to clean the aligner/retainer.

Indication for use

Invisalign Cleaning Crystals are intended to clean Invisalign System aligners and retainers after the patient has removed them from their mouth.

Warnings

Do not place crystals in mouth or ingest. In case of accidental ingestion, immediately contact a poison-control center. Keep this product out of the reach of children and pets. Do not use to clean natural teeth, orthodontic appliances containing metal parts, jewelry, or other objects. Avoid splashing cleaning solution on clothing and fabrics.

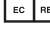
Aligner/retainer cleaning instructions

- Remove aligners/retainers and rinse under tap water.
- Place aligners/retainers into a cleaning container and fill with 100 ml warm water or to fill mark on tub.
- Add one packet of cleaning crystals.
- Gently shake/agitate for 20 seconds to dissolve and distribute crystals.
- Let stand for 15 minutes.
- Gently shake/agitate again for 20 seconds.
- Rinse aligners/retainers and container thoroughly with warm water.

Storage

Store crystals in a cool, dry place.

Symbols

	Catalogue number		Caution
	Batch code		Manufacturer
	Use-by date		Authorized Representative in the European Community
	Consult instructions for use		Conformity with the European Community Council Directive 93/42/EEC, the Medical Device Directive.

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Align Technology, Inc.
Align Technology B.V.