


REDGUM^{BRAND}
Daily Living and
Lifestyle Products

SPACE SAVER CORNER SHOWER STOOL

RG553

Edition 3

 **REGISTERED
DESIGN**

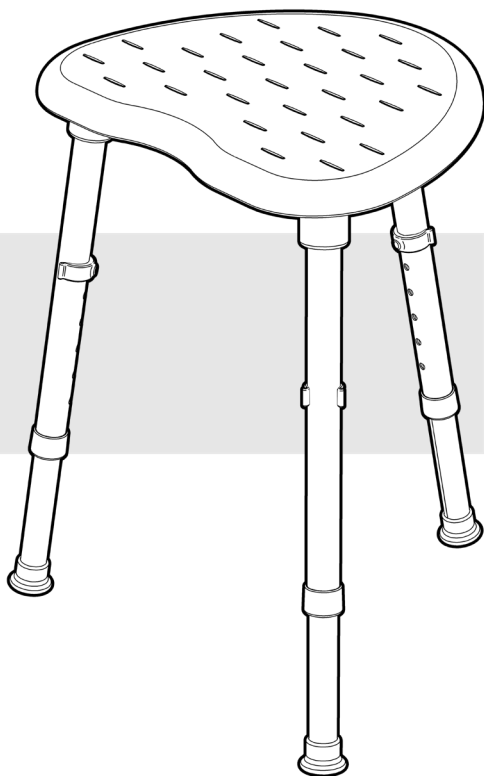

USER MANUAL

Please read and keep these instructions


ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT IMPROVEMENT

INTRODUCING THE ALIGNMENT LINE

Setting up the legs to the required height has now become easier with the addition of the Alignment line!

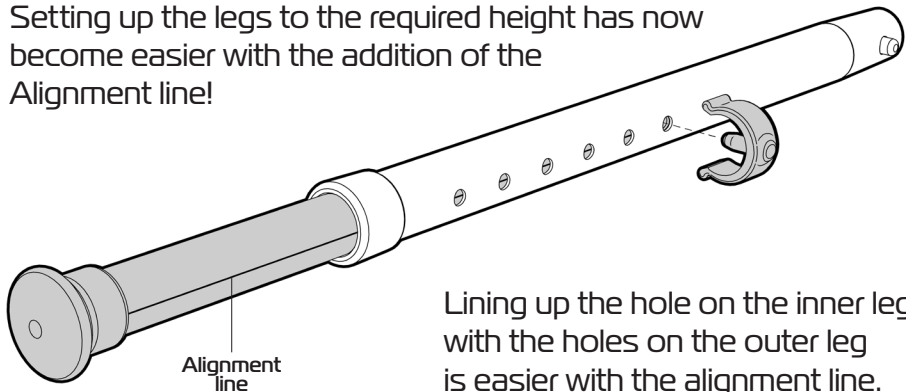

Lining up the hole on the inner leg with the holes on the outer leg is easier with the alignment line.

REDGUM^{BRAND}
*Daily Living and
Lifestyle Products*

REDGUM Brand owns the copyright to the **Space Saver Corner Shower Stool** user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**

FOR-DE
GROUP PTY LTD


REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - LEG ASSEMBLY
5. **ASSEMBLY** - LEG INSTALLATION
6. **USAGE GUIDELINES** - DO's + DON'Ts
7. **PRODUCT CARE** - INSPECTIONS / CLEANING
8. **WARRANTY**
9. **ACCESSORIES**
10. **SPACE SAVER PRODUCT RANGE**
11. **REDGUM BRAND PRODUCT RANGE**
12. **NOTES**

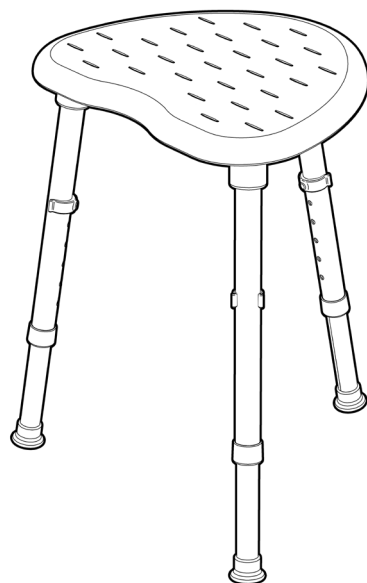

INTRODUCTION

Thank you for purchasing this quality **REDGUM Brand** product.

If you have any issues or questions about the **Space Saver Corner Shower Stool** please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use your **Space Saver Corner Shower Stool** for its intended purpose which is as a seating solution that can adjust to suit the users individual requirements.


The Space Saver Corner Shower Stool is a **Type B** product.
(Limited usage - Domestic)

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 130kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

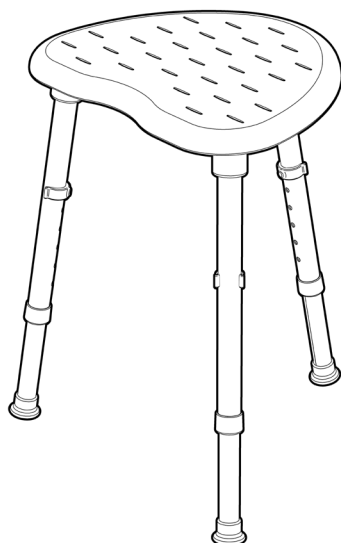

1.


ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

Modern / Functional / Compact design
Ideal for use in confined spaces
Easy to assemble - No tools required
Adjustable seat height
Three leg stability on uneven floors
Grip seating surface
Aluminium legs
Flexible PVC leg tips
Strong lightweight construction
Corrosion free materials
Life time guarantee on corrosion
Designed for easy cleaning

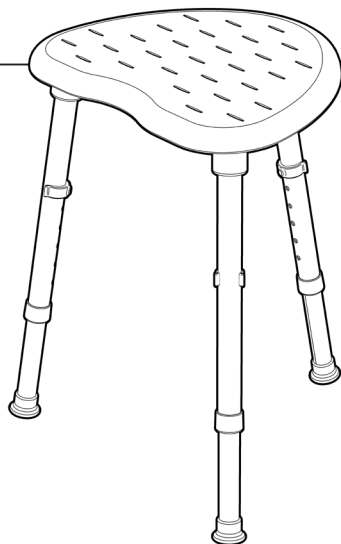

SPECIFICATIONS

(All dimensions shown are nominal)

Overall Width: **430mm** (at lowest setting)
Overall Depth: **360mm** (at lowest setting)
Overall Height: **490mm** (at lowest setting)
Seat Width: **395mm**
Seat Depth: **295mm**
Seat Height: **490 - 615mm: 6 settings** (floor to top of seat)
Hole 1: **490mm**
Hole 2: **515mm**
Hole 3: **540mm**
Hole 4: **565mm**
Hole 5: **590mm**
Hole 6: **615mm**

Product Weight: **1.5kg**


Maximum User Weight: **130kg**

Please contact **REDGUM Brand** if you would like any further information on your **Space Saver Corner Shower Stool**.

ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

PARTS:

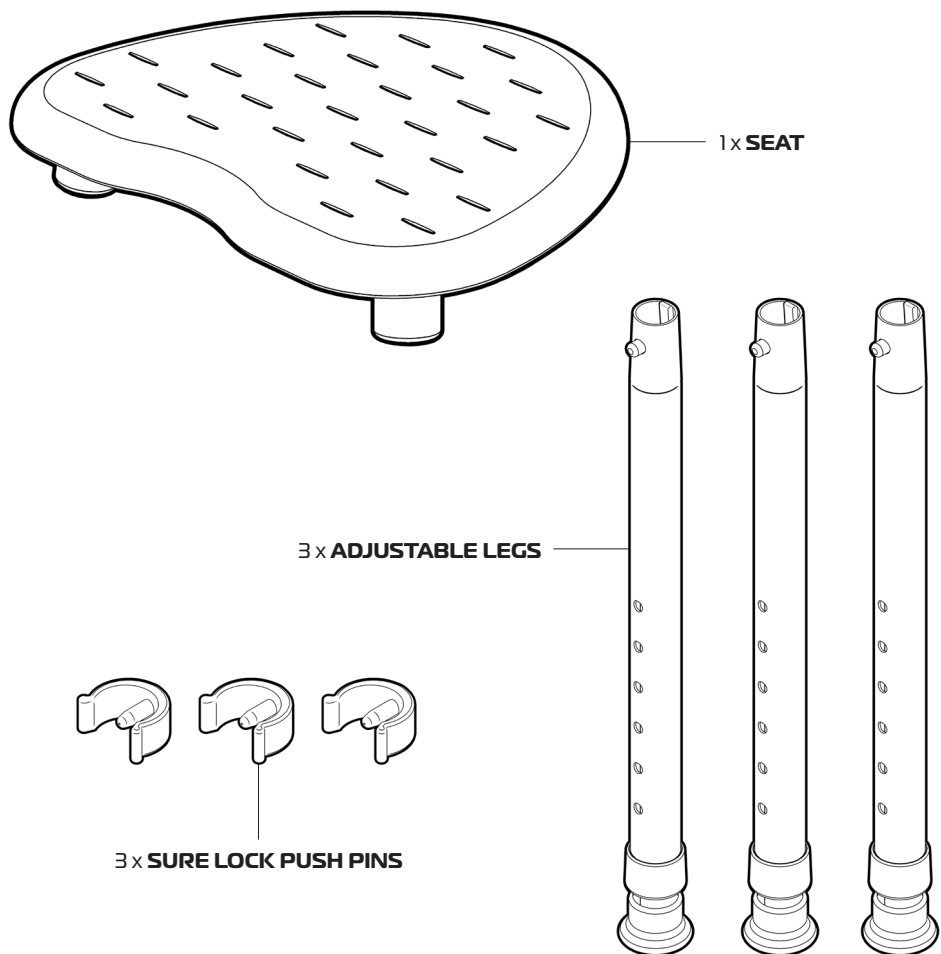

IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE YOUR PRODUCT** and contact your supplier for support.

3.


ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - LEG ASSEMBLY

Each adjustable leg needs to be assembled prior to fitting to seat:

1. Pull out the inner leg.
2. Line up the holes, re insert the leg at the required setting.
(Setting can be changed at any time).
3. Insert sure lock leg clip into hole and push clip until it locates around tube.
4. **Repeat this process on all 3 adjustable legs.**

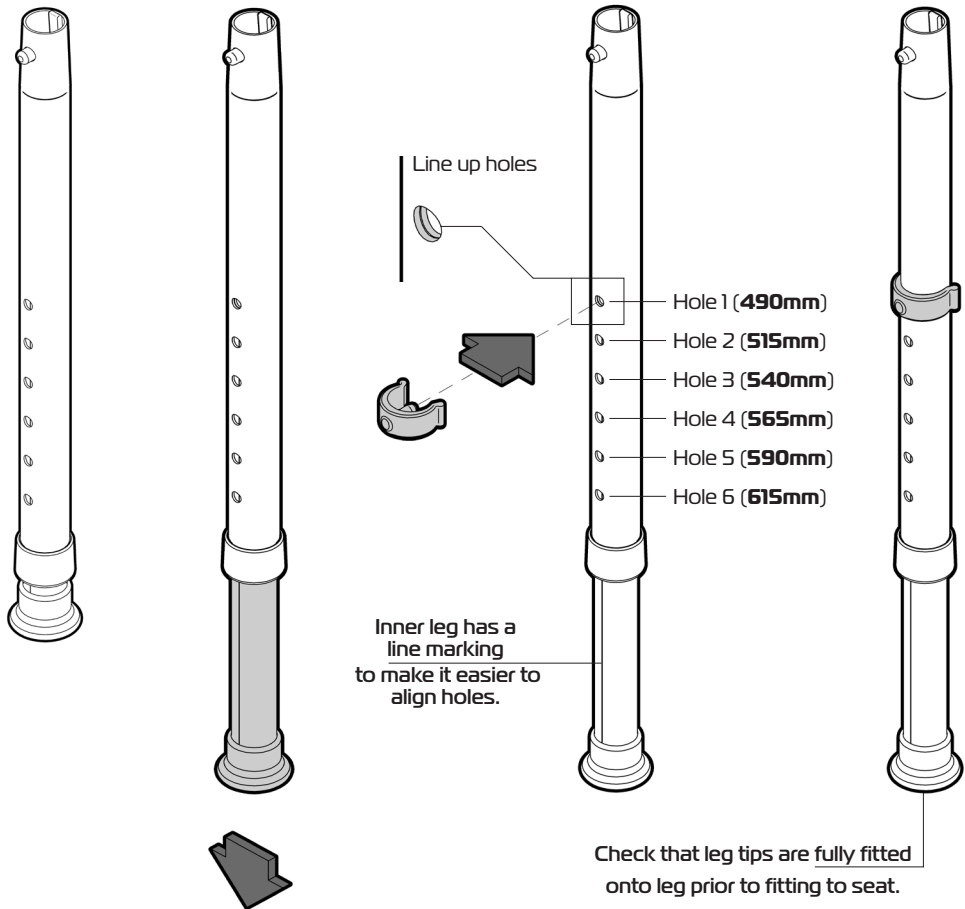

To adjust the legs after they have been fitted to seat:


1. Remove sure lock leg clip.
2. Adjust leg to required setting (line up holes).
3. Insert sure lock leg clip into hole and push clip until it locates around tube.

4.



ASSEMBLY - LEG INSTALLATION

Each adjustable leg needs to be installed to seat:




1. Place seat upside down as shown below.
2. Insert assembled leg into socket.
3. Line up red press button with hole in wall of socket.
4. Depress press button and push leg down into socket.
5. Ensure head of button protrudes out of hole.
6. **Repeat process for all 3 adjustable legs.**

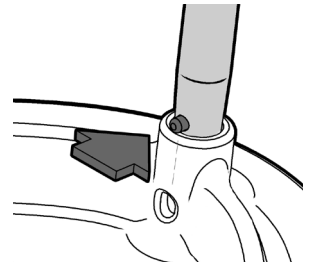

Line up press button with hole.

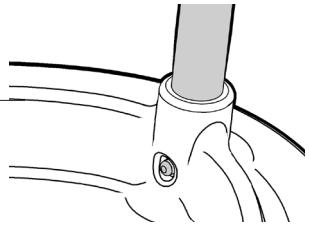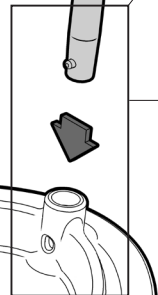

IMPORTANT NOTE:
Press button head **MUST** protrude out of hole.

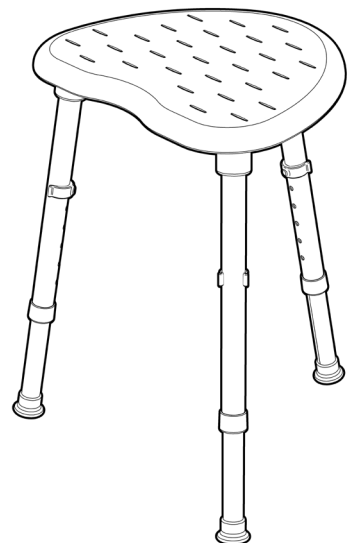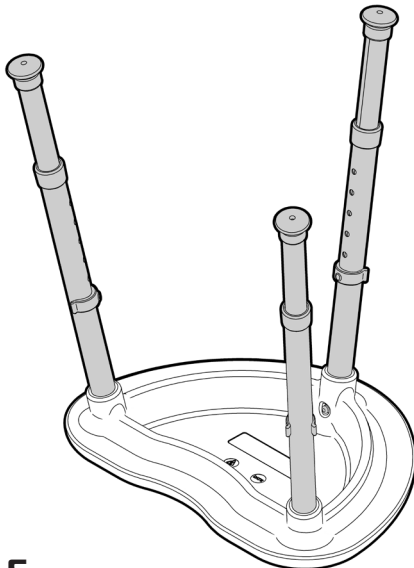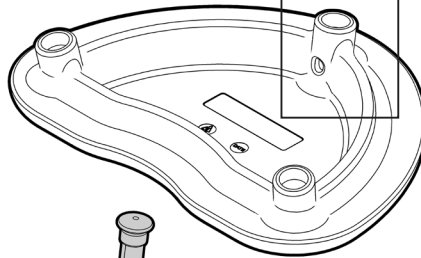

5.

ASK FOR THE BRAND

redgumbrand.com.au



USAGE GUIDELINES - DO's + DON'Ts

Please follow these guidelines to get the optimal benefit from your Space Saver Corner Shower Stool.

Failure to follow these guidelines may result in damage to the Space Saver Corner Shower Stool and put the user at risk.

130kg Maximum User Weight.

Ensure ALL THREE LEGS remain on ground at all times when Space Saver Corner Shower Stool is in use.

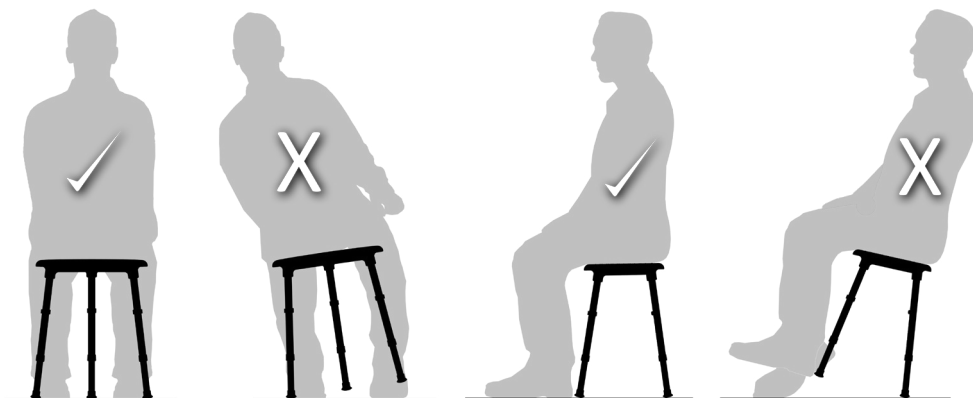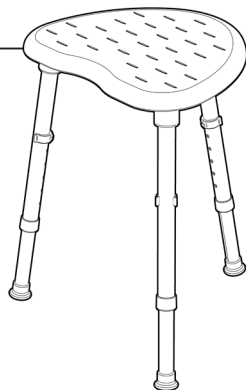

Do not lean too far to the side or backward causing any leg to leave the ground.


SITTING GUIDELINES

The Space Saver Corner Shower Stool can be set up in various ways to suit the individuals requirements.

Adjust the seat height to line up with back of the users knees. (Refer to Page 4)

To assist the user with getting on and off of the Space Saver Corner Shower Stool, extend the rear legs to open up the seat angle. (Max. 90°)

Product is now ready to use.

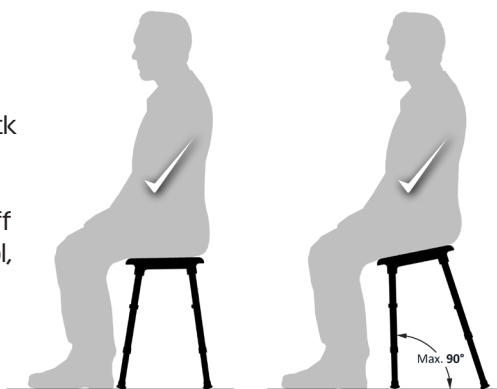

6.

PRODUCT CARE - INSPECTIONS / SERVICING / CLEANING

INSPECTIONS

We suggest that you visually check the Space Saver Corner Shower Stool periodically for signs of damage or wear prior to use. Overall check that no components are loose or worn.



Withdraw the Space Saver Corner Shower Stool immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.


CLEANING

We recommend that you attend to any required cleaning as soon as possible to reduce the possibility of staining.

Clean your Space Saver Corner Shower Stool using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the Space Saver Corner Shower Stool and should not be used.

If you have any questions about the cleaning of the product please contact your supplier or REDGUM brand direct.

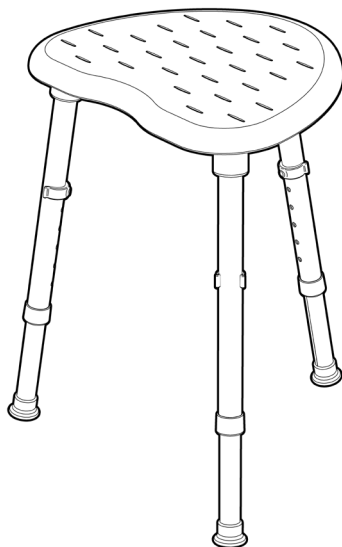

WARRANTY

Your REDGUM Brand **Space Saver Corner Shower Stool** (RG553) is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.


Do not modify the Space Saver Corner Shower Stool in any way or fit any parts or accessories that are not specified as an option for the stool. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the **Space Saver Corner Shower Stool** please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**


A: **1 Business Way, Malaga, Australia 6090**

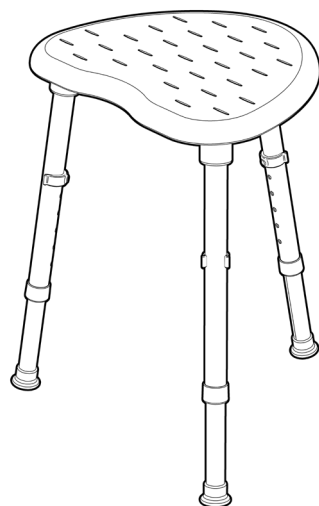

8.

ACCESSORIES


The Space Saver Corner Shower Stool can be fitted with an optional gel pad to offer pressure relief when sitting.

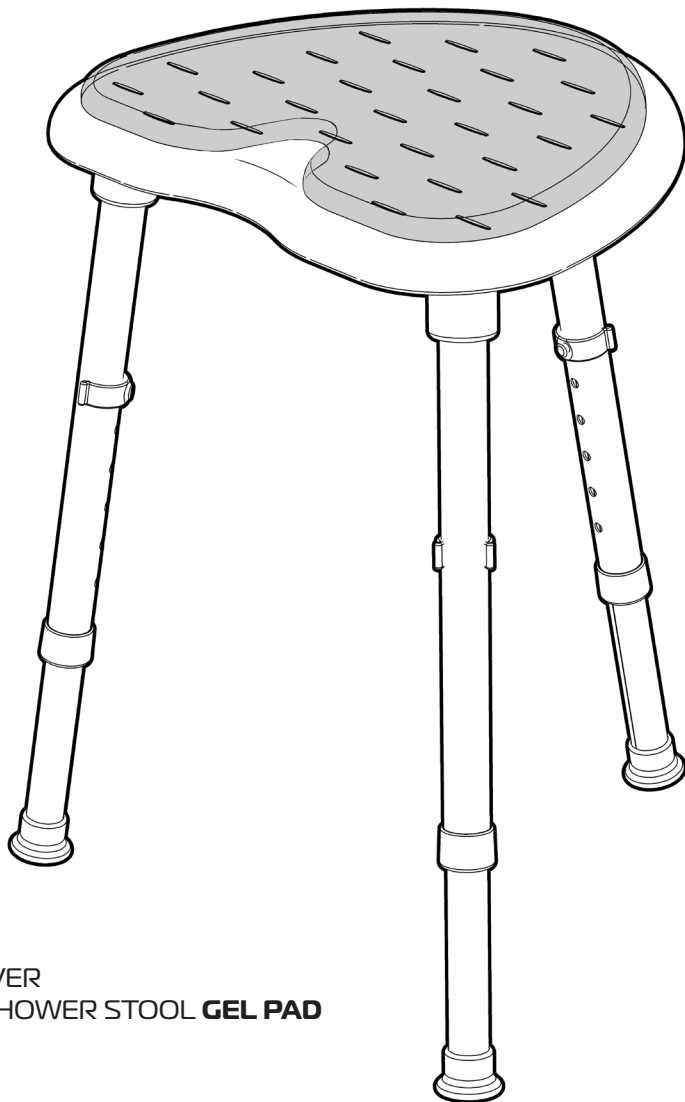

SPACE SAVER
CORNER SHOWER STOOL **GEL PAD**
RG53GP

SPACE SAVER RANGE

The Space Saver Corner Shower Stool is part of the Space Saver range.
Other products in this range include:

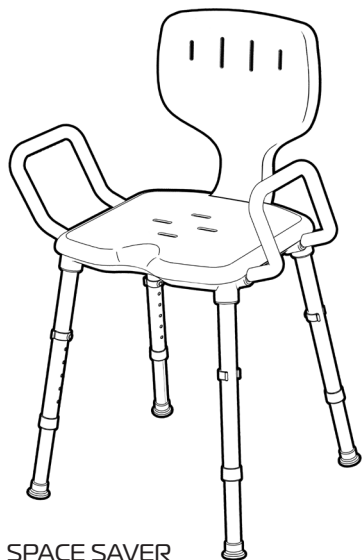

**SPACE SAVER
SHOWER CHAIR**
RG555H

Options:


Space Saver Shower Chair Gel Pad: RG55GP
Space Saver Caddy: RG55C
Space Saver Low Handles: RG5LHW
Space Saver Red Handles: RG5SHR
Space Saver Blue Handles: RG5SHB

Please contact your supplier
or **REDGUM Brand** direct for
further information on the
Space Saver range of products
and accessories.

**SPACE SAVER
SHOWER STOOL**
RG554H2

Options:

Space Saver Shower Stool Gel Pad: RG54GP
Space Saver Caddy: RG55C
Space Saver Low Handles: RG5LHW
Space Saver Red Handles: RG5SHR
Space Saver Blue Handles: RG5SHB

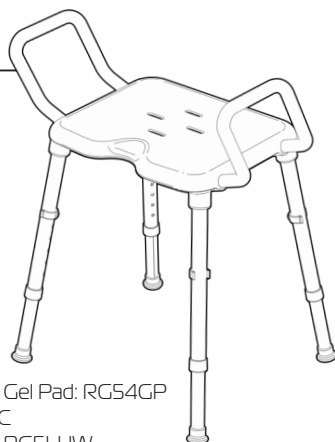

**SPACE SAVER
TOILET SEAT RAISER**
RG515

Options:

Space Saver Toilet Seat Raiser Gel Pad: RG51GP
Space Saver Bowl + Lid: RG5B
Space Saver Bowl Lid Holder: RG5BLH
Space Saver Caddy: RG55C
Space Saver Low Handles: RG5LHW
Space Saver Red Handles: RG5SHR
Space Saver Blue Handles: RG5SHB

Freestanding Toilet Roll Holder: RG5TRH

REDGUM Brand - Product Range

REDGUM Brand offer a comprehensive range of products including:

Seat Walkers

Side Folding Seat Walkers

Walking Frames

Wheelchairs

Walking Sticks + Quads

Crutches

Reachers

Shower / Bathroom products

Toilet products

Cushions: Air / Foam


Electric Lift Chairs

Electric Beds

Aluminium Ramps


Pedal Exercisers

Personal Alarms

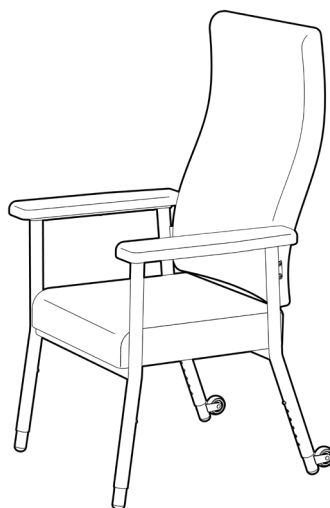

REDGUM Brand stock a large range of spare parts for all REDGUM brand products.

Please contact your supplier or **REDGUM Brand** direct for further information.

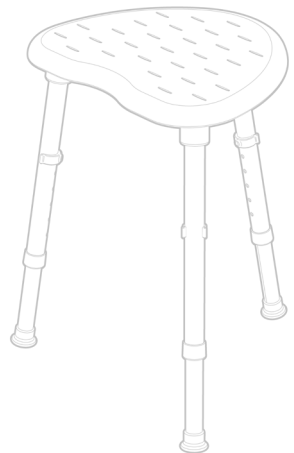

DISTRIBUTOR:

REDGUM **BRAND**
*Daily Living and
Lifestyle Products*



**Proudly
Australian Owned
and Operated**

ASK FOR THE BRAND

redgumbrand.com.au