


How to Enable Noxturnal Admin Mode and Remove Default Settings

! **Note:** This guide is for Internal Use Only

A. Change Noxturnal Properties to Admin Mode

Right Click on the Noxturnal Icon on your desktop



In the **Target location**, add the extension **“/admin”** to the end of the location path with a space in between and click **OK**.

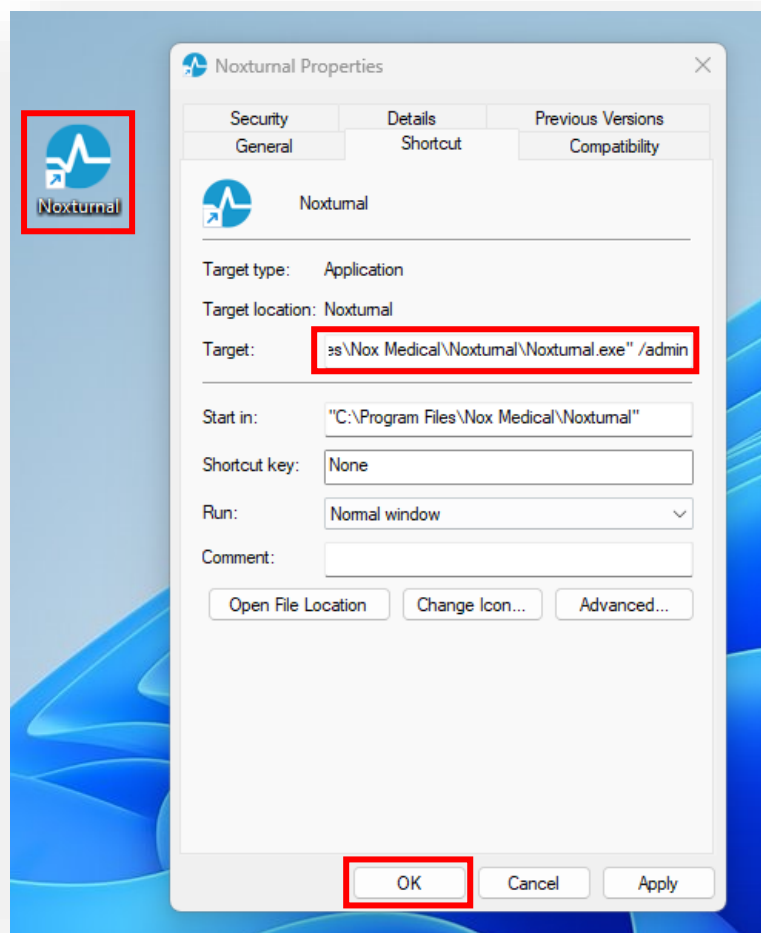

B. Run Noxturnal as Administrator

Right Click on the Noxturnal Icon and click on **“Run as Administrator”**.

Noxturnal should then launch in **Admin Mode** which is indicated on the top left of the software.

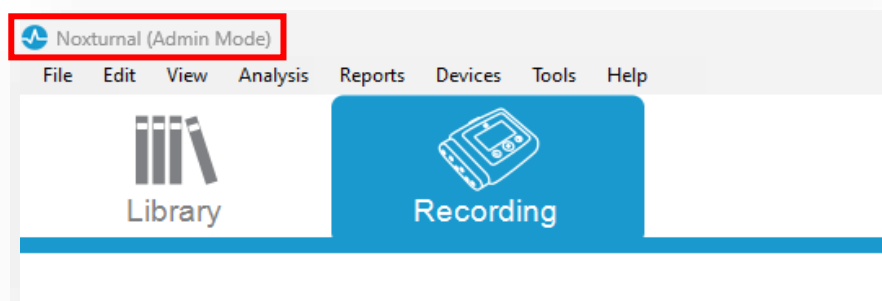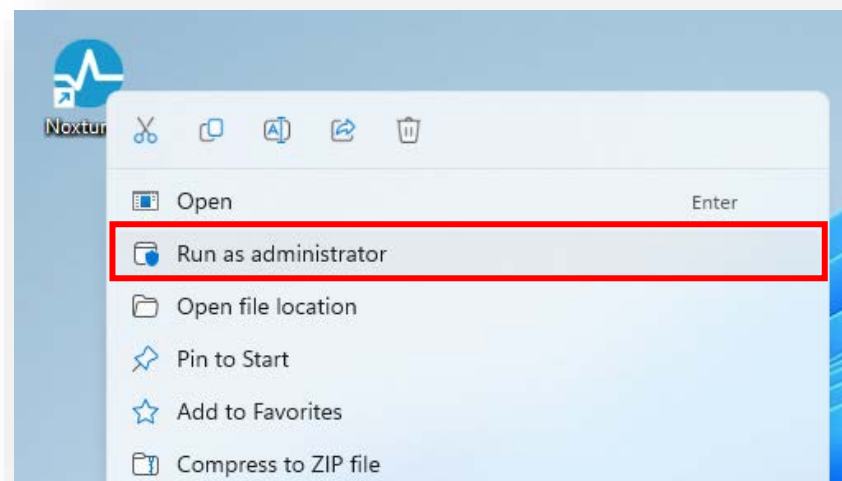

C. Delete Recording Types

In Admin mode, go to Devices >>> Recording Types

Under the Nox A1 (ambulatory) list, delete the following:

- Nox SAS
- Nox SAS without Audio
- Standard PSG
- Standard PSG without Audio

Highlight each one and select “Remove”

