

Instructions For Use

FlexiSEQ **MAX** STRENGTH

FOR OSTEOARTHRITIS

Read the whole leaflet carefully before you start using this product. Keep this leaflet for future reference. If you have any further questions, consult your doctor or pharmacist.

1. Composition

Aqua, Phosphatidylcholine, Alcohol, Glycerin, Carbomer, Polysorbate 80, Disodium Phosphate, Sodium Hydroxide, Benzyl Alcohol, Methylparaben, Ethylparaben, Linalool, Disodium EDTA, Sodium Phosphate, Sodium Metabisulfite, BHT.

2. Presentation and pack size

FlexiSEQ Max Strength is a white to yellow gel and comes in 30g, 50g and 100g tubes.

3. Intended purpose

FlexiSEQ Max Strength lubricates the cartilage in joints to relieve pain and stiffness and improve impaired joint function associated with all stages of osteoarthritis.

4. Contraindications, precautions and safety measures

- Do not use FlexiSEQ Max Strength if you are allergic (hypersensitive) to any of the ingredients.
- Do not use FlexiSEQ Max Strength on open wounds, broken or infected skin or mucous membranes (the lining of the openings of the body, such as the mouth, nose or eyes).
- In case of minor skin irritations such as dryness and redness at the site of application, reduce the frequency of application to once daily. If this does not improve the condition, you may discontinue application until the problem resolves, usually within a few days. You may then restart the application. If the problem persists then discontinue use.
- There are no known interactions with medicines.
- Keep out of the reach of children.
- FlexiSEQ Max Strength has not been tested on pregnant women or children.
- In the event of serious side effects, or if you notice any side effects not listed in this leaflet, please tell your doctor or pharmacist.



Temperature limitation
(store between 8°C and 25°C)



Use by (see imprint on the tube and carton)
Do not use the product after the expiry date



Caution



Batch number
(see imprint on the tube and carton)



Consult instructions for use



Non Sterile



PBB Devices Limited
4th Floor Reading Bridge House,
George Street,
Reading, Berkshire,
RG1 8LS,
UK



Date of manufacture
(see imprint on the carton)



NEXTEC medical GmbH, Zöllnplatz 4,
79410 Badenweiler, Germany

EU Importer:

Pro Bono Bio International Trading Limited,
Palazzo Pietro Stiges, 103 Strait Street,
Valetta, VLT1436, Malta

HOW TO GET THE BEST FROM FLEXISEQ MAX STRENGTH

Application	Drying Time	Frequency	Effect Time	Storage
 Spread a thin layer around the joint and do NOT rub it in. Allow to dry.	 Once applied, the correct amount should take roughly 10 minutes to air dry.	 You should look to apply FlexiSEQ Max Strength twice daily in the morning and evening.	 FlexiSEQ Max Strength can take anywhere between 2-14 days to start taking effect.	 Store at room temperature (not in the fridge) and use within three months of opening.

APPLYING FLEXISEQ MAX STRENGTH TO YOUR JOINTS

FlexiSEQ Max Strength is not like other gels. It is important to apply our gel correctly to joints to ensure it works effectively. It should NOT be rubbed into the skin. It should be allowed to dry completely.

Hands	Hips	Knees	Shoulders
 Apply a pea-sized amount of gel to each affected knuckle and spread evenly around the joint. Avoid washing hands for 10-15 minutes.	 Locate the edge of your hip bone and apply a line of gel as long as your index finger to the soft tissue below, where the bone can no longer be felt.	 Apply a line of gel as long as your index finger to the back and sides of your knee joint. It should not be applied to the kneecap.	 Apply a line of gel as long as your index finger to the area corresponding to your pain. It should not be applied to the bone or shoulder blade.

WELCOME TO THE FLEXISEQ MOVEMENT

We're delighted to welcome you to the FlexiSEQ family. Our products are designed to help you move without pain and to give you the freedom to say 'yes' to life.

Our help and support doesn't end with our range of products. Visit our website to sign up to our free FlexiSEQ newsletter and each month we'll send you updates from our amazing community and news of our latest promotions. If you have any questions, comments or feedback, you can also find us on social media or feel free to contact us at enquiries@flexiseq.com

Together we are stronger, that's what the #FlexiseqMovement is all about.

FIND OUT MORE AT
www.flexiseq.com



WHAT IS FLEXISEQ MAX STRENGTH?

FlexiSEQ Max Strength is a topically applied gel specially formulated to help fight against the pain and stiffness associated with osteoarthritis.

Clinically proven to relieve joint pain and improve mobility, FlexiSEQ Max Strength uses unique SEQ Tech™ to help lubricate and protect your joints.

It is also drug free and can be used long-term and alongside medications without drug interactions or exacerbating pre-existing conditions.

HOW FLEXISEQ MAX STRENGTH WORKS

FlexiSEQ Max Strength helps to relieve pain and improve joint mobility thanks to billions of plant-based, oily droplets called Sequessomes™.

Activated when our gel dries on the skin, these tiny, flexible droplets pass through the skin and into target joints, where they localise on cartilage surfaces to form a lubricating layer.

We call our unique lubricating mode of action SEQ Tech™ and it's the revolutionary driving-force powering FlexiSEQ Max Strength.

No drugs, no heating or cooling, FlexiSEQ Max Strength is a one-of-a-kind treatment for joint pain and stiffness.

THE BENEFITS OF A DRUG-FREE PRODUCT

As a drug-free treatment, FlexiSEQ Max Strength has several added benefits for the following osteoarthritis patients:

<p>Patients looking for a drug-free formula with reduced systemic side effects.</p>	<p>Patients who need treatment that can be used longer than 14 days.</p>	<p>Patients for whom NSAIDs are not suitable.</p>	<p>Patients taking medications that may interact with traditional painkillers.</p>	<p>Children, pregnant and breastfeeding women should seek the advice of a doctor.</p>
---	--	---	--	---

OUR FLEXISEQ MAX STRENGTH RANGE

Our Max Strength formula comes in three sizes:

- 30g - perfect if you are on the go
- 50g - our best-seller
- 100g - designed for regular users

Patents:			
Europe	EP2437726, validated in: AT, BE, CH, CZ, DE, DK, EE, ES, FI, FR, GB, IE, IT, LT, LU, LV, MT, NL, NO, PL, PT, SE, SK, TR	Singapore	176297
GB	See: www.ascension.co.uk/patents	South Africa	2012/00017
Malaysia	MY-170121-A		
Trademarks:			
Flexiseq®, Sequessome Technology®, Sequessomes™, SEQ Tech™ and the SEQ TECH™ roundel are trademarks of Ascension Healthcare PLC or its subsidiaries			