

RELIWASH

Redcap

A cost-effective acid and alkali treatment which is more effective than water.

Independent test data has confirmed that one bottle of Redcap 500ml is significantly more effective at neutralising Acid & Alkali splashes than standard water treatment of the same volume. Progressive dilution tests comparing the pH neutralising effect of Redcap against that of deionized water also shows Redcap continues to neutralise Acid & Alkali pH levels long after water slows



RECOMMENDATIONS FOR USE


Redcap is a first aid treatment for accidents involving acids and alkali. Rinsing with Redcap will help to neutralise the harmful pH values of both acids and alkali. Special care should be taken when treating incidents involving the eyes to ensure all fluids run off the face and do not pool in the ears nose or mouth. It is recommended to use up to 2 litres of Redcap solution, after which, you should continue to rinse with Reliwash Eye Wash until the emergency services arrive.

NOTE: All eye related injuries require medical attention.


HYDROFLUORIC ACID NOTICE

Redcap is not recommended for the treatment of Hydrofluoric acid incidents. Hydrofluoric acid is toxic and highly corrosive to skin and bones. Redcap may have a neutralizing effect and help to rinse but will not limit the acids corrosive effect. Dedicated Hydrofluoric Acid Treatment kits are available for customers who have a first aid requirement for Hydrofluoric Acid incidents.

Acid Effectiveness (Sulphuric Acid)

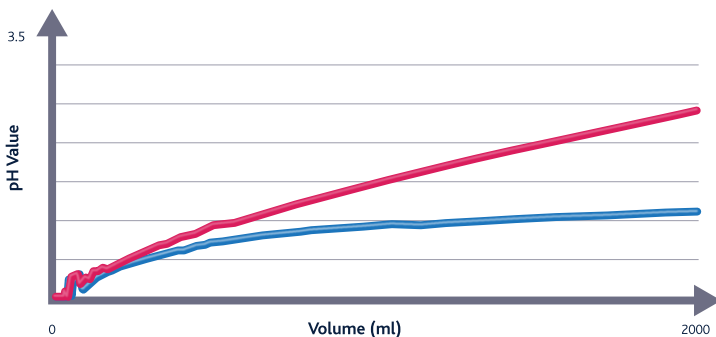

Alkali Effectiveness (Sodium Hydroxide)

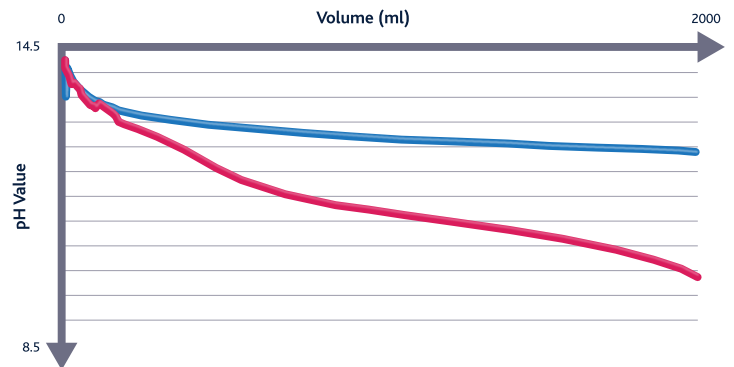

In vitro titration studies were independently conducted to assess the effectiveness of Redcap. Sulphuric Acid (96%) and Sodium Hydroxide (40%) were identified as the most commonly treated acid and alkali related injuries. 10ml of each substance were pipetted into a conical flask and placed in a magnetic stirrer. Initial pH values were recorded. Redcap was added in progressive increments, with the pH value recorded after each increment. Further test reports on the effectiveness of Redcap are available upon request.

Reliwash Redcap is available in a range of solutions to ensure you always have an effective Acid and Alkali treatment to hand when it is required:



Chemical Splash Eye Wash Station

Neutralising effect, reduces flushing time and volume of fluid needed to dilute acid and alkali splashes. Luminous, printed instructions. Bottle includes integral eye bath. Eye wash for acid and alkali.

CONTENTS

- 2 Dressings, Eye Pad
- 5 Eye Wash, Pods 20ml
- 2 Eye Wash Redcap™, Bottles 500ml
- 1 Mirror

5996 Chemical Splash Station Red/Clear Lid
36cmH x 51cmW x 10cmD

5995 Chemical Splash Station - empty Red/Clear Lid
36cmH x 51cmW x 10cmD



Eye Wash and Skin Flush Kit

Neutralising effect, reduces flushing time and volume of fluid required to dilute acid and alkali splashes.

CONTENTS

- 2 Dressings, Eye Pad
- 2 Redcap Eye Wash, Bottles 500ml
- 2 Eye Wash, Pods 20ml
- 1 Mirror

5970 Eye Wash and Skin Flush Kit
in Red/Orange Integral Aura Box
27cmH x 29.5cmW x 10cmD



Spectra First Aid System

Neutralising effect, reduces flushing time and volume of liquid required. Colour-coded for easy recognition. A practical, convenient and innovative system. Interlocks with other Spectra first aid systems.

CONTENTS

- 1 Accident Book
- 1 Sign 'Chemical System Guidance'
- 1 Eye Wash and Skin Flush First Aid Kit
- 1 Hydrofluoric Acid Treatment Kit

9201 Chemical Splash First Aid System
75cmH x 34.7cmW x 10.5cmD



Redcap Eye Wash and Skin flush

Purified water 95.1%. Phosphate buffered solution. Eye Wash and skin flush for acid and alkali.

5990 Reliwash Redcap 500ml Bottle with Eye Cap
Sterile
500ml Bottle

5991 Reliwash Redcap 500ml Bottle
Sterile
500ml Bottle

