



Applicable to AcoSound Sonar Series hearing amplifier



AcoSound SonarSe Hearing Amplifier  
《 User Manual 》

## How to put on the Sonar hearing amplifier



Before wearing a hearing amplifier, please clean the ear canal, gently buckle the ear cap direction of the hearing amplifier into the ear canal

\*Definition of touching key:

### (1) ON/OFF

Hearing amplifier needs to be turned on separately


1. Press and hold 2-3s to power on with toot four times

Put the hearing amplifier back into the charging box, automatically turn off

### (3) volume adjustment

1. Maximum volume, system prompts "beep beep"  
2. Minimum volume, system prompts "toot toot"  
Press and sound one "toot" each time to switch volume (1-5 volume cycle)

\*Volume cycle 1-5: From level 1 to 5 then back to 1

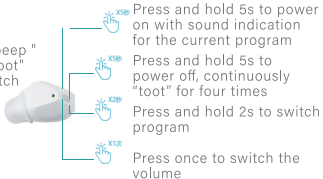

After wearing, it can be rotated to adjust and fit into a comfortable position in the ear canal and keep it stable

### (2) Program files

Double-click the touching area to select the appropriate mode

1. Full frequency amplification, toot once

2. Medium and high-frequency amplification, toot twice

3. Medium frequency amplification, toot three times

4. Low-frequency amplification, toot four times

## Program Files And Function Information

Program files	Input mode, function features
Program 1 (P1)	Full frequency amplification, toot nice
Program 2(P2)	Medium and high frequency amplification toot twice
Program 3(P3)	Medium frequency amplification, toot three times
Program 4(P4)	Medium and low frequency amplification tout four times
Lock function	Adjust the hearing amplifier to the most comfortable volume and mode for yourself, long press the touch area for 5 seconds to lock the mode and volume (any operation or misoperation after locking will be invalid)
Unlock Function	Long press the touch area for 5 seconds to unlock and and adjust