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**Hi, I'm Greta<sup>EN</sup>**

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**Hallo, ich bin Greta<sup>DE</sup>**

---

**Instrucciones de uso. Instructions of use.**

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# Instructions<sup>EN</sup>

## Product purpose:

Greta is a menstrual cup that serves to retain the flow of menstruation. It is designed to be reusable and has a useful life of up to 10 years. It is durable, respectful of intimate flora and the environment, and more economical in the long term. It can be used for 12 hours in a row, allowing you to carry out your daily activities without hindrance. It should only be taken out prior to sexual intercourse with penetration.

## Instructions for proper use:

- To use it for the first time, boil the glass for 10 to 15 minutes. Before each use, boil for 3 to 5 minutes.
- Wash your hands and find a comfortable position to insert the cup into the vagina. It is advisable to fold it to reduce the size, so it will enter more easily. When it causes no discomfort and remains still if you twist the end, it means it is well positioned.
- To remove the cup, gently insert your fingers and pinch the cup to break the vacuum. It is important never to remove the cup by pulling directly on the end without first breaking the vacuum.
- Empty the glass and wash it with neutral soap and water.
- It is advisable to empty it every 4-8 hours, depending on the amount of bleeding, although it can be worn for up to 12 hours.
- After using it, boil it for 3 to 5 minutes and store it in its bag, away from bacteria and dirt.

## Composition:

Made from high quality hypoallergenic silicone, free from latex and phthalates.

## Possible risks associated to their use:

- This product is for personal use. Do not share it to prevent the spread of infection.
- Do not use silicone or oil-based lubricants, as they can harm the product's silicone or leave it feeling permanently slimy.
- It is important that no abrasives or harsh disinfectants containing petroleum-based substances, tea tree oil, alcohol or bleach are used to clean the product.
- For proper maintenance of the product, it is recommended to store it in a dry place, away from dust and direct sunlight.
- It is recommended to store the product in an antibacterial bag.
- Keep out of reach of children.

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