

Fingertip Pulse Oximeter

iP900AP



USER MANUAL Ver1.0

General Description


SpO₂ stands for peripheral capillary oxygen saturation. Oxygen saturation is defined as the ratio of oxyhemoglobin (HbO₂) to the total concentration of hemoglobin (i.e. Oxyhemoglobin + reduced hemoglobin) present in the blood. It is an important physiological parameter involved in respiration and circulation. The Pulse Oximeter feature herein is small, portable, non-invasive and easy to use. The user only needs to insert a finger into the chamber to measure his/her SpO₂ level and pulse rate. It has also been proven to be highly precise and reliable in clinical tests.

Measurement Principle

Oxygenated blood absorbs light preferentially at 905nm (near infrared light), whereas deoxygenated blood absorbs light preferentially at 660nm (red light). A pulse oximeter works by passing a beam of red and infrared light through a pulsating capillary bed and then measure the amount of red and infrared light emerging from the tissues via a sensor. To improve accuracy, the iP900AP uses a proprietary algorithm to collect data from pulsatile arterial blood and excludes noise from the tissues. The relative absorption of light by oxyhemoglobin (HbO) and deoxyhemoglobin is then calculated according to the Beer-Lambert's law and a quantitative measurement of the users' oxyhemoglobin status i.e. Oxygen saturation level (SpO₂) is derived.

Diagram of Operation Principle

1. Red and Infrared-ray Emission Tube
2. Sensor

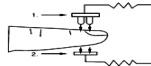

Precautions For Use

- 1 Please read the manual carefully before use.
- 2 Do not use the fingertip pulse oximeter in an MRI or CT environment.
- 3 This device is not for continuous monitoring.
- 4 Do not use the fingertip pulse oximeter in an explosive environment.
- 5 In order to ensure proper sensor alignment and skin integrity, the maximum application time at a single site for the device should be less than half an hour.
- 6 Do not sterilize the device using autoclaving, ethylene oxide sterilizing, or immersing the device in liquid. The device is not intended for sterilization.
- 7 Follow local ordinances and recycling instructions regarding disposal or recycling of the device and device components, including batteries.
- 8 This equipment complies with IEC 60601-1-2:2007 for electromagnetic compatibility for medical electrical equipment and/or systems. However, because of the proliferation of radio-frequency transmitting equipment and other sources of electrical noise in healthcare and other environments, it is possible that high levels of such interference due to close proximity or strength of a source might disrupt the performance of this device.
- 9 Portable and mobile RF communications equipment can affect medical electrical equipment.
- 10 Do not disassemble, repair or modify the equipment.
- 11 It may be unsafe to:
 - use accessories, detachable parts and materials not described in the instructions for use
 - interconnect this equipment with other equipment not described in the instructions for use
 - disassemble, repair or modify the equipment
- 12 The medical silicone and ABS plastic enclosure which contact the user's skin when the device is used have been assessed by and passed the ISO10993-5 Tests for in vitro cytotoxicity and ISO10993-10 Tests for irritation and delayed-type hypersensitivity.
- 13 The fingertip pulse oximeter is dependent on blood flow to obtain an accurate SpO₂ measurement. Verify that nothing is hindering your blood flow before taking your SpO₂ readings.
- 14 **This device is not intended to diagnose or treat any medical condition or disease. It is intended for non-medical use by healthy people to monitor their pulse rate and blood oxygen levels. It is for sports and/or aviation use. People who need SpO₂ and pulse rate measurements because of a medical condition should not use this pulse oximeter and should consult with their physician.**

Inaccurate measurements may be caused by

- 1 Significant levels of dysfunctional hemoglobin (such as carboxyl - hemoglobin or methemoglobin).
- 2 Intravascular dyes such as indocyanine green or methylene blue.
- 3 High ambient light. Shield the sensor area if necessary.
- 4 Excessive user movement.
- 5 High-frequency electrosurgical interference and defibrillators.
- 6 Venous pulsations.
- 7 Placement of a sensor on an extremity with a blood pressure cuff, arterial catheter, or intravascular line.
- 8 The user has hypotension, severe vasoconstriction, severe anemia, or hypothermia.
- 9 The user is in cardiac arrest or is in shock.
- 10 Fingernail polish or artificial fingernails.
- 11 Weak pulse quality (low blood perfusion).
- 12 Low hemoglobin.

Product Features


- 1 Easy to operate. Measure your SpO₂ non-invasively.
- 2 Small, light and portable.
- 3 OLED screen displays SpO₂, Pulse Rate, Pulse bar, Plethysmograph and Perfusion Index (PI). Six different display options.
- 4 Level 1-10 adjustable brightness.
- 5 Preset alarm function.
- 6 2pcs AAA-size alkaline batteries; low-battery indicator and low power consumption.
- 7 When no or low signal is detected, the screen will display 'Finger Out' and the device will power off automatically in about 8 seconds.

Intended Use

The iP900AP Fingertip pulse oximeter is a portable non-invasive device intended for spot-checking of oxygen saturation of arterial hemoglobin (SpO₂) and pulse rate of children (>12 years old) and adult. The device is for sports and/or aviation use.

Operation Instructions

- 1 Install two AAA batteries according to the Battery Installation instructions.
- 2 Place one of your fingers into the finger chamber of the pulse oximeter.
- 3 Press the power button on the front panel to turn the pulse oximeter on.
- 4 Keep your hand and finger still for the reading. It is recommended that you do not move your body while taking a reading.
- 5 Read the data from the display screen.
- 6 Press (<1 sec) the button again to toggle between six display modes (see below).

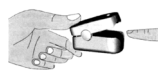

After turning the oximeter on, each time you press the power button, the oximeter will switch to another display mode. There are 6 display modes shown as follows:



Tips to getting a good reading


- 1) Make sure that your finger is inserted deep into the chamber so that the fingertip is placed directly in between the LED sensor and the LED light source.
- 2) Avoid making any body movement, especially your finger while taking measurement.
- 3) Long fingernails may obstruct the light sensor and prevent accurate measurement. Please keep the fingernails short while using the device.
- 4) Excessive ambient infra-red light, especially in an overly bright lit room, can interfere with the sensor, preventing an accurate

measurement.

5) Poor blood circulation can affect oximeter readings. Warm your hands and fingers before taking your measurements. Note that the pulse oximeter is measuring your SpO₂ and PR based on your blood flow. If the blood flow in your finger drops below a perfusion index of 0.3, the pulse oximeter will not be able to get a reading.

6) Some people with medical conditions such as anemia, hypotension and hypothermia may experience an inaccurate reading during use. In such case, we suggest that you consult a physician.

Front Panel

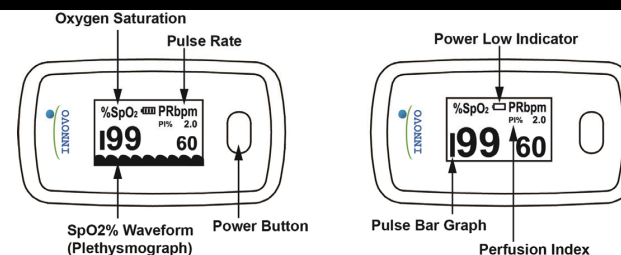

The pulse bar graph, plethysmograph and perfusion index (PI) are useful features that can be used to determine the reliability of a reading. If the height of the pulse bar is less than 30%, this indicates signal inadequacy and the displayed SpO₂ or pulse rate value is potentially incorrect. Adjust your finger so that it is directly between the LED lights and sensor.

The plethysmograph indicates the amount of blood flow detected by the pulse oximeter and the perfusion index indicates the strength of your pulse. Each wave in the plethysmograph corresponds to a heartbeat and the wave amplitude corresponds to the amount of blood detected by the pulse oximeter. The pulse oximeter is optimized when the height of the wave amplitude is consistent throughout as shown in the figure above. That is when you should take the reading. The pulse oximeter can function with a PI reading as low as 0.3%. If your PI is below 0.3%, this means that your blood perfusion is too low for a reliable read. Warm your hands to increase blood flow and retake your measurement. In general, a higher PI will give you a more reliable reading.

Settings

In the measuring interface, press and hold the power button to enter the settings interface. To scroll down the menu, short press (<1 sec) the power button. The ► icon indicates the line selected. To select an option, long press (>1 sec) the power button.

Settings	Settings menu																												
<table border="1"> <thead> <tr> <th colspan="2">Settings</th> </tr> </thead> <tbody> <tr> <td>► Alm Setup</td> <td></td> </tr> <tr> <td>Alm</td> <td>Off</td> </tr> <tr> <td>Beep</td> <td>Off</td> </tr> <tr> <td>Brightness</td> <td>4</td> </tr> <tr> <td>Reset</td> <td></td> </tr> <tr> <td>Exit</td> <td></td> </tr> </tbody> </table>	Settings		► Alm Setup		Alm	Off	Beep	Off	Brightness	4	Reset		Exit		<table border="1"> <tbody> <tr> <td>Alm Setup</td> <td>To enter the Alarm Setup submenu</td> </tr> <tr> <td>Alm</td> <td>To turn audible alarm on/off</td> </tr> <tr> <td>Beep</td> <td>To turn audible beep for pulse beat on/off</td> </tr> <tr> <td>Brightness</td> <td>To adjust screen brightness from 1-10. Level 10 is the brightest. Default level is 4.</td> </tr> <tr> <td>Reset</td> <td>To restore default settings</td> </tr> <tr> <td>Exit</td> <td>To return to the measuring interface</td> </tr> </tbody> </table>	Alm Setup	To enter the Alarm Setup submenu	Alm	To turn audible alarm on/off	Beep	To turn audible beep for pulse beat on/off	Brightness	To adjust screen brightness from 1-10. Level 10 is the brightest. Default level is 4.	Reset	To restore default settings	Exit	To return to the measuring interface		
Settings																													
► Alm Setup																													
Alm	Off																												
Beep	Off																												
Brightness	4																												
Reset																													
Exit																													
Alm Setup	To enter the Alarm Setup submenu																												
Alm	To turn audible alarm on/off																												
Beep	To turn audible beep for pulse beat on/off																												
Brightness	To adjust screen brightness from 1-10. Level 10 is the brightest. Default level is 4.																												
Reset	To restore default settings																												
Exit	To return to the measuring interface																												
<table border="1"> <thead> <tr> <th colspan="2">Alm Setup</th> </tr> </thead> <tbody> <tr> <td>► +/-</td> <td>+</td> </tr> <tr> <td>SpO₂ Alm Hi</td> <td>100</td> </tr> <tr> <td>SpO₂ Alm Lo</td> <td>90</td> </tr> <tr> <td>PR Alm Hi</td> <td>100</td> </tr> <tr> <td>PR Alm Lo</td> <td>60</td> </tr> <tr> <td>Exit</td> <td></td> </tr> </tbody> </table>	Alm Setup		► +/-	+	SpO ₂ Alm Hi	100	SpO ₂ Alm Lo	90	PR Alm Hi	100	PR Alm Lo	60	Exit		<table border="1"> <thead> <tr> <th colspan="2">Alm Setup submenu</th> </tr> </thead> <tbody> <tr> <td>+/-</td> <td>Select + to increase value. Select - to decrease value.</td> </tr> <tr> <td>SpO₂ Alm Hi</td> <td>Upper limit for SpO₂ alarm (71% - 100%)</td> </tr> <tr> <td>SpO₂ Alm Lo</td> <td>Lower limit for SpO₂ alarm (70% - 99%)</td> </tr> <tr> <td>PR Alm Hi</td> <td>Upper limit for PR alarm (35bpm - 250bpm)</td> </tr> <tr> <td>PR Lo</td> <td>Lower limit for PR alarm (30bpm - 245bpm)</td> </tr> <tr> <td>Exit</td> <td>To return to the Settings menu</td> </tr> </tbody> </table>	Alm Setup submenu		+/-	Select + to increase value. Select - to decrease value.	SpO ₂ Alm Hi	Upper limit for SpO ₂ alarm (71% - 100%)	SpO ₂ Alm Lo	Lower limit for SpO ₂ alarm (70% - 99%)	PR Alm Hi	Upper limit for PR alarm (35bpm - 250bpm)	PR Lo	Lower limit for PR alarm (30bpm - 245bpm)	Exit	To return to the Settings menu
Alm Setup																													
► +/-	+																												
SpO ₂ Alm Hi	100																												
SpO ₂ Alm Lo	90																												
PR Alm Hi	100																												
PR Alm Lo	60																												
Exit																													
Alm Setup submenu																													
+/-	Select + to increase value. Select - to decrease value.																												
SpO ₂ Alm Hi	Upper limit for SpO ₂ alarm (71% - 100%)																												
SpO ₂ Alm Lo	Lower limit for SpO ₂ alarm (70% - 99%)																												
PR Alm Hi	Upper limit for PR alarm (35bpm - 250bpm)																												
PR Lo	Lower limit for PR alarm (30bpm - 245bpm)																												
Exit	To return to the Settings menu																												

Setting the alarm parameters

When SpO₂ or PR reading exceeds a defined range, an alarm will trigger and the SpO₂ or PR reading will start to flash respectively. In addition, an audible alarm can be turned on by setting the 'Alm' under the Settings menu to on.

The upper and lower limit for the SpO₂ and PR alarm can be user defined. Under the Settings menu, select 'Alm Setup' to enter the Alarm Setup submenu. To increase a value, '+' should be selected. To decrease a value, '-' should be selected. SpO₂ limit can be adjusted in increment of 1%; PR limit in increment of 5bpm.

Default limits: SpO₂ Alm High 100%, Low 90%; PR Alm High 100bpm, Low 60bpm

Note: Audible alarm can be snoozed/silenced for 30 seconds by pressing the power button. Visual alarm will remain. After 30 seconds, if reading still exceeds the set limit, the audible alarm will turn back on.

Beep - An audible beep to indicate each pulse beat can be turned on by setting the 'Beep' under the Settings menu to on.

Brightness - The screen brightness can be adjusted from 1-10. Level 10 is the brightest. Default level is 4.

Reset - To restore default parameters setting, press and hold the power button until 'Yes' appears.

Exit - Press and hold the power button to return to the measuring interface.

Note: In the settings interface, if no operation is detected, device will return to the measuring interface automatically in about 8 seconds.

Product Accessories

1. One pulse oximeter
2. One lanyard
3. Two AAA batteries
4. One instruction manual

Battery Installation

1. Open the battery door cover.
2. Install two AAA batteries into the battery compartment by matching the plus (+) and minus (-) signs in the compartment. **Note:** Incorrectly installed batteries may damage the device.
3. Align the battery door cover and press until it snaps back in place.

Note:


- ✧ Please remove the batteries if the pulse oximeter will not be used for a long period of time.
- ✧ Please replace the batteries when the low power indicator appears.

Using the Lanyard

1. Thread the thin end of the lanyard through the lanyard hole on the device.
2. Thread the thick end of the lanyard through the thin loop (threaded in step 1) and pull to tighten.

Warnings!

- ✧ Keep the oximeter away from young children. Small items such as the battery door, battery, and lanyard are choking hazards.
- ✧ Do not hang the lanyard from the device's electrical wire.
- ✧ Please note that the lanyard tied to the oximeter may cause strangulation due to its length.

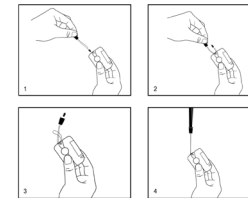

Maintenance and Storage

1. Replace the batteries when the low power indicator appears.
2. Clean the finger chamber and surface of the oximeter before use.
3. Remove the batteries if you are not planning to use the oximeter for a long period of time.
4. Store the oximeter in a cool and dry place. Extreme moisture may damage the oximeter or affect its lifespan.
5. Dispose battery properly. Follow any applicable local battery disposal laws.

Cleaning the fingertip pulse oximeter

It is recommended to clean the oximeter before and after use. To clean, use a soft cloth lightly dampened with water to wipe the finger chamber and the surface of the oximeter. Allow the oximeter to dry thoroughly before use.

CAUTION: Do not pour or spray liquids onto the oximeter, and do not allow any liquid to enter any openings in the device.

Disinfecting the fingertip pulse oximeter

The oximeter can be disinfected as needed. To disinfect, use a soft cloth lightly dampened with 70% isopropyl alcohol. Other recommended disinfectants include: 70% ethanol or glutaraldehyde-type 2% liquid disinfectants.

CAUTION: Do not use EtO (Ethylene oxide) or formaldehyde for disinfection.

The lifespan of the pulse oximeter is estimated to be five years if 15 measurements are taken per day and each measurement takes 10 minutes. Stop using and contact local service center if one of the following incidents occurs:

- An error in the *Possible Problems and Solutions* is displayed on the screen.
- The oximeter cannot be powered on even though new batteries are installed.
- There is a crack on the oximeter or damage to the display resulting in unidentifiable readings.
- The spring is defective.
- The button is unresponsive.

The pulse oximeter is factory calibrated. You do not need to calibrate it again during its lifespan.

Specifications

1. Display Type

OLED display

2. SpO₂

Display range: 0%~100%

Measurement range: 70%~100%

Accuracy: 70%~100% ± 2%; 0%~69% no definition

Resolution: 1%

A functional tester cannot be used to assess the accuracy of a pulse oximeter monitor or sensor. Clinical testing is used to establish the SpO₂ accuracy. The measured arterial hemoglobin saturation value (SpO₂) of the sensors is compared to arterial hemoglobin oxygen (SaO₂) value, determined from blood samples with a laboratory CO-oximeter. The accuracy of the sensors in comparison to the CO-oximeter samples measured over the SpO₂ range of 70%~100%. Accuracy data is calculated using the root-mean-squared (Arms value) for all subjects, per ISO 9919:2005, Medical Electrical Equipment - Particular requirements for the basic safety and essential performance of pulse oximeter equipment for medical use.

A functional tester is used to measure how accurately Fingertip Pulse Oximeter is reproducing the specified calibration curve and the PR accuracy.

The model of functional tester is Index2 FLUKE simulator and the version is 2.1.3.

3. Pulse Rate

Display range: 30bpm~250bpm

Measurement range: 30bpm~250bpm

Accuracy: 30bpm~99bpm, ±2bpm; 100bpm~250bpm, ±2%

Resolution: 1bpm

4. Perfusion Index

Measurement range: 0.2%~20.0%

5. Probe LED Specifications

	Wavelength	Radiant Power
RED	660 ± 3nm	3.2mW
IR	905 ± 10nm	2.4mW

NOTE: The information about wavelength range can be especially useful to clinicians.

6. Power Requirements

Two AAA alkaline Batteries

Power consumption: Less than 40mA

Battery Life: Two AAA 1.5V, 1200mAh alkaline batteries could be continuously operated as long as 18 hours.

7. Environment Requirements

Operation Temperature: 5°C~40°C


Storage/ Transport Temperature: -25°C~+70°C

Ambient Humidity: 15%~93% no condensation in operation; ≤93% no condensation in storage/transport

Atmosphere pressure: 70kPa~106kPa

8. Equipment data update period

As shown in the following figure. Data update period of slower average is 8s.

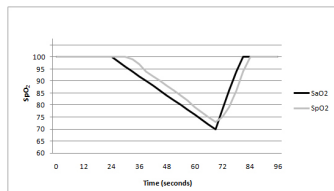

9. Classification

According to the type of protection against electric shock: INTERNALLY POWERED EQUIPMENT;

According to the degree of protection against electric shock: TYPE BF APPLIED PART, (applied part: the rubber hole of the device);


According to the degree of protection against ingress of water: IP22

According to the mode of operation: CONTINUOUS OPERATION

Clinical Study Summary

The following details are provided to disclose actual performance observed in the clinical validation study of healthy adult volunteers. The ARMS value analysis statement and Bland-Altman plot of data is shown as following:

ARMS Value Analysis Statement			
Item	90~100	80~<90	70~<80
#pts	78	66	63
Bias	1.02	0.40	-0.48
ARMS	1.66	1.46	1.93

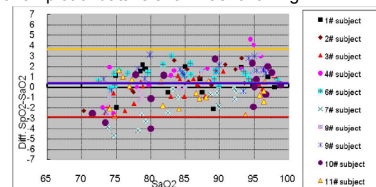

Bland-Altman Plot Graphic

Declaration

Guidance and Manufacturer's declaration – electromagnetic emissions-For all EQUIPMENT and SYSTEMS

Guidance and Manufacturer's declaration - electromagnetic emission

The iP900AP Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer or the user of iP900AP Pulse Oximeter should assure that it is used in such an environment.

Emission test	Compliance	Electromagnetic Environment – guidance
RF emissions CISPR 11	Group 1	The iP900AP Pulse Oximeter uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The pulse Oximeter (iP900AP) is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not Applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not Applicable	

Guidance and Manufacturer's declaration – electromagnetic immunity-For all EQUIPMENT and SYSTEMS

Guidance and Manufacturer's declaration - electromagnetic immunity

The iP900AP Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer or the user of the iP900AP Pulse Oximeter should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance Level	Electromagnetic Environment – guidance
Electrostatic Discharge (ESD) IEC 61000-4-2	+/- 6kV contact +/- 8kV air	+/- 6kV contact +/- 8kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristics of a typical location in a typical commercial or hospital environment.

Guidance and Manufacturer's declaration – electromagnetic immunity-For all EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and Manufacturer's declaration - electromagnetic immunity

The iP900AP Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer or the user of the iP900AP Pulse Oximeter should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance Level	Electromagnetic Environment – guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the Pulse Oximeter (iP900AP), including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a , should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations, Electromagnetic propagation is affected by absorption and reflection structures, objects and people.

a Field strengths from fixed transmitters, such as base station for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Pulse Oximeter (iP900AP) should be observed to verify normal operation. If abnormal performance is observed, additional measurements may be necessary, such as reorienting of the relocating the Pulse Oximeter (iP900AP).

b Over the frequency range 150 kHz to 80 MHz, fields strengths should be less than 3 V/m

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEMS - For all EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and Pulse Oximeter (iP900AP)

The Pulse Oximeter (iP900AP) is intended for use in electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Pulse Oximeter (iP900AP) can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Pulse Oximeter (iP900AP) as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)	
	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d=1.2\sqrt{P}$	$d=2.3\sqrt{P}$
0.01	0.1167	0.2334
0.1	0.3689	0.7378
1	1.1667	2.3334
10	3.6893	7.3786
100	11.6667	23.3334

For transmitters rated at a maximum output power not listed above, the recommended separation distance in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Possible Problems and Solutions

Problems	Possible reason	Solution
The oximeter fails to display SpO ₂ and/or PR	1. Finger is not inserted correctly 2. User's blood flow is too low to be measured	1. Make sure that the finger is placed right in between the sensor and LED lights 2. Make sure that nothing is hindering the user's blood flow
SpO ₂ or PR changes erratically	1. Finger may not be inserted deep enough 2. Excessive body movement	1. Insert the finger deep into the chamber 2. Please do not move during measurement
The oximeter cannot be powered on	1. Batteries are drained 2. Batteries might be installed incorrectly 3. The oximeter might be damaged or defective	1. Please replace the batteries 2. Please refer to 'Battery Installation' instruction 3. Please contact local distribution center
The display screen turns off suddenly	1. The oximeter automatically powered off when no signal is detected for more than 8 seconds 2. The batteries are drained	1. This is normal. Just turn the oximeter on again. 2. Replace the batteries
"Error7" is displayed on screen	Err 7 means the emission LED or reception diode is damaged	Please contact local distribution center

Symbol Definitions

Symbol	Definition	Symbol	Definition
	Type BF applied part.		Attention
IP22	Protected against dripping water.	% SpO ₂	Oxygen saturation
PR bpm	Pulse rate (BPM)		Low power indication
PI %	Perfusion Index (%)	SN	Serial No.
	Storage temperature and relative humidity		Follow instruction for use
	Manufacturer's information		Date of Manufacture
	Conformity to WEEE Directive		

Applicable Models

iP900AP

Notes:

1. The illustrations used in this manual may differ slightly from the appearance of the actual product.
2. The specifications are subject to change without prior notice.

Distributed by:

Innovo Medical

10101 Stafford Centre Dr, Ste B, Stafford, TX, 77477

858-888-9781 cs@innovogroups.com

www.innovo-medical.com

ALL RIGHTS RESERVED

Issued Date: December 2018