


FAQ: What is the best way to **clean** the head bumpers and pastern wraps of my Equinosis Q?

A: If the wraps seem to get dirty after every use, simply rinse them, let dry overnight, and machine wash once a week or as needed. The wraps **will** wear out over time and should be replaced if you see signs of wear on the Velcro or stitching. Under normal use, **we recommend replacing the wraps yearly** to maintain secure and safe attachment of the sensors to the horse. Order replacement components at equinosis.myshopify.com/collections/sensor-attachment-accessories.

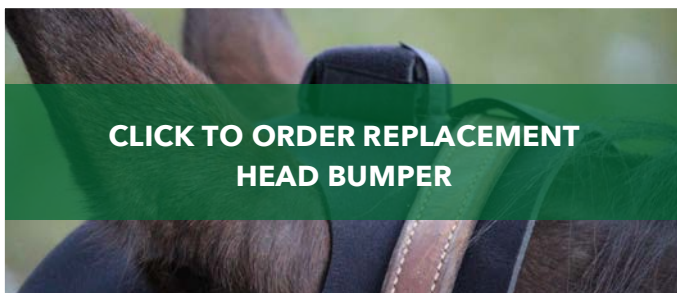

Take the following steps to properly clean and care for your Q's head bumper and pastern wrap:

Precautions

- Our Wraps DO NOT contain latex.
- Allergic reactions generally appear within the first few hours. Reactions that occur days or weeks later can often be attributed to heat, moisture or hygiene issues.
- If unusual swelling, skin discoloration, rash or discomfort occur, discontinue use.
- Care should be taken to ensure that the support does not fit so tight that circulation is impaired.
- Not for horses that have an allergic reaction to neoprene, or who tend to develop dermatitis.
- Not for use over open wounds and infected areas.

TIP: Wash frequently to remove body oils and other soil.

Step One:

HAND WASH in warm water and mild soap.

*Can be **machine washed** on gentle with mild soap.*

Step Two:

RINSE thoroughly.

*The pieces should be rinsed **really** well.*

Step Three:

AIR DRY. Do not wring.

*Be sure both pieces are **completely dry** before using on the next horse.*

DO NOT dry clean or expose to heat or sunlight for extended periods of time. DO NOT use ointments or oils under the support.