

Orthodontic Dummy INSTRUCTIONS

WARNING!

Inspect carefully before each use. Pull the soother in all directions. Throw away at the first signs of damage or weakness. Only use dedicated soother holders tested to EN 12586. Never attach other ribbons or cords to a soother, your child may be strangled by them. **Do not leave a dummy in direct sunlight or near a source of heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. If the dummy has a removable teat protector, keep it away from children to avoid suffocation.**

CLEANING & STERILISATION:

Before first use place the dummy in boiling water for 5 minutes, allow to cool, and squeeze out any trapped water from the dummy. This is to ensure hygiene. Clean before each use. Wash in warm soapy water and rinse in clean water or use a dishwasher (top rack only). Remove any water that may have entered the teat by squeezing the teat. Do not clean with solvents or harsh chemicals. Suitable for all forms of sterilisation.

STORAGE & USE:

Do not store in direct sunlight or near a source of heat. Do not store with or allow to come into contact with solvents or harsh chemicals. Never dip teat in sweet substances or medication, your child may get tooth decay. **REPLACE DUMMY AFTER ONE MONTH, FOR SAFETY AND HYGIENE REASONS.** In the event that the dummy becomes lodged in the mouth, **DO NOT PANIC**; it cannot be swallowed and is designed to cope with such an event. Remove from the mouth with care, as gently as possible. Complies with EN 1400.

Nuby[™]

Nuby UK LLP
13 Burford Way
Baldon Business Park
Baldon Colliery
Newcastle upon Tyne
NE35 9PZ
Freephone number: 0800 334 5844
enquiries@nuby-uk.com
www.nuby-uk.com
www.nuby.com



Form CWY_UK
99237

© 2022 Luv n' care
Nuby TM and Nuby logo licensed to
Luv n' care by Admar International
Product Design and all other TMs owned by
or licensed to Luv n' care

EC REP

DC PHARM
Calle Filadores, S/N - NAVE 3
Olesa de Montserrat,
08640 Barcelona, SPAIN

Nuby[™]

Silicone Breast Pump



Instructions
for use

How to use your Nuby Silicone breast pump:

1. Place the top of the breast pump over your breast and nipple – make sure your nipple is in the neck of the pump and comfortable.
2. Gently squeeze the pump until you feel suction– you may need to adjust a couple of times. Once the pump is secure and comfortable you can squeeze the base of the pump (not interfering with the top/suction).
You can also use to collect breast milk while you are feeding your baby.
3. To help encourage the milk flow you can gently massage the top of your breast or may also like to try placing a warm flannel on top of your breast.
4. The breast milk will flow out naturally.
5. Once the pump becomes full you can empty into your nuby natural touch bottle and reposition the pump.
TOP TIP: for easy pouring pinch the top of the pump to make a funnel.
6. Wash and Sterilise your pump thoroughly before and after each use, this product is suitable for all forms of sterilisation & is dishwasher safe.
7. Store in a dry, cool place, the lid helps keep pump clean for storage.
8. Check the condition of the pump regularly and replace with a new one if product shows any signs of damage.

WARNING! Adult use only. Keep Silicone Pump, Stopper & Storage Lid out of reach of children at all times.

Combat Colic Bottle & Slow Flow Teat INSTRUCTIONS

For your child's safety and health. Please read and retain this important information. WARNING!

- Continuous and prolonged sucking of fluids will cause tooth decay.
- Always check food temperatures before feeding.
- Throw away at first signs of damage or weakness.
- Keep components not in use out of reach of children.
- Never attach to cords, ribbons, laces or loose parts of clothing. The child can be strangled.
- Never use feeding teats as a soother.
- Always use this product with adult supervision.
- When making up a feed, follow the formula milk manufacturer's instructions and allow the freshly boiled water to cool before pouring it into the bottle.

Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or the product has become disassembled. Never use feeding teats as a soother. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the Baby is

allowed to use the bottle for long periods through the day and night, when saliva flow is reduced or if it is used as a soother.

CLEANING & STERILISATION

Before first use, disassemble and clean the product and then place the components in boiling water for 5 mins. This is ensure hygiene. Before each subsequent use clean carefully to ensure hygiene. Always wash the product immediately after use. Wash in warm soapy water and rinse in clean water or use a dishwasher (top shelf only). Do not turn the teat inside out for cleaning. Damage could result. Use a teat brush. Suitable for all forms of sterilisation.

STORAGE & USE

INSPECT BEFORE EACH USE AND PULL THE TEAT IN ALL DIRECTIONS. Do not store in direct sunlight or near a source of heat. Do not store with or allow to come into contact with solvents or harsh chemicals. Not suitable for use in a conventional oven or a combination microwave oven with the grill feature switched on. Do not leave a feeding teat in direct sunlight or heat, or leave in disinfectant ("sterilising solution") for longer than recommended, as this may weaken the teat. **FOR SAFETY AND HYGIENE REASONS IT IS RECOMMENDED THAT THE FEEDING TEAT IS REPLACED AT LEAST EVERY 2 MONTHS.** Do not heat in a microwave with the bottle sealed. Remove the teat, locking ring and dome cap. Take extra care when microwave heating; localised oven heating may occur. Always mix the heated food and check the temperature before feeding. Microwave safe.

ASSEMBLY INSTRUCTIONS;

- Wash hands in warm soapy water and rinse in clean water.
- Push the teat through the screw ring collar.
- Screw the bottle base to the teat collar.

Complies with EN 14350.