

# Deltatac Battlefield Tourniquet Application

## STANDARD USE GUIDELINES

- When applied in accordance with directions, The DT Battlefield Tourniquet (DT-BFT-AO1) is a safe and effective device for controlling life-threatening extremity bleeding.
- If you cannot be sure or cannot take the additional time to examine where the bleeding is coming from based on the situation, The DT Battlefield Tourniquet (DT-BFT-AO1) can be effectively applied over clothing as high on the arm or leg as possible.
- The DT Battlefield Tourniquet (DT-BFT-AO1). must NOT be applied over solid objects within the clothing.
- **As soon as the situation permits, the injured limb should be evaluated and the CAT re-positioned 2"-3" above the injury directly to the skin.**

## TECHNICAL SPECIFICATIONS

Brand	DeltaTac
Part No.	DT-BFT-AO1
Length	6.5 Inch
Width	2.4 Inch
Height	1.5 Inch
Open Length	37.5 Inch
Weight	90 Grams (approx.)
Category	Battle Field Wound Care
Windlass Rod	Yes
Single Routing Buckle	Yes
Windlass Clip	Yes
Windlass Strap	Yes
Stabilization plate	Yes
Packaging Dimensions	16 x 6.5 x 5 cms.
Country of origin	India

## LIMITATION OF LIABILITY:

Deltatac.shop, its parent company DTSP, its employees, agents, contractors, suppliers, and distributors shall assume no liability for injury or damages arising from the application and use of the DT Battlefield Tourniquet (DT-BFT-AO1) . The user assumes all risk of liability. The DT Battlefield Tourniquet (DT-BFT-AO1). should only be used as directed by user's military service component guidelines, EMS authority, or under the supervision of a physician.



# Deltatac Battlefield Tourniquet Application

Instruction Of Usage | [www.deltatac.shop](http://www.deltatac.shop)

## Step-1



## Step-2



## Step-3



## Step-4



## Step-5



## Step-6

