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USER INSTRUCTIONS

The product consists of two parts; the pill shaped filter and the reusable universal fit ear tip (Image A).
The filter is fitted into the largest supplied ear tip size (1-12 mm). First try inserting this size into the ear canal by following the instructions - 1. Inserting into the ear. If the ear tip feels uncomfortable or it is difficult to insert, place the filter into a smaller size ear tip (6-11 mm) - 3. Changing ear tip size. Please contact us if it is not possible to achieve a proper seal with the supplied ear tip sizes.

1. Inserting into the ear

- Ensure the filter is correctly positioned in the ear tip (Image G, H & I).
- Grip the tongue of the ear tip between thumb and forefinger (Image B).
- Pull the top of the ear outward and upward to ease insertion (Image C).
- Position the tongue at the top of the ear making the product easier to remove (Image D).
- Gently push and twist the product into the ear until it sits comfortably in the ear canal whilst forming a good seal with the ear canal.

2. Removing, Cleaning and Storage

- Find and grip the tongue of the ear tip (Image D).
- Remove the product with a slow twisting motion to gradually break the seal with the ear canal.
- Clean the ear tips after each use. It can be wiped clean with a damp cloth or antibacterial cleaning tissues. We recommend only cleaning the filters if necessary. Do not use alcohol-based cleaning solutions, soaps or detergents.
- Always store the product in a clean resealable carrying case. Recommended storage temperature range is between -10°C and 50°C.

3. Changing ear tip size

- Remove the filter from the current ear tip by gently squeezing the ear tip just below the filter (Image E).
- Ensure that the logo in the filter is facing upward (Image A).
- Push the filter in at an angle to ease insertion (Image F).
- Press down on the filter until it is correctly positioned in the ear tip (Image G, H & I).

WARNINGS

- If these instructions and warnings are not adhered to, the protection afforded by the product will be severely impaired.
- **Choking hazard** - Keep the product out of reach of young children.
- The product must be fitted, adjusted and maintained in accordance with the instructions. Improper fitting can reduce its effectiveness.
- In order to achieve the designed attenuation level, it's important that the outside of the ear tip has an airtight seal with the ear canal.
- Performance of the product may be reduced after contact with sharp objects e.g. by puncturing or cutting (Image K).
- Regularly inspect the product to ensure serviceability (Image E).
- The product may be adversely affected when coming into contact with chemicals or alcohol.
- Sudden or fast removal of the product out of the ear canal can damage the eardrum.
- Suitable hearing protection must be worn at all times in noisy surroundings.

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