

LimbCoat®


A waterproof cover to protect bandages, casts and dressings from getting wet when bathing, showering or swimming.

Measurements


There are two measurements that are required to select the most suitable LimbCoat for the stump. All measurements to be taken in centimeters.

The Stump

- 1. Circumference:** At the point of the measurement of **5cm to 8cm** above the bandage, cast or dressing, measure the circumference of the leg at this point.

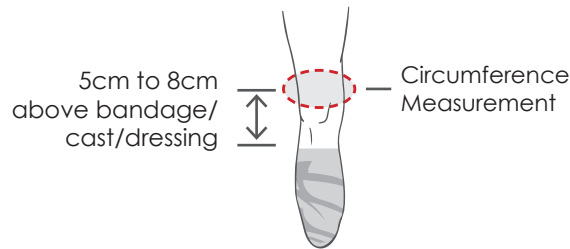

- 2. Length:** Measure from the point of where the circumference measurement was taken to the bottom of the leg by ones stump.

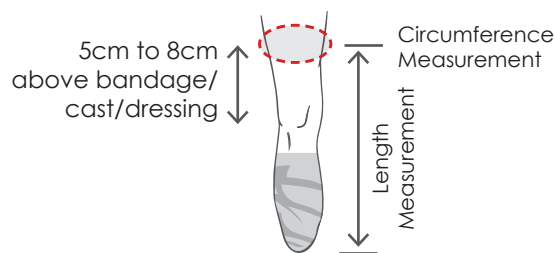

Useful Information to Assist in Measuring a stump

The circumference measurement must be located on the skin of muscle or fleshy skin area, so as to ensure a watertight seal.

A watertight seal depends on the point of the leg where the circumference measurement has been taken.

A circumference measurement located on a boney area or boney prominence (avoid ankles, elbows and knees) will compromise a watertight seal.