






BC-EA8000R Backrest Adjustment Manual

When you receive the wheelchair, please do not open it in a hurry. Please read this manual carefully

This wheelchair backrest is not manually adjusted. Please follow the steps below

(Do not force adjustment manually)

Backrest adjustment procedure


1: Click the on key on the controller  Start the machine and wait for more than 3 seconds, press and hold the white backrest adjustment key  Two seconds,  The speed light in the second row of the controller will jump to the middle  Press the OK key  immediately



This is the electric adjustment state of the backrest. Push the operating lever of the controller backward to lie back, and push forward to return to the position,



Press OK again  Exit adjustment state.

2: During the operation, pay attention to observe whether the wheelchair is stuck by foreign matters. Any part of the body must not extend to the rear and middle of the wheelchair, and children should stay away from it.

3: Install the remote control, install the matching (No. 7 battery) on the remote control first, and long press the start button  in 5 seconds,

Press the on the controller  When the key light is on and flashing, press and hold the next key  Lie back

Press on the control  When the key light is on and flashing, press and hold the up key  go back and press again ,Click again  sign out.