

# Compatibility List Instructions

Use the Compatibility List below to see if compatibility information for your medication already exists.

Please note this is **NOT** a comprehensive list.

***Nanodropper is not compatible with any preservative-free bottles/vials.***

## STEP 1



### Manufacturer Medication

Find the **medication name** and **manufacturer** on your current eyedrop bottle.

## STEP 2

Look for your medication name in the left column on the list below. Note that there might be multiple entries for your medication.

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Latanoprost	Bausch + Lomb	Compatible – Size 15
Latanoprost	Sandoz	Compatible – Size 15
Latanoprost	Greenstone	Not Compatible

## STEP 3

Check your medication manufacturer listed to the right of your medication name in the middle column.

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Latanoprost	Bausch + Lomb	Compatible – Size 15
Latanoprost	Sandoz	Compatible – Size 15
Latanoprost	Greenstone	Not Compatible

## STEP 4

If listed, check the compatibility status/size in the far-right column. If not listed, please contact us [here](#) to determine the compatibility of your bottle. If incompatible, see resources on pages 12-13.

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Latanoprost	Bausch + Lomb	Compatible – Size 15
Latanoprost	Sandoz	Compatible – Size 15
Latanoprost	Greenstone	Not Compatible

# Compatibility List

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Acuvail	Allergan	<b>Not Compatible</b>
Alaway	Bausch + Lomb	<b>Compatible – Size 15</b>
Alphagan P	Allergan	<b>Compatible – Size 15</b>
Alrex	Bausch + Lomb	<b>Compatible – Size 15</b>
Altafuor Benox	Altair Pharma	<b>Not Compatible</b>
Artificial Tears	GoodSense	<b>Compatible – Size 15</b>
Artificial Tears Solution	Rugby	<b>Compatible – Size 15</b>
Atropine sulfate	OMNI by OSRx	<b>Compatible – Size 15</b>
Atropine sulfate	Akorn	<b>Compatible – Size 14</b>
Atropine sulfate	Amneal	<b>Compatible – Size 14</b>
Atropine Sulfate PF	Imprimis Rx	<b>Not Compatible</b>
Auro-CMC	Aurolab	<b>Compatible – Size 14</b>
AuroProst RT	Aurolab	<b>Not Compatible</b>
AuroSol	Aurolab	<b>Compatible – Size 14</b>
AuroTim	Aurolab	<b>Compatible – Size 14</b>
Autologous Serum Eyedrops	Kelley Ross Pharmacy	<b>Compatible – Size 15</b>
Avatan (Travoprost)	Aristopharma Ltd.	<b>Compatible – Size 14</b>
Azarga	Alcon	<b>Not Compatible</b>
Azopt	Alcon (Novartis)	<b>Compatible – Size 15</b>
Bepreve	Bausch + Lomb	<b>Compatible – Size 15</b>
Betaxolol	Sandoz	<b>Compatible – Size 15</b>
Betimol	Akorn	<b>Compatible – Size 15</b>
Betoptic S	Novartis	<b>Compatible – Size 15</b>
Bimatoprost	Alembic	<b>Compatible – Size 14</b>
Bimatoprost	Fosun Pharma	<b>Compatible – Size 14</b>
Bimatoprost	Somerset	<b>Compatible – Size 14</b>
Bimatoprost	Apotex Corp	<b>Not Compatible</b>



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Blink Gel Tears Lubricating Eye Drops	Johnson & Johnson	<b>Not Compatible</b> See possible alternative*: Refresh Optive Gel Drops
Brimdus (Brimonidine 0.2%)	Indus Life Sciences	<b>Compatible – Size 14</b>
Brimonidine tartrate	Apotex Corp	<b>Compatible – Size 15</b>
Brimonidine tartrate	Bausch + Lomb	<b>Compatible – Size 15</b>
Brimonidine tartrate	Sandoz	<b>Compatible – Size 15</b>
Brimonidine tartrate	Poen	<b>Compatible – Size 14</b>
Brimonidine tartrate	Somerset	<b>Compatible – Size 14</b>
Brimonidine tartrate	Akorn	<b>Not Compatible</b>
Brimonidine tartrate/ Dorzolamide	OMNI by OSRx	<b>Compatible – Size 15</b>
Brimonidine tartrate/ Timolol maleate	Apotex Corp	<b>Compatible – Size 15</b>
Brimonidine tartrate/ Timolol maleate	Pacific Pharma	<b>Compatible – Size 15</b>
Brimonidine tartrate/ Timolol maleate	Sandoz	<b>Compatible – Size 15</b>
Brimonidine tartrate/ Timolol maleate	Teva	<b>Compatible – Size 14</b>
Brimonidine tartrate/ Timolol maleate	Upsher-Smith	<b>Compatible – Size 14</b>
Brinzolamide	Oceanside Pharma	<b>Compatible – Size 15</b>
Brinzolamide	Sandoz	<b>Compatible – Size 15</b>
Brinzolamide	Teva	<b>Compatible – Size 14</b>
Brinzolamide	Patagis	<b>Not Compatible</b>
Bromfenac	Alembic	<b>Compatible – Size 14</b>
Bromfenac	Fosun Pharma	<b>Compatible – Size 14</b>
Can-C	Can-C	<b>Not Compatible</b>
Cataract Clear 2% NAC Ultra	Costcuttersrus	<b>Compatible – Size 14</b>

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.



[nanodropper.com](https://nanodropper.com) | (507) 405-5676 | [support@nanodropper.com](mailto:support@nanodropper.com)

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Celdus (Carboxymethyl-cellulose 0.5%, 1%)	Indus Life Sciences	Compatible – Size 14
Ciprodux (Ciprofloxacin 0.3%)	Indus Life Sciences	Compatible – Size 14
Combigan	Allergan	Compatible – Size 15
Cortisol (Prednisolone acetate)	Aristopharma Ltd.	Compatible – Size 14
Cosopt	Akorn	Compatible – Size 15
Cosopt PF	Akorn	Not Compatible
Cromolyn sodium	Sandoz/Alcon	Compatible – Size 15
Cyclofez	Entod Pharma Ltd.	Compatible – Size 14
Cyclomune	Sun Pharma	Compatible – Size 14
Cyclomydril	Alcon	Compatible – Size 15
Cyclopentolate HCl	Akorn	Compatible – Size 15
Cyclopentolate HCl	Bausch & Lomb	Compatible – Size 15
Cyclopentolate HCl	Sandoz	Compatible – Size 15
Cyclopentolate HCl	Poen	Compatible – Size 14
Cyclosporine Corn Oil	Epicur Pharma	Compatible – Size 15
Dortimopt	Mylan	Not Compatible
Dorzolamide HCl	Bausch + Lomb	Compatible – Size 15
Dorzolamide HCl	Sandoz	Compatible – Size 15
Dorzolamide HCl	Biocon	Compatible – Size 14
Dorzolamide HCl	Micro Labs Ltd.	Compatible – Size 14
Dorzolamide HCl	Akorn	Not Compatible
Dorzolamide HCl	Leading Pharma	Not Compatible
Dorzolamide PF	Imprimis Rx	Not Compatible
Dorzolamide HCl/ Timolol maleate	Bausch + Lomb	Compatible – Size 15
Dorzolamide HCl/ Timolol maleate	Sandoz	Compatible – Size 15
Dorzolamide HCl/ Timolol maleate	Micro Labs	Compatible – Size 14



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Dorzolamide HCl/ Timolol maleate	Poen	Compatible – Size 14
Dorzolamide HCl/ Timolol maleate	Somerset	Compatible – Size 14
Dorzolamide HCl/ Timolol maleate	Akorn, GSMS, Hi-Tech	Not Compatible
Dorzolamide HCl/ Timolol maleate	Aurobindo Pharma	Not Compatible
Dorzolamide HCl/ Timolol maleate	Leading Pharma	Not Compatible
Dorzolamide HCl/ Timolol maleate PF	Aurobindo Pharma	Not Compatible
Dozodus (Dorzolamide)	Indus Life Sciences	Compatible – Size 14
Dozodus-T (Dorzolamide/Timolol)	Indus Life Sciences	Compatible – Size 14
Durezol	Allergan	Compatible – Size 15
Eyegonics Organic Tears	Eyegonics	Not Compatible
Flarex	Eyevance	Compatible – Size 15
Fluorometholone acetate	Pacific Pharma	Compatible – Size 15
Fluress	Akorn	Not Compatible
Focus PI Post-Injection Eye Drops	Focus Vitamins	Compatible – Size 14
Freshkote Preservative Free (PF) Lubricant Eye Drops	Santen	Not Compatible
GenTeal Tears	Alcon	Compatible – Size 15
Glanatec	Kowa	Not Compatible
Hylube	Aurolab	Compatible – Size 14
Ilevro	Alcon	Not Compatible
Inveltys	Kala Pharmaceuticals	Not Compatible
iVIZIA	Similasan	Not Compatible
Ketorolac tromethamine	Sandoz	Compatible – Size 15



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Ketorolac tromethamine	Micro Labs Ltd.	Compatible – Size 14
Klarity Drops PF	Imprimis Rx	Not Compatible
Lastacaft	Allergan	Compatible – Size 15
Latanoprost	Bausch + Lomb	Compatible – Size 15
Latanoprost	Sandoz	Compatible – Size 15
Latanoprost	Somerset	Compatible – Size 14
Latanoprost	Greenstone	Not Compatible
Latanoprost	Rising Pharma	Not Compatible
Lotemax Gel	Bausch + Lomb	Compatible – Size 15
Lotemax Suspension	Bausch + Lomb	Compatible – Size 15
Loteprednol etabonate	Oceanside Pharma	Compatible – Size 15
Loteprednol etabonate	Sun Pharma	Compatible – Size 14
Loteprednol etabonate	Akorn	Not Compatible
Lubricant Eye Drops	CVS Health	Compatible – Size 14
Lumify	Bausch + Lomb	Compatible – Size 15
Lumigan	Allergan	Compatible – Size 15
Maxidex	Alcon	Compatible – Size 15
Methopt-Tears	Mylan (NZ)	Compatible – Size 14
Miebo	Bausch + Lomb	Compatible – Size 14
Moxifloxacin	Apotex Corp	Compatible – Size 15
Moxifloxacin	Alembic	Compatible – Size 14
Moxifloxacin	Aurobindo Pharma	Compatible – Size 14
Moxifloxacin/Bromfenac	OMNI by OSRx	Compatible – Size 15
Muro 128	Bausch + Lomb	Compatible – Size 15
Mydracyl	Alcon	Compatible – Size 15
Naphcon A	Alcon	Compatible – Size 15
Natacyn	Alcon (Novartis)	Not Compatible
Natacyn	Eyevance	Not Compatible



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Natural Eyes	TRP	Compatible – Size 15
Neomycin-Polymyxin-Dexamethasone	Bausch + Lomb	Compatible – Size 15
Ofloxacin	Akorn	Not Compatible
Olopatadine HCl	Apotex Corp	Compatible – Size 15
Olpadin (Olopatadine HCl)	Aristopharma Ltd.	Compatible – Size 14
OphtHAVet Complete Ophthalmic Gel	Decra	Not Compatible
Opti-Free PureMoist Rewetting Drops	Alcon (Novartis)	Compatible – Size 15
Paremyd	Akorn	Compatible – Size 15
Pataday	Alcon	Not Compatible See possible alternative*: Lastacaft
Phenylephrine HCl	Akorn	Compatible – Size 15
Phenylephrine HCl	Paragon	Compatible – Size 15
Phenylephrine HCl	Lifestar Pharma	Compatible – Size 14
Pilocarpine HCl	Akorn	Compatible – Size 15
Pilocarpine HCl	Sandoz	Compatible – Size 15
Pilocarpine HCl	Martindale Pharma	Compatible – Size 14
Pilocarpine HCl	Somerset	Not Compatible
Polyethylene Glycol 400 and Propylene Glycol	Intas Pharma	Compatible – Size 14
Polymyxin B Sulfate/Trimethoprim	Sandoz	Compatible – Size 15
Polyvinyl alcohol	Rugby	Compatible – Size 14
Pred-Acetate PF	Imprimis Rx	Not Compatible
Pred Forte	Allergan	Compatible – Size 15
Pred-Moxi-Brom	Imprimis Rx	Compatible – Size 15
Pred-Moxi-Nepaf	Imprimis Rx	Compatible – Size 15

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.



nanodropper.com | (507) 405-5676 | support@nanodropper.com

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Prednisolone acetate	Bausch + Lomb	Compatible – Size 15
Prednisolone acetate	Pacific Pharma	Compatible – Size 15
Prednisolone acetate	Sandoz	Compatible – Size 15
Prednisolone phosphate/ Bromfenac	OMNI by OSRx	Compatible – Size 15
Prednisolone phosphate/ Moxifloxacin	OMNI by OSRx	Compatible – Size 15
Prednisolone phosphate/ Moxifloxacin/Bromfenac	OMNI by OSRx	Compatible – Size 15
Prednisolone phosphate/ Moxifloxacin/Ketorolac tromethamine	OMNI by OSRx	Compatible – Size 15
Prolensa	Bausch + Lomb	Compatible – Size 15
Proparacaine HCl	Akorn	Compatible – Size 15
Proparacaine HCl	Bausch + Lomb	Compatible – Size 15
Proparacaine HCl	Sandoz	Compatible – Size 15
Refresh Digital	Allergan	Compatible – Size 15
Refresh Liquigel	Allergan	Compatible – Size 15
Refresh Optive Advanced	Allergan	Compatible – Size 15
Refresh Optive Gel Drops	Allergan	Compatible – Size 15
Refresh Relieva	Allergan	Compatible – Size 15
Refresh Tears	Allergan	Compatible – Size 15
Refresh Plus (Preservative-Free)	Allergan	Not Compatible
Regener-Eyes® Lite	Regener-Eyes®	Not Compatible
Regener-Eyes® Professional Strength	Regener-Eyes®	Not Compatible
Restasis	Allergan	Not Compatible
Restasis Multidose	Allergan	Not Compatible

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.



[nanodropper.com](http://nanodropper.com) | (507) 405-5676 | [support@nanodropper.com](mailto:support@nanodropper.com)



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Rhopressa	Alcon/Aerie	Compatible – Size 15
Rocklatan	Alcon/Aerie	Compatible – Size 15
Simbrinza	Alcon (Novartis)	Compatible – Size 15
Sodium Chloride Hypertonicity Solution	Rugby	Compatible – Size 15
Systane	Alcon	Compatible – Size 15
Systane Complete	Alcon	Compatible – Size 15
Systane Complete PF	Alcon	Not Compatible
Systane Ultra	Alcon	Compatible – Size 15
Systane Zaditor	Alcon	Compatible – Size 15
Systear	Aristopharma Ltd.	Compatible – Size 14
Tacrolimus	Epicur Pharma	Compatible – Size 15
Tetracaine HCl	Oceanside Pharma	Compatible – Size 15
TetraVisc Plus	Primera	Compatible – Size 15
TheraTears Dry Eye Therapy	Thera Tears	Compatible – Size 14
Timodus (Timolol 0.5%)	Indus Life Sciences	Compatible – Size 14
Timolol maleate	Akorn	Compatible – Size 15
Timolol maleate 0.5% 10 mL, 15 mL	Sandoz	Compatible – Size 15
Timolol maleate 0.5% 5 mL	Sandoz	Compatible – Size 14
Timolol maleate	Alembic	Compatible – Size 14
Timolol maleate	Pacific Pharma	Not Compatible
Timolol maleate	Rising Pharma	Not Compatible
Timolol maleate/Bimatoprost	OMNI by OSRx	Compatible – Size 15
Timolol maleate/Brimonidine tartrate/Dorzolamide	OMNI by OSRx	Compatible – Size 15
Timolol maleate/Brimonidine tartrate/Dorzolamide/Bimatoprost	OMNI by OSRx	Compatible – Size 15



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Timolol maleate PF	Akorn	<b>Not Compatible</b>
Timoptic in Ocudose	Valeant/Bausch	<b>Not Compatible</b>
Tobradus (Tobramycin 0.3%)	Indus Life Sciences	<b>Compatible – Size 14</b>
Tobradus-D (Tobramycin 0.3%, Dexamethasone 0.1%)	Indus Life Sciences	<b>Compatible – Size 14</b>
Tobramycin/Dexamethasone	Sandoz	<b>Compatible – Size 15</b>
Tobramycin/Dexamethasone	Poen	<b>Compatible – Size 14</b>
T PROST	Aurolab	<b>Not Compatible</b>
TPC Drops	Pine Pharma	<b>Compatible – Size 15</b>
Travatan Z	Novartis, Alcon/Novartis	<b>Not Compatible</b>
Travoprost	Micro Labs	<b>Compatible – Size 14</b>
Travoprost	Mylan	<b>Compatible – Size 14</b>
Travoprost	Apotex Corp	<b>Not Compatible</b>
Travoprost	Sandoz	<b>Not Compatible</b>
Tropicamide	Akorn	<b>Compatible – Size 15</b>
Tropicamide	Bausch + Lomb	<b>Compatible – Size 15</b>
Tropicamide	Sandoz	<b>Compatible – Size 15</b>
Tropicamide	Somerset	<b>Compatible – Size 14</b>
Tropicamide/ Phenylephrine HCl	OMNI by OSRx	<b>Compatible – Size 15</b>
Tropi-Phen	Pine Pharma	<b>Compatible – Size 15</b>
Vevye	Novaliq	<b>Compatible – Size 14</b>
Vigamox	Novartis	<b>Not Compatible</b> See possible alternative*: Moxifloxacin
Vuity	Allergan	<b>Compatible – Size 15</b>
Vyzulta	Bausch + Lomb	<b>Compatible – Size 15</b>

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.



MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Xalatan	Pharmacia/Pfizer	<b>Not Compatible</b> See possible alternative*: Latanoprost
Xdemvy	Tarsus	<b>Compatible – Size 15</b>
Xelpros	Sun Pharma	<b>Compatible – Size 14</b>
Xiidra	Novartis	<b>Not Compatible</b>
Zioptan	Théa	<b>Not Compatible</b>
Zirgan	Bausch + Lomb	<b>Not Compatible</b>

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.

**Don't see your bottle on the list?**

Contact us [here](#) to determine compatibility.

**Have an incompatible bottle?**

Please see resources on pages 12-13.

# Is Your Bottle Incompatible?

Here are some resources for you!

Using the Compatibility List, you determined that your medication is not compatible with the Nanodropper Adaptor.

Please see below for a list of common incompatibility scenarios and resources.

Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.

## SCENARIO

### 1

#### Your bottle is incompatible with Nanodropper but compatible options for this medication DO exist.

*Example: Latanoprost manufactured by Bausch & Lomb and Sandoz are compatible alternatives to latanoprost manufactured by Greenstone, which is not compatible with Nanodropper.*

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Latanoprost	Bausch + Lomb	Compatible – Size 15
Latanoprost	Sandoz	Compatible – Size 15
Latanoprost	Greenstone	Not Compatible

You can **call local pharmacies** to see if they stock the same medication made by a compatible manufacturer. Please note that insurance coverage may differ between manufacturers.

## SCENARIO

### 2

#### Your bottle is incompatible with Nanodropper and compatible options for this medication DO NOT exist.

*Example: Pataday manufactured by Alcon is not compatible with the Nanodropper Adaptor. For some incompatible eyedrop bottles, compatible medications that may be considered as alternatives are provided.*

MEDICATION	MANUFACTURER	COMPATIBILITY STATUS
Pataday	Alcon	Not Compatible See alternative*: Lastacraft

\*Please talk to your doctor before making any changes to your medications. This is not intended to serve as medical advice.

## NOTES

### We want to hear from you!

We are working to design additional adaptors to meet all your medication needs.

If you can confirm compatibility or incompatibility of any bottles not on this list, please send a picture of the bottle along with the medication name, concentration, and manufacturer to [support@nanodropper.com](mailto:support@nanodropper.com).

## REFUND POLICY

We want you to feel confident in purchasing our products. If you purchase a Nanodropper and find that it does not fit your bottle as described in the Compatibility List, please take a picture of your bottle with the medication name and manufacturer visible and send the photo to [support@nanodropper.com](mailto:support@nanodropper.com) for a refund. Shipping & handling fees are non-refundable.

Find our [full refund and return policies here](#).

