

- REQUIREMENTS:** HOME tonometer  
HOME tonometer USB cable  
Android tablet and adapter [if needed] with Wi-Fi Internet connection  
PC or MAC computer with internet connection: Chrome, Edge, Firefox or Safari web browsers

## STEP 1 *Order CLINIC\**

1. Proceed to <https://icarehome.com/home-usa/>
2. Click "**Accept terms and conditions**"
3. Click "**Order Here**"
4. Click box for "**I have a discount coupon**" if applicable
5. Click "**Company**" in Billing information
6. Fill out your Email, Company name and Address
  - a. Make sure it is an email you have access to
7. Click "**Continue**"
8. Review your information for accuracy
9. Click "**Place Order**"
10. You will receive your login information within 24 hours

\*This step may have already been completed for you.

## STEP 2 *Icare CLINIC login*

1. After receiving your "**Icare CLINIC**" login email
2. Click on the link in the email
3. Create a password (min 7 characters: upper case, lower case, and a number)
4. You are now in the system

## STEP 3 *Add HOME tonometer to CLINIC*

1. Log into CLINIC by going to [icare-usa.com](https://icare-usa.com)
2. Click "**CLINIC LOGIN**"
3. Enter your username and password click "**Log In**"
4. Click "**Devices**" at the top
5. Click "**Add New Device**"
6. Retrieve your serial number by one of the following.
  - a. Open the battery cover and look below the batteries
  - b. Review box or invoice
  - c. Connect HOME to EXPORT software [explained next]

## STEP 3 *Cont'd*

7. Add serial number to form and click "**Add as new Device**"
8. Give the device a name
  - a. Suggestion is to label the HOME with the serial number and device name
9. Choose "**Icare HOME**" from the "**Device Type**"
10. Click "**Add**"
11. Your device is now ready

## STEP 4 *Icare EXPORT (download HOME tonometer measurements via PC)*

1. Download EXPORT to PC computer via internet by going to [icare-usa.com/export/](https://icare-usa.com/export/)
2. Double click on EXPORT installer
3. Follow prompts
4. Open "**Icare EXPORT**"
5. Connect HOME to computer with USB cable provided

## STEP 5 *Setup Android Tablet*


1. Turn on tablet
2. Connect to Wi-Fi by tapping the top edge [Settings, Wi-Fi, enter password]
3. Open "**Google Play**" store, search for "**Icare PATIENT**" and install
4. Open Icare PATIENT
5. Connect your HOME tonometer to the tablet using the provided cables/adaptor  
(If first time connecting to the tablet, the tablet may ask you to trust the device; click "**YES**")
6. Icare PATIENT should say ..... "**Measurements Saved**"

**TO BEGIN:**

Log-in to CLINIC by going to [icare-usa.com](http://icare-usa.com) and click "**CLINIC LOGIN**"

**TASK** *How do I add a patient?*

1. Click "**PATIENTS**"
2. Click "**ADD NEW PATIENT**"
3. Fill in form with First name, Last name, Patient ID, and Date of birth
  - a. Best practice is to type the birthdate mm/dd/yyyy i.e. 02/19/1974
4. Click "**SAVE**"

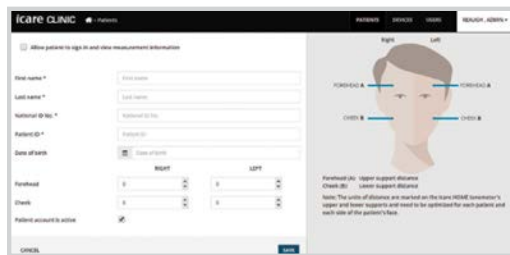

**TASK** *How do I add a user?*

1. Click "**USERS**"
2. Click "**ADD NEW USER**"
3. Fill in Email, First name, Last name, and Role
  - a. Administrator can add users
4. Click "**SUBMIT**"
5. The User will receive an email to set their password



**TASK** *How do I add a HOME device?*

1. Click "**DEVICES**"
2. Click "**ADD NEW DEVICE**"
3. Retrieve serial number from battery area of HOME device
4. Type serial number in form and click "**ADD AS NEW DEVICE**"
5. Fill in a Name for the device and choose "**icare HOME**"
6. Click "**ADD**"



**TASK** *How do I allow a patient to see their measurements?*

1. Click "**PATIENTS**"
2. Click on the patient's name
3. Click box at top that says "**Allow patient to sign in and view measurement information**"
4. Type in patients email address
5. Click checkbox that says "**User account is active**"
6. Click "**SAVE**" button at the bottom
7. The patient will receive an email to set their password

