SILICONE SUPER-ABSORBENT DRESSINGS

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Vitále® Silicone Super-Absorbent Dressings are extremely absorbent and provide an ideal moist environment for wound healing. They are soft and conformable to fragile skin and are effective in minimizing pain and discomfort.

Vitále® Silicone Super-Absorbent Dressings are indicated for the treatment of light to heavily exuding wounds, and they assist in protecting the wound, preventing bacterial contamination and skin maceration.

Vitále® Silicone Super-Absorbent Dressings have 3 layers:

1. A soft silicone wound contact layer that is conformable for intimate skin contact. The silicone layer provides flexible adhesion, to the wound & to the border, and has many holes to pass exudates through to the absorbent layer, controlling excess moisture.
2. An absorbent pad made of Super-Absorbent fibers, which is extremely absorbent for highly-exuding wounds and helps provide an ideal moist wound environment.
3. The waterproof yet vapor permeable outer film is gentle and conformable for fragile skin.

Vitále® Silicone Super Absorbent Dressings are sterilized by ethylene oxide and must not be resterilized.

The silicone adhesive covers the entire wound or covers the primary dressing surface, & allows the center of the dressing to gently adhere to the wound or the primary dressing. The border attaches firmly yet gently to the peri-wound tissue, helping prevent breakdown of surrounding tissue. The silicone is elastic & flexible so that the dressing may be gently peeled back for inspection.

INDICATIONS

Vitále® Silicone Super-Absorbent Dressings are for single use only and may be used as:

- A primary dressing in direct contact with the wound bed.
- A secondary (cover) dressing over an appropriate wound filler or other primary dressing.

Can be used for the treatment of the following wounds:

Deep & Superficial

Light to Heavy Exuding

- Pressure Ulcers
- Arterial Ulcers
- Venous Ulcers
- Skin Donor Sites
- Diabetic Ulcers
- Second Degree Burns

Wounds being treated may have light to heavy exudate, and they may be superficial or deep.

SIDE EFFECTS & INTERACTIONS

None have been observed to date.

STORAGE

Vitále® Silicone Super-Absorbent Dressings should be stored in a cool & dry area. The products are sterile unless the package is opened or damaged.

APPLICATION

1. Clean and rinse the wound per the usual protocol. Wound Cleanser Spray helps condition & prepare the wound bed for healing.
2. Using sterile technique, take the dressing out of the package, & remove the center release film exposing the soft silicone layer.
3. Place the dressing on the moist wound, centering & positioning the pad on the wound. The soft silicone contact layer is so flexible that it may be adhered directly to the wound bed.
4. When used as a secondary dressing, apply primary dressing or wound filler to the wound bed per that product's Instructions for Use. Then apply the Vitále® Silicone Dressings by removing the center release film & applying to the center of the wound.
5. Remove the border release film & adhere to peri-wound tissue. Do not stretch the dressing while applying.
6. If the absorbent pad becomes saturated, exudates will leak out the side of the absorbent pad and may be observed in the border. When this occurs, the dressing should be changed.
7. Change dressing as directed, when the absorbent pad is saturated, or at least every 7 days.

Dressings may remain in place for 1-7 days based on wound characteristics and drainage amounts.

PRODUCT LINE

CATALOG NUMBER	DESCRIPTION	SIZE	UNITS/BOX
20234	Vitále® Silicone Super-Absorbent Dressing	3½" x 4" (9 cm x 10 cm)	10
20245	Vitále® Silicone Super-Absorbent Dressing	4¾" x 5½" (12 cm x 14 cm)	10
20267	Vitále® Silicone Super-Absorbent Dressing	6" x 7" (15 cm x 18 cm)	10
20280	Vitále® Silicone Super-Absorbent Dressing	8" x 10" (20 cm x 25 cm)	10
20290	Vitále® Sacral Silicone Super-Absorbent Dressing	7" x 6¾" (18 cm x 17 cm)	5
20292	Vitále® Sacral Silicone Super-Absorbent Dressing	6" x 5¾" (15 cm x 14.5 cm)	10

CELLERA

CINCINNATI, OH

www.CELLERALLC.com

Do not use if package is damaged

Single-Use

Keep Dry

Storage Temperature

Sterilized by exposure to Ethylene Oxide

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