# INSTRUCTIONS FOR USE INSTRUCCIONES DE USO

## **EASY CLEAN**





MICROWAVE BAG

BOLSA PARA MICROONDAS



Ardo medical AG  
Gewerbstrasse 19  
6314 Unterägeri  
Switzerland  
T +41(0)41 754 70 70  
F +41(0)41 754 70 71  
info@ardo-usa.com  
www.ardo-usa.com

Distributed in USA by:  
Comit Developers  
116 Toledo Drive  
Lafayette, LA 70506, USA  
T 844-411-ARDO (2736)  
www.ardo-usa.com  
customer-service@ardo-usa.com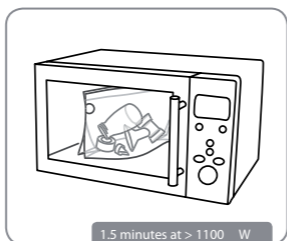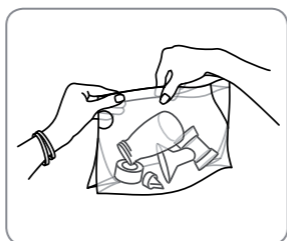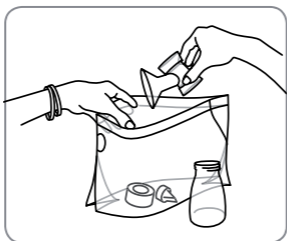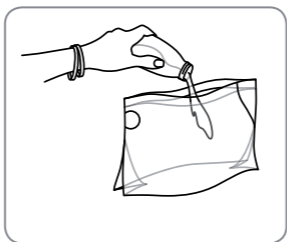

1.5 minutes at > 1100 W  
3 minutes at 800 - 1100 W  
5 minutes at 600 - 750 W

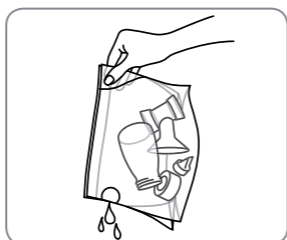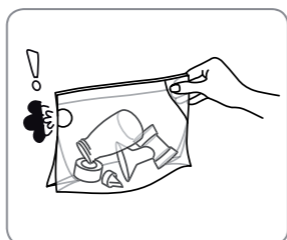## INSTRUCTIONS FOR USE

### EASY CLEAN MICROWAVE BAG

#### Instructions for use

1. Clean all parts after use with warm soapy water and rinse them with clean water.
2. Add 60 ml (2 ounces) of water to the bag.
3. Place the parts to be steam-cleaned in the bag. Seal the bag tightly.
4. Place the bag in the centre of the microwave.
5. Set the microwave to the correct power and time and start it:
  - 1.5 minutes at > 1100 W
  - 3 minutes at 800 – 1100 W
  - 5 minutes at 600 – 750 W
6. Open the microwave when it has finished and let the bag cool down for at least 2 minutes.  
**CAUTION: Bag will be hot. Do not touch the bag.**
7. Pick up the bag using the safe grip and remove it from the microwave.
8. Hold the bag away from you and carefully pour out the remaining water through the steam vent.  
**CAUTION: Avoid contact with hot water.**
9. Open the bag and remove the steam-cleaned parts.  
**CAUTION: Let components cool before touching them.**
10. The bag can be used up to 20 times. Tick a box after each use so you can check how many times are left.

#### Warnings

- Always add at least 60 ml (2 ounces) of water to the bag to avoid damaging the parts.
- Do not overfill the bag.
- Do not place the bag into the microwave on a metal rack.
- Do not exceed the stated times for steam cleaning to prevent the parts from damage.
- If the microwave does not have a turntable, rotate the bag halfway through the cleaning process.
- **CAUTION: When removing the bag from the microwave and opening it, hot steam or water may escape.**
- Only use the bag with items deemed suitable for microwaving.
- Do not use the bag in a conventional oven and do not grill it.
- **WARNING: To avoid danger of suffocation, keep the plastic bag away from babies and children.**
- To prevent limescale, you can add a drop of citric acid.
- Certain parts may discolour when used. This has no effect on their function.
- Follow the instructions for the microwave.

Instructions for use in other languages are available at [www.ardomedical.com](http://www.ardomedical.com)

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