


vive®

# FOLDING CANE

User Manual

MOB1001

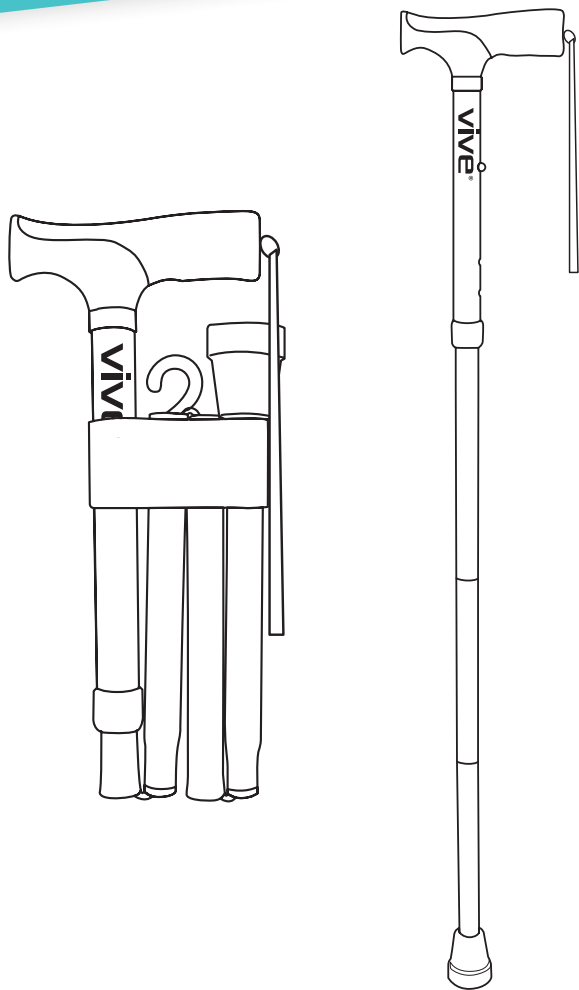

[vivehealth.com](http://vivehealth.com)

## QUICK START

1. Remove the white plastic protector from the locking latch.
2. Slide the black plastic sleeve down and over the latch.
3. Hold the handle and release the folded sections. The cane will automatically snap into place.




## FIND YOUR CORRECT CANE HEIGHT

1. Let your arm hang naturally by your waist. Adjust the cane height so the handle touches your wrist.



## CANE HEIGHT ADJUSTMENT

1. Loosen the locking latch
2. Adjust brass pin to desired position
3. Tighten the locking latch

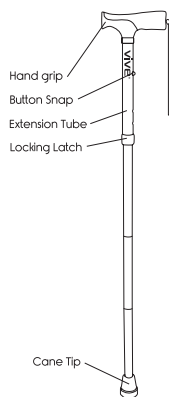

## WARNING

- Make sure the brass pin is locked into place before use.
- Weight Capacity: 250 LBS

## WARRANTY REGISTRATION

You are protected by Vive Health's industry leading guarantees and customer service:



If you did not purchase through [vivehealth.com](http://vivehealth.com), activate your guarantee at: [www.vivehealth.com/register](http://www.vivehealth.com/register).

**Product Code: MOB1001**



We sincerely appreciate your business.  
We strive to provide you with the best  
quality products at great value.

If you have any questions please contact us.



[service@vivehealth.com](mailto:service@vivehealth.com)



1-800-487-3808

Monday - Friday 9am - 5pm EST