


vive® SOCK ASSIST


User Manual
LVA1O67WHT

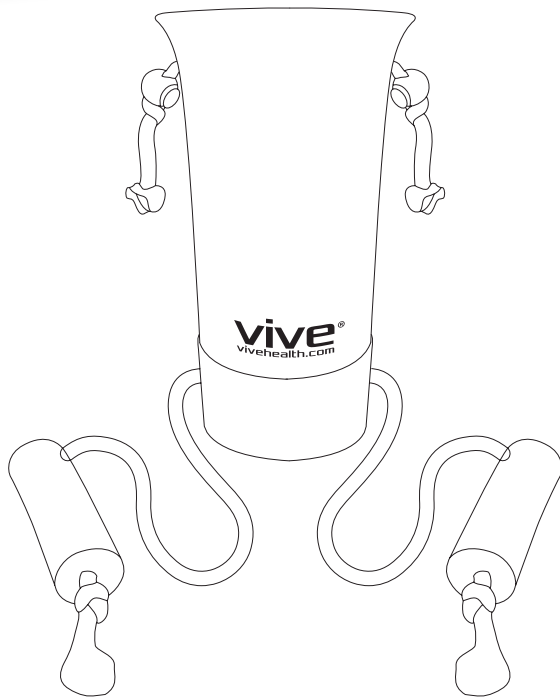


vivehealth.com

HOW TO USE

1. Pull the sock over the Sock Assist Device as far as the sock will go.

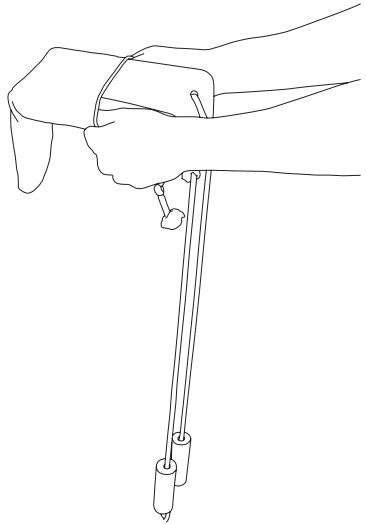

2. Place the Sock Assist Device on the floor. Keep the handles in your hands.

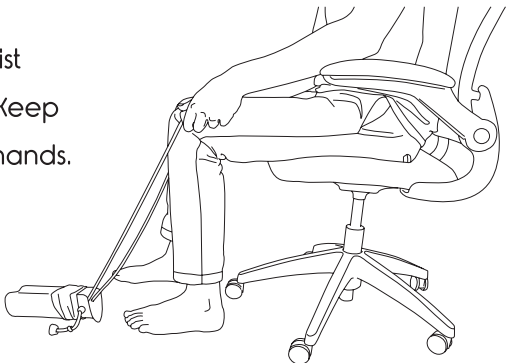

3. Insert your foot through the open hole.



4. Pull the handles until your sock is completely over your foot.



ACTIVATE YOUR GUARANTEE



You are protected by Vive Health's industry leading guarantees and customer service:

If you did not purchase through vivehealth.com, please register at vivehealth.com/register to validate your guarantee.

Product Code: LVA1O67WHT

✉ service@vivehealth.com

☎ 1-800-487-3808
Monday - Friday 9am - 5pm EST