

ALTERNATING PRESSURE MATTRESS REPLACEMENT PAD

QUICK START GUIDE

WHAT'S INCLUDED

- 1x Pressure Pad
- 2x Inflation Tubes (connected)
- 1x Patch Kit

HOW TO SET UP MATTRESS

1. Unroll the pressure pad and lay it flat on your mattress. Make sure the inflation valves are facing up and positioned at the foot of the bed.
2. Connect the inflation tubes to the pressure pad and then to your pressure pump. The air tubes can be pulled apart for easier connection if needed.

3. Follow the instructions for your alternating pump to finish setting up your alternating pressure mattress.

HOW IT WORKS

The Vive alternating pressure pad has 130 bubble air cells divided into 2 sections. Each section is connected separately to the pressure pump. The cells will alternate inflation and air pressure depending on the pressure pump you will be using.

CARE

Use a cloth with a mild detergent solution or a surface detergent / disinfectant to clean the pad and hose. Dry off completely before use

REPAIR GUIDE



For a video demonstration check out vhealth.link/sq

Only for use with small punctures

1. Unhook the inflation tubes and fully deflate the alternating pad.
2. Locate the puncture and clean off any dirt within a 3 inch radius.
3. Take a repair patch and add glue around the edges.
4. Firmly place the patch over the puncture, and apply pressure with your hands or a heavy object.
5. Let the glue sit for at least 2 hours before use.
6. Check the seal around the patch and add more glue if needed.

SPECIFICATION

- Dimensions: 36" x 78" x 3" (fully inflated)
- Materials:
 - Pad: (EN-71) 0.3mm
 - Tube: Natural Rubber (6mm inside diameter)
- Weight Capacity: 300 lbs.

WARNINGS

- Do not use corrosive cleaning products such as industrial degreasers or acetic solvents to clean the pad.
- Keep away from sharp objects as they may puncture the pressure pad.
- Do not use device near an open flame.

vive®