# SVAKOM

## Nova Manual

@svakomusa

@svakomtheofficial

SVAKOM Official

@svakom\_usa

**SVAKOM Design USA Limited**

E-mail: [Customer-service@svakom.com](mailto:Customer-service@svakom.com) Tel: 1-844-478-2566  
Facebook: [Svakom Design USA Limited](#) Twitter: [Svakom5](#) [www.svakom.com](http://www.svakom.com)

# QUICK GUIDE

## Contents

Chapter One.....	About SVAKOM Nova Balls
Chapter Two.....	The SVAKOM Nova Balls Exercise Course
Chapter Three.....	How to Use SVAKOM Nova Balls
Chapter Four.....	Product Maintenance & Safety Warning
Chapter Five.....	Warranty Terms & Conditions
Chapter Six.....	Disclaimer

## CHAPTER ONE ABOUT SVAKOM NOVA BALLS

SVAKOM Nova Balls are a combination of three different exercise balls, for your convenience after exercising, each set of Nova ball is equipped with one silicone string.

The entire Nova ball is made of silicone which is 100% waterproof. According to the kegel exercise theory, we designed these exercise courses especially to help those who have a loose vagina before or after giving birth. Three sets of Nova balls have three gradual increased weights for different exercise needs. Effective kegel exercise helps to strengthen woman's PC muscle, improving health while bringing sexual benefits and satisfaction. For daily exercise or postpartum recovery, SVAKOM Nova balls are the perfect choice for you.

Inside of every SVAKOM Nova ball, there's a built-in moving metal ball. As you walk, run or swim, the metal ball will collide with the inner surface, because of gravity the Nova ball tends to slip out of your body, but the subtle vibration stimulates your PC muscle to shrink and stop the ball from slipping out. The vibration may be subtle, but the repetitive shrinking will help strengthen your PC muscle through your every movement. That is how a SVAKOM Nova ball exercises your vagina.

## TECHNICAL DATA

Dimension:

Single Ball Dimension:Φ36X135 mm

Double Ball Dimension (small):Φ32X170 mm

Double Ball Dimension (big):Φ28X162 mm

Weight: 49g 75g 95g

Water Repellency: 100% Waterproof

Colors: Green, Violet, Plum Red

This manual is dedicated to introduce the usage and precautions of SVAKOM Nova Ball as a kegel exercise tool, please read this manual carefully before using the product.

## CHAPTER TWO SVAKOM NOVA BALL EXERCISE COURSE

Every set of SVAKOM Nova Balls has three different balls with different sizes and weights, which can be applied to different types of scientific training courses. The exercise intensity increases gradually in different ways and uses different balls, be sure to follow these the instructions.

**Phase One:** For the first time use, in order to help your body adapt to the ball. Please use the 49g single ball, 30 minutes each day, for about 30 days.

**Phase Two:** After your body has adjusted to the single ball, you can use the 75g double ball to increase the exercise intensity, exercise for 30 minutes each day, for about 30 days.

**Phase Three:** During this phase, your body can easily control the double ball, so to step up the training intensity, you can use the 95g double ball, 30 minutes each day, for about 30 days.

### SVAKOM Tips

1. When doing the exercises, you should use the Nova ball while walking, running, swimming, dancing or doing housework, instead of sitting or lying still. Because these kind of movements can make the ball inside your body move to stimulate your vagina muscle, therefore exercising the PC muscle.
2. During the exercise, if you begin to feel unwell after using it, please stop using the ball and consult with your doctor as soon as possible.
3. If you have serious Vaginitis, Vulvitis or PID you should not use the ball until you are cured.
4. Do not use the ball within 30 days after an abortion surgery or within 3 months after giving birth. For other special situations please consult with your doctor.
5. Do not use the ball during pregnancy or during your period.

## CHAPTER THREE HOW TO USE SVAKOM NOVA BALL

### How to Put in The Ball

1. In order to put the SVAKOM Nova Ball into your vagina easily, please use a proper amount of water-based lubricant on the ball or in your vagina.
2. Lean on a chair or on the sofa, open your legs, relax your vagina, and slowly push in the ball. Some women think it's easier to put in the ball standing with one leg lifted up.
3. If using a double ball, please rub both balls with water-based lubricant. Because there's some space between the balls, it will be a little harder to push in the second ball. So you need to take a deep breath, relax your vagina; and push the ball inside with your hand.

Note: Please use water-based lubricant only.

### Where to Place the Ball & How to Get it out

1. Please keep the ball at least 2cm from your vaginal orifice to make sure the ball does not slip out of your vagina, normally the depth of woman's vagina is 7cm to 12cm. Choose a ball and place the ball in a proper position of your vagina, so as to improve exercising effect.
2. When taking out the ball, relax your PC muscle and pull out the ball using the silicone string slowly. If you cannot get the ball out, lie down, relax and try again.

Note: In order to get the ball out, please make sure the silicone string can be seen outside your vagina during the whole exercising.

## CHAPTER FOUR PRODUCT MAINTENANCE & SAFETY WARNING

### Knowing how to use and maintain the Nova ball properly will extend the products lifespan.

1. Be sure to clean before and after using the SVAKOM Nova Ball. Firstly, wash the balls with warm water and antibacterial soap, then wash it off with clean warm water again, dry the balls with a towel after cleaning. **Do not use alcohol, gasoline, or acetone detergent for cleaning.**
2. Please don't use silicone-based lubricant when using the product, for it may cause stickiness to the ball permanently. **Do not use massage oil or hand cream as lubricant. Water-based lubricant is recommended.**
3. Please store the SVAKOM Nova Balls in a clean and dry environment, do not store it with toys made of other materials. Do not expose the product in high temperature or in direct sunlight, and keep the product out from children's reach.
4. This product is only for people 18 years and older. Pregnant women and patients with heart disease, diabetes, phlebitis or thrombosis should consult with their doctor about potential health risks before using a sex toy.

## Chapter Five Warranty Terms & Conditions

### 1 Year Warranty

The 1 year warranty covers to one year since the purchase date. If there is a quality problem during this period, you can claim and receive a free replacement. Noted that our warranty range includes the problems caused by material flaws, manufacturing flaws, and the parts that influences the normal functioning. Damages caused by improper operation, overused abrasion and disassembling the product are excluded from our **warranty range**.

To get the 1 Year Warranty, you need to show us the proof of purchase and that the product is still in the 1 year warranty time limit. Therefore, to ensure the validity of the warranty, please carefully save the original receipt and warranty card.

### 10 Year Quality Guarantee

The 10 year quality guarantee covers your product for the ten years subsequent to your 1 year warranty's expiration date. If you have a problem with your SVAKOM product (purchased from our official website) within this time, you will receive a 50% discount on your next purchase of a SVAKOM product.

## CHAPTER SIX DISCLAIMER

Customers using SVAKOM Nova Balls should assume the risk by themselves. SVAKOM and its retailers don't assume any legal liability. Product may be improved according to customers' requirement without prior notice.

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## WARNING & QUALITY

**WARNING!**

Proper maintenance and care may extend the product's service life. Before using the product, please read the following safety instruction carefully.

**1. Please comply with the following before using the product:**

- Clean the product completely: Scrub it with an antibacterial soap and then wash with clean hot water; at last, wipe it gently with towel or soft cloth.

**2. During using the product**

- No using massage oil or hand cream as lubricant.
- No using silicon-based lubricant, because it'll damage the organosilicone surface.

**Reminder: Do not use any cleaning agent containing alcohol, gasoline or acetone to wipe the product. Avoid direct solar radiation and long-time exposure to the high-temperature environment. the product shall be stored in a clean and dry place without contact with other plastic products.**

## Quality

Surpassing standards from Europe and America

### Material Safety

All SVAKOM products are manufactured using environmental friendly material including Silicone and ABS plastics. They are harmless to humans body.

### International Certification

SVAKOM products have passed EU ROHS certification and meet EU CE Directive and WEEE Directive. SVAKOM products are "Green" meeting the official safety and environmental protection standard issued by agencies in Europe and North America.

**Additional Information:** The EU CE Directive and the WEEE Directive are referred to as the "Twin Directives for Environmental Protection" or "Double Green Directives" and are used to restrict hazardous substances in electronic products including lead, cadmium, and mercury,. ROHS stands for "Restriction of the Use of Certain Hazardous Substances"; and WEEE stands for "Waste Electrical and Electronic Equipment". Products that meet the standards set out in these two directives can be advertised as "Green Products".

**EU CE Directive:** is the collective name of a series of product standards issued by the European Union; enterprises complying with the standard may apply for CE certification mark on their products and labelling. SVAKOM products surpass the standards set out by these directives and are authorized to use the mark identifying that the products comply with all EU requirements.

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Dear Valued Customer,

Thank You for choosing SVAKOM! We are dedicated to bring you Intelligent, Sleek, Luxury, Elegant lifestyle products ! Once you registered on our website, you will get the latest updates of our innovative products with creative ways of playing. Enjoy the great pleasure!

## SVAKOM Customer Service

BY REGISTERING YOUR PRODUCT ONLINE, YOU WILL VALIDATE YOUR 1-YEAR WARRANTY / 10 YEAR QUALITY GUARANTEE COVERING ALL MANUFACTURING DEFECTS.

[WWW.SVAKOM.COM](http://WWW.SVAKOM.COM)

[HELP CENTER](#)

[WARRANTY CLAIM](#)

Support: [customer-service@svakom.com](mailto:customer-service@svakom.com)

[@svakomusa](#) [@svakomtheofficial](#) [SVAKOM Official](#) [@svakom\\_usa](#)