



How It Works

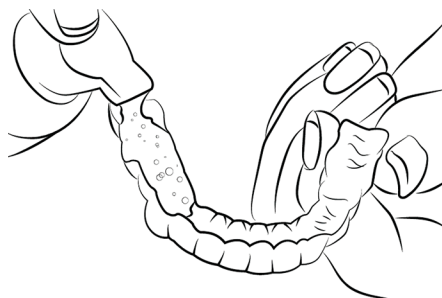


STEP 1

Apply one-half to one pump of EverSmile® WhiteFoam™ to trays.

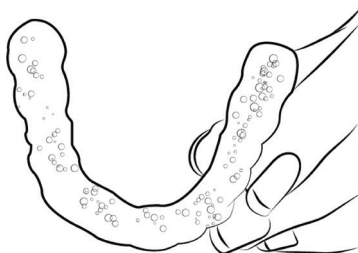

STEP 2

Spread the foam evenly to cover and clean all areas.

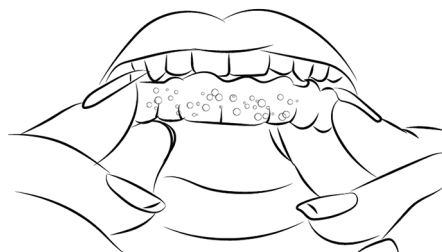

STEP 3

Replace trays in mouth & spit out excess foam.
Wear as directed. Foam will dissipate.

Optional Soaking: Remove aligners or retainers and place them inside their storage case. Cover trays with foam and let them sit for up to 5 minutes. Rinsing and brushing trays before replacing them in mouth is optional.

FAQ

Does EverSmile® WhiteFoam™ cause tooth sensitivity?

Tooth sensitivity has not been reported with the proper use of EverSmile® WhiteFoam™. If sensitivity does occur reduce usage to once per day and consult your orthodontist.

How often can I use EverSmile® WhiteFoam™?

EverSmile® WhiteFoam™ can be used up to 4 times per day. We recommend once in the morning and evening.

How long does one bottle of EverSmile® WhiteFoam™ last?

Each full size 50ml bottle will last approximately 30-60 days depending on how often it is used.

How long until my teeth start to whiten?

The whitening process will begin immediately, with visible results dependent on preexisting stains and individual biology.

Does WhiteFoam™ whiten under the attachments (buttons) placed during Invisalign treatment?

Yes. The hydrogen peroxide foam travels in many directions through the surrounding enamel and reaches under the composite resin attachments.

Is this product safe to use while pregnant?

As with any tooth whitening product, EverSmile® WhiteFoam™ cannot be recommended while pregnant.

Can I use EverSmile® WhiteFoam™ to soak my aligners?

Absolutely! You can soak your clear aligners while they are outside of the mouth to clean and freshen them throughout the day. Please follow the instructions for use.

Which appliances can EverSmile® WhiteFoam™ be used on?

EverSmile® WhiteFoam™ can be used with all orthodontic retainers, soft mouth guards, combination soft and hard mouth guards, and teeth whitening trays. Caution should be taken when using WhiteFoam™ on hard, clear, or acrylic dental splints as they may discolor.

follow us @eversmilewhite



eversmilewhite.com

EverBrands, Inc. • info@eversmilewhite.com
401 N Oak St. Inglewood, CA 90302

EverSmile®

WhiteFoam™

On-The-Go Aligner & Retainer Cleaning & Whitening



Orthodontist Developed • Orthodontist Recommended

- ✓ Wear & Go!
- ✓ No Rinsing
- ✓ No Soaking
- ✓ Freshens Breath
- ✓ Kills Viruses and Bacteria



The Problem

- **Aligners and retainers get dirty the moment they sit in the mouth.**
- **Bacteria levels under aligners soar** due to the lack of saliva flow which acts as a natural tooth cleanser.
- Odoriferous compounds build up, releasing odors and **causing bad breath.**
- Bacteria/plaque biofilm begins adhering to teeth and aligners causing them to **appear cloudy and, in some cases, yellow.**



The Solution

- WhiteFoam™ uses **hydrogen peroxide-based**, anionic EverClean™ technology.
- **99.999% of common oral bacteria are killed within 60 seconds** of exposure to EverSmile® WhiteFoam™ in laboratory testing*.
- **Hydrogen peroxide destroys odor and color molecules** to clean & lighten aligners, attachments, and teeth.
- Hydrogen peroxide is a safe, **effective oral disinfectant** and whitener/cleaner in use for over 100 years.

America's #1 Aligner and Retainer Cleaner!



Whiten While You Straighten!

WhiteFoam™ uses a low dose of hydrogen peroxide meaning you can use WhiteFoam™ daily with virtually no sensitivity.

You will see gradual whitening of your teeth during your aligner treatment. Rest assured that WhiteFoam™ works both under and over attachments so you won't be left with tooth discoloration after your attachments are removed.

Ingredients

Water, hydrogen peroxide, glycerine, PVP, PEG, sodium lauryl sulfate, sucralose, sodium citrate, sodium benzoate, etidronic acid, mentha arvensis (mint) oil.



Gluten Free



Sugar Free



Vegan



Cruelty Free

Amazing Results In As Little As 5 Minutes!



Without WhiteFoam

With WhiteFoam

Compatibility

- Invisalign®
- SmileDirectClub®
- CandidCo®
- ClearCorrect® Aligner Trays
- Essix® & Vivera® Retainers
- Hawley Retainers
- Bleaching Trays
- Dentures/False Teeth

Visit eversmilewhite.com/compatibility for full list. EverSmile® WhiteFoam™ is not affiliated in any way with Align Technology, Inc., Clear Correct Holdings, Inc., or Dentsply Sirona, Inc.