

## 3 Ways to Use FreshDent™




### ON-THE-GO SPRAYER

Spray 10ml sprayer once in mouth, wipe solution across dentures with tongue



### BRUSH

Pump 50ml foamer onto toothbrush, then brush dentures for 2 minutes.

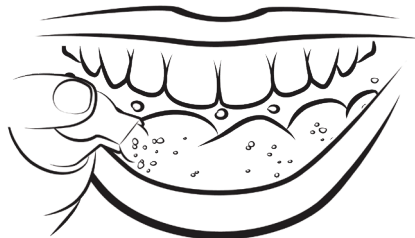

### RINSE

Rinse with EverSmile® FreshDent™ foam for 60 seconds

# FAQ

### Does EverSmile® FreshDent™ cause tooth sensitivity?

Tooth sensitivity for partial wearers has never been reported with EverSmile® FreshDent™ when used as directed.

If sensitivity does occur reduce usage to once per day and consult your dental professional.

### How many times per day can I use EverSmile® FreshDent™?

EverSmile® FreshDent™ can be used safely up to 6 times per day.

### How long does one bottle of EverSmile® FreshDent™ last?

Each bottle (50ml or 10ml) will last approximately 30 days if used 3 times daily.

### Is FreshDent™ safe to use while pregnant?

The use of any oral hydrogen peroxide-based formulations cannot be recommended to those who are pregnant.

### Why was FreshDent™ developed?

FreshDent™ was developed for patients wearing partials and dentures who needed an easier, better way to clean, remove stains, and freshen breath.

Using a patent-pending formula that combines hydrogen peroxide and cleaning agents, FreshDent™ is a safe and convenient way to kill the bacteria that cause bad breath, tooth decay, and mouth sores. FreshDent whitens and renews by breaking down stains and bacterial biofilms that get embedded into partials and dentures over time.

### Where can FreshDent™ be purchased?

FreshDent™ can be purchased through our website at [eversmilewhite.com](http://eversmilewhite.com), and on [amazon.com](http://amazon.com)

Follow us @eversmilewhite



[eversmilewhite.com](http://eversmilewhite.com)

EverBrands, Inc. • [info@eversmilewhite.com](mailto:info@eversmilewhite.com)  
401 N Oak St. Inglewood, CA 90302

EverSmile®

# FreshDent™

The Only Wearable  
Denture & Partial Cleaner



Dentist Developed • Dentist Recommended

- ✓ Cleans Dentures
- ✓ Freshens Breath
- ✓ Cleans Partial
- ✓ Kills Bacteria

## Finally, Innovation in Denture Care!

The only wearable denture cleaner you can keep in your pocket! Foamer for at-home use and sprayer for on-the-go so your dentures are always fresh and clean. **We use NO harsh chemicals**, which means you can clean your dentures while you're wearing them-- FreshDent™ is discreet and convenient!



### The Problem

- Cleaning dentures and partials on-the-go is embarrassing and inconvenient.
- **Unclean dentures harbor bacteria and cause bad breath.**
- **Biofilm builds up**, making dentures feel slimy and unpleasant in the mouth.
- **Denture adhesive does not always grip well.**



### The Solution

- FreshDent™ comes with a **sprayer that cleans dentures while they are worn**, allowing for convenient and discreet maintenance, anywhere.
- The hydrogen peroxide foam **kills 99.999% of bacteria within 60 seconds**, eliminating odors and providing minty fresh breath.
- **A clean oral environment allows denture adhesive to work better and last longer.**

## Clean your Dentures On-The-Go with NO HARSH CHEMICALS!



### Ingredients

Water, hydrogen peroxide, glycerine, PVP, PEG, sodium lauryl sulfate, sucralose, sodium citrate, sodium benzoate, etidronic acid, mentha arvensis (mint) oil.



Gluten Free



Sugar Free



Vegan



Cruelty Free

### Compatibility

- Dental prosthesis
- Partial
- Dental implants
- Fake teeth