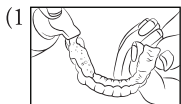
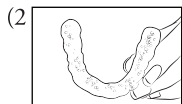
ENGLISH INSTRUCTIONS

Apply one-half to one pump of EverSmile® WhiteFoam to trays.

Spread foam evenly to cover all areas.

Place trays in mouth and spit out excess foam. Wear as directed.

Description: EverSmile® WhiteFoam is a patented cleaning formulation designed to clean and freshen orthodontic aligners and clear retainers (trays). EverSmile® WhiteFoam utilizes EverClean™ (anionic technology) powered by hydrogen peroxide and kills 99.999% of bacteria.

Directions:

On-The-Go Tooth Whitening and Cleaning: Remove trays. Dispense 1/2 to 1 full pump of foam inside each tray, spreading evenly with a finger. Place trays back into mouth and spit out excess foam. Wipe off fingers. Use up to 4 times daily to whiten, clean, and freshen trays and teeth.

Outside of Mouth Cleaning: Remove trays and place them inside their storage case. Cover trays with foam and let them sit for up to 5 minutes. Rinsing and brushing trays before replacing them in mouth is optional.

CAUTION & First Aid: If irritation (such as redness, swelling, soreness) of the gums or mouth occurs, discontinue use and consult a dentist. Adult supervision is required for use by children under 12. Avoid contact with eyes, skin, or hair; flush with water if contact is made. Avoid swallowing; do not induce vomiting. Do not apply to gums or under tongue. Do not use if nursing or pregnant. Keep away from sunlight and heat as contents may pressurize. Use only with orthodontic aligners and clear vacuum formed plastic retainers. Prolonged use may require dentist supervision. US Poison Control Center: (800) 222-1222. Australia Poisons Info Centre: 13 11 26. New Zealand Poisons Info Center: 0800 764 766. Canada Emergencies: 911. EU Poison Centres: echa.europa.eu

Ingredients: water, hydrogen peroxide, glycerine, PVP, PEG, sodium lauryl sulfate, sucralose, sodium citrate, sodium benzoate, etidronic acid, mentha arvensis (mint) oil.

How Supplied: 50ml, 25ml, 15ml. Mint flavor.

Compatibility: (visit eversmilewhite.com/compatibility for full list)

- Invisalign® Aligner Trays
- ClearCorrect® Aligner Trays
- Bleaching Trays
- Essix® Retainers
- Vivera® Retainers

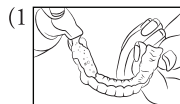
EverSmile® WhiteFoam™ is not affiliated in any way with Align Technology, Inc., Clear Correct Holdings, Inc., or Dentsply Sirona, Inc.

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EverSmile, Inc.