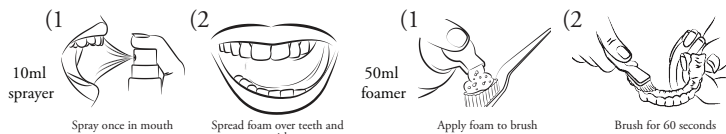
## ENGLISH INSTRUCTIONS

**Description:** EverSmile® FreshDent™ is a patented, deep cleaning, on-the-go denture and partial cleaner that kills bacteria and freshens dentures and partials.

**Directions: 50ml Foamer**

**Brushing:** Remove dentures or partials & pump foam on all surfaces or on toothbrush then brush for 60 seconds. Rinse with water.

**Rinsing:** Dispense 1 to 2 pumps inside mouth and swish for up to 60 seconds, then spit out foam and rinse mouth with water.

**Directions: 10ml Sprayer**

**On-The-Go:** Spray 1 pump into mouth on dentures and tongue. Use tongue to spread solution over teeth and swish with product for up to 20 seconds.

**CAUTION & First Aid:** If irritation (such as redness, swelling, soreness) of the gums or mouth occurs, discontinue use and consult a dentist. Adult supervision is required for use by children under 12. Avoid contact with eyes, skin, or hair; flush with water if contact is made. Avoid swallowing; do not induce vomiting. Do not apply to gums or under tongue. Do not use if nursing or pregnant. Keep away from sunlight and heat as contents may pressurize. Prolonged use may require dentist supervision. U.S. Poison Control Center: (800) 222-1222. Australia Poisons Info Centre: 13 11 26. New Zealand Poisons Info Center: 0800 764 766. Canada Emergencies: 911. EU Poison Centres: echa.europa.eu.

**Ingredients:** water, hydrogen peroxide, glycerine, PVP, PEG, sodium lauryl sulfate, sucralose, sodium citrate, sodium benzoate, etidronic acid, mentha arvensis (mint) oil.

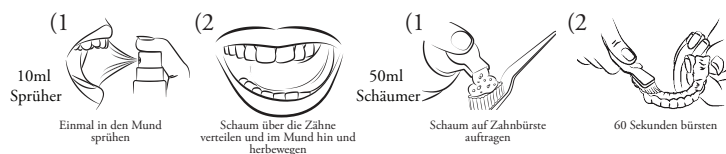
**Compatibility:** (visit [eversmilewhite.com/compatibility](http://eversmilewhite.com/compatibility) for full list)

- Dentures
- Partial
- Dental Implants
- False Teeth

**Manufactured By:** EverBrands, Inc. 401 N. Oak St. Inglewood, CA 90302 +1 (855) 595-2999 info@eversmilewhite.com

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