

CERVOS KEYPRP Single (40ml)

Single Spin PRP

Materials & Preparation

CERVOS KEYPRP Single:
Incl. 1x CERVOS Process Disposable (CPD)



Process Steps



Extract PPP



Extract PRP



Process: **No Brake!**

Conditioned Plasma: 1900g ; 2 min.

Buffy Coat EcoPRP: 1900g ; 6 min.

Note: Swipe Luer Port with Alcohol Wipe Prior to Each Access. Do not Shake or Tilt Filled CPDs to Avoid Inadequate Mixing of Fluids.

** Ensure Luer Connection is Tightly Fastened after Removing Syringe from Process Disposable:*

*** Use Syringe Aspiration Technique or included Screw Set*