Bedienungsanleitung / Manual

AXION STIM-PRO Comfort

2-Kanal Nervenstimulator

INHALTSVERZEICHNIS / CONTENT

2 – 15	Deutsche Bedienungsanleitung	DE
16 – 29	Instruction Manual in English	EN
30 – 43	Manuel d'instruction Français	FR
44 – 57	Manual de Instrucciones Español	ES
58 – 71	Istruzioni per l'uso Italiano	IT
72 – 85	Instrukcja Obsługi Polski	PL

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DE

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INTENDED APPLICATION

The primary functions of the STIM-PRO Comfort are pain relief and pain treatment with the TENS mode. The device is used by the patient himself directly.

EN

What is pain?

Pain is a warning system and the body's method of telling us that something is wrong. Without it, abnormal conditions and injuries to vital parts of the body may go undetected. A persisting (long-term) pain is known as chronic pain. Pain occurs when information signals travel to the brain via nerve fibres, where it gets decoded, analyzed, and then reacted to. The pain information travels from the injured area along small nerves leading to the spinal cord. Here it travels up the spinal cord to the brain. The information is then interpreted, referred back and the pain is felt.

Explanation of TENS

There is nothing "magic" about **T**ranscutaneous **E**lectrical **N**erve **S**timulation (TENS). TENS is intended to be used to relieve pain. The TENS unit sends comfortable impulses through the skin that stimulate the nerve (or nerves) in the treatment area. In many cases, this stimulation will greatly reduce or eliminate the pain sensation the patient feels. Pain relief varies by individual patient, mode selected for therapy, and the type of pain. In many patients, the reduction or elimination of pain lasts longer than the actual period of stimulation (sometimes as much as three to four times longer). In others, pain is only modified while stimulation actually occurs. You may discuss this with your physician or therapist.

How TENS works

Transcutaneous Electrical Nerve Stimulation is a non-invasive, drugfree method of controlling pain. TENS uses tiny electrical impulses sent through the skin to nerves to modify your pain perception. TENS does not cure any physiological problem; it only helps control the pain. TENS does not work for everyone; however, in most patients it is effective in reducing or eliminating the pain.

Indications

People use TENS to relieve pain for several different types of illnesses and conditions. They use it most often to treat muscle, joint, or bone problems that occur with illnesses such as osteoarthritis or fibromyalgia, or for conditions such as low back pain, neck pain, tendinitis, or bursitis. People have also used TENS to treat sudden (acute) pain, such as labor pain, and long-lasting (chronic) pain, such as cancer pain.

SCOPE OF DELIVERY

Each STIM-PRO Comfort is fully equipped with the following standard equipment.

- ▶ **1 piece** **TENS machine**
- ▶ **4 pieces** **Self-adhesive electrodes**
- ▶ **1 piece** **9V battery**
- ▶ **1 piece** **Storage case**
- ▶ **2 pieces** **Electrode cables**
- ▶ **1 piece** **Instructions manual**

IMPORTANT SAFETY INFORMATION

Read instruction manual before operation. Be sure to comply with all “CAUTIONS” and “WARNINGS” in the manual. Failure to follow instructions can cause harm to user or device.

EN

SIDE EFFECTS

Side effects of TENS machines are generally mild, even with extended use. If any effects are especially bothersome, a physician can evaluate the continued use of the device.

Skin irritation and minor burns are potential side effects. If this occurs, discontinue use and consult your physician.

CAUTIONS & CONTRAINDICATIONS

- ▶ Only use original accessories from the manufacturer for your device.
- ▶ Do not use this device for undiagnosed pain symptoms until consulting a physician.
- ▶ Patients with an implanted electronic device, such as a cardiac pacemaker, implanted defibrillator, or any other metallic or electronic device should not undergo TENS treatment without first consulting a doctor.
- ▶ Patients with heart disease, epilepsy, cancer, fever or any other health condition should not undergo TENS treatment without first consulting a physician.
- ▶ Stimulation delivered by this device may be sufficient to cause electrocution. Electrical current of this magnitude must not flow through the thorax or across the chest because it

may cause a cardiac arrhythmia. **Do not place electrodes above the heart!**

- ▶ Do not place electrodes on the front of the throat as spasm of the Laryngeal and Pharyngeal muscle may occur. Stimulation over the carotid sinus (neck region) may close the airways, make breathing difficult, and may have adverse effects on the heart rhythm or blood pressure.
- ▶ Do not place electrodes on your head or at any sites that may cause the electrical current to flow transcerebrally (through the head).
- ▶ This device should not be used while driving, operating machinery, close to water, or during any activity in which involuntary muscle contractions may put the user at undue risk of injury.
- ▶ Always turn off the device before applying or removing TENS accessories to the skin.
- ▶ Isolated cases of skin irritation may occur at the site of electrode placement following long term application. If this occurs, discontinue use and consult your physician, if the irritation does not disappear.
- ▶ If TENS therapy becomes ineffective or unpleasant, stimulation should be discontinued until its use is re-evaluated by a physician.
- ▶ Keep this device out of the reach of children.
- ▶ The units have no AP/APG protection. Do not use it in the presence of explosive atmosphere and flammable mixture.
- ▶ Only apply electrodes to healthy, uninjured skin.

WARNINGS


EN

- ▶ The unit should be used only under the continued supervision of a physician.
- ▶ The unit devices have no curative value.
- ▶ The unit is a symptomatic treatment and as such suppresses the sensation of pain, which would otherwise serve as a protective mechanism.
- ▶ The unit is not effective for pain of central origin. (This includes headache)
- ▶ Electronic monitoring equipment (such as ECG monitors and ECG alarms) may not operate properly when electric stimulation is in use.
- ▶ Caution should be used in applying electric stimulation to patients suspected of having heart disease. Further clinical data is needed to show there are no adverse results.
- ▶ Electrodes should not be placed over the eyes, in the mouth, or internally.
- ▶ Do not use while sleeping.
- ▶ Do not use during pregnancy unless directed by your physician.
- ▶ The long-term effects of chronic electrical stimulation are unknown.
- ▶ Stimulation should not be applied over the carotic sinus nerves, particularly in patients with a known sensitivity to the carotid sinus reflex.
- ▶ Stimulation should not be applied transcerebrally (through the head).
- ▶ Stimulation should not be applied over swollen, infected, or inflamed areas or skin eruptions, e.g., phlebitis, thrombophlebitis, varicose veins, etc.
- ▶ Stimulation should not be applied over, or in proximity to, cancerous lesions.

OPERATING THE DEVICE

Power On/Off switch and intensity controls

The device can be switched on and off by turning the intensity controls. The intensity can be increased by turning the intensity controls clockwise and decreased by turning the intensity controls counter-clockwise. The intensity can be adjusted from 0 to 80mA. The display lights up as soon as the device is turned on.

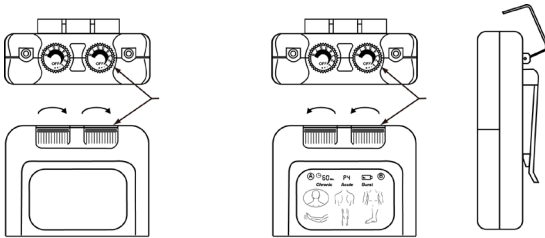Program control

There are 9 TENS programs available. They can be selected by pressing the button "program". The program is activated automatically 3 seconds after selecting the program.

Timer ⌚

The device has a timer that can be adjusted for 15, 30, 45 and 60 minutes as well as continuous mode (C). The application ends automatically when the timer runs out.

Low battery indicator

If this symbol is shown, the battery is low and needs to be replaced.

Program mode (P)

All parameters are set by default.

EN

TENS STIMULATION

The programs have proven to be helpful in most cases for the suggested applications. However, they might be used for the treatment of other injuries or discomforts as well. Every patient responds individually to stimulation current.

TENS programs P1 - P4

P 01 Frequency modulation

Chronic

For most cases of chronic pain. Anti-habituation effect for long-term applications. Choose a convenient intensity.

Pulse frequency: modulated from 5 - 125 Hz

Pulse width: 120 μ s

Application time: 30 - 60 minutes as needed

P 02 Classic TENS – gate control effect

Acute

For most cases of acute pain. Usually takes effect quickly. Choose a convenient intensity.

Pulse frequency: 80 Hz

Pulse width: 80 μ s

Application time: 30 - 45 minutes as needed

P 03 Burst TENS

Burst

For acute pain. Choose high intensity.

Pulse frequency: 2 Hz

Pulse width: 200 μ s

Application time: 30 - 45 minutes as needed

P 04 For pain and tensions in neck and cervical spine area

Choose a convenient intensity.

Pulse frequency: 80 Hz

Pulse width: 120 μ s

Application time: 15 - 45 minutes as needed

TENS programs P5 - P9

P 05 For pain in thoracic spine and shoulder

Ideal for longer treatments. Anti-habituation factor can be reduced by the strong pulse width change.
Choose a convenient intensity.
Pulse frequency: 100 Hz
Pulse width:: 60 - 200 μ s
Application time: 45 minutes

P 06 For pain in lumbar spine and lumbago

Massage effec. Choose a convenient intensity.
Pulse frequency: 15 - 150 Hz
Pulse width:: 160 - 260 μ s
Application time: 30 - 45 minutes

P 07 For pain in arm, elbow, tennis elbow, and carpal tunnel syndrome

Choose a convenient intensity.
Pulse frequency: 24 - 80 Hz
Pulse width:: 180 μ s
Application time: 15 - 30 minutes

P 08 Knee pain, osteoarthritis

For long-term treatments. For most pain indications, for example in joints. Choose a convenient intensity.
Pulse frequency: 80/2 Hz
Pulse width:: 250 μ s
Application time: 30 - 60 minutes

P 09 Ankle pain

Also for pain inn arms, legs, feet, muscular deep pain, and sciatica.
Pulse frequency: 8 - 80 Hz
Pulse width:: 150 μ s
Application time: continuously as needed


CONSTRUCTION OF THE DEVICE

Front

EN


Back

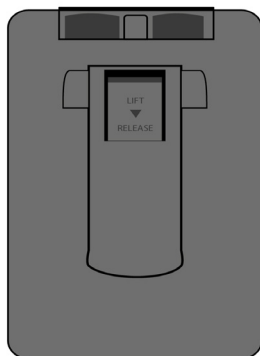BATTERY RECHARGING

When the low battery indicator is shown, the battery needs to be replaced. Remove the battery if the device is not likely to be used for some time.

- ▶ **Make sure the power is off.**
- ▶ **Open the battery compartment (push the cover downwards)**
- ▶ **Remove the battery**
- ▶ **Insert the new battery. Please check that the polarity is correct!**
- ▶ **Close the battery compartment**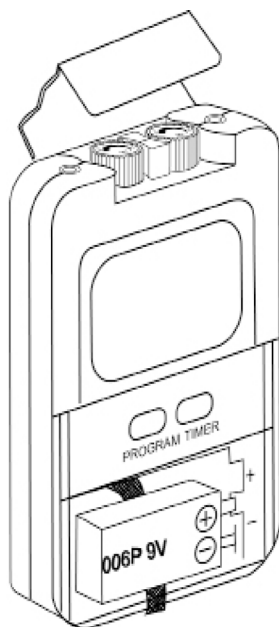

MAINTENANCE, TRANSPORTATION AND STORAGE OF THE DEVICE

EN

- ▶ Non-flammable cleaning solution (containing 70% of alcohol) is suitable for cleaning the device.
- ▶ Stains and spots can be removed with a cleaning agent.
- ▶ Do not submerge the device in liquids or expose it to large amounts of water.
- ▶ Return the device to the carrying box with sponge foam to ensure that the unit is well-protected before transportation.
- ▶ If the device is not to be used for a long period of time, remove the battery and put it back into the carrying box and keep it in a cool, dry place.
- ▶ The packed TENS device should be stored and transported under the temperature range of -20°C $\sim +60^{\circ}\text{C}$, relative humidity 20% \sim 95%, atmosphere pressure 500 hPa \sim 1060 hPa.

SAFETY CHECKS

For safety reasons, review the following checklist once a week.

- ▶ Check the device for deformation of the housing or damage to the output sockets.
- ▶ Make sure that the descriptions and labels are not damaged.
- ▶ Check that the LED is on when the device is turned on.
- ▶ Check the cables and electrodes for damage.

- ▶ The device must be subjected to safety checks and maintenance by authorized technicians before use and each re-use, but at least every 24 months.
- ▶ Please consult your distributor if there are any problems with the device and accessories.
- ▶ The manual must always be carried with the device.

MALFUNCTIONS

Should any malfunctions occur while using your device, check the following points. If none of these can solve the problem, please contact the customer service.

- ▶ Check if the cables are correctly connected to the device.
- ▶ The display does not turn on? Replace the battery.
- ▶ Check the cables for possible damage. The device may only be used when subjected to a load.
- ▶ There is no stimulation current? Check if the electrodes are correctly connected to the same channel and if the intensity is high enough.

CONFORMITY TO SAFETY STANDARDS

The STIM-PRO Comfort device is in compliance with the safety standards EN60601-1:2006/A1:2013 Medical electrical equipment - Part 1: General requirements for basic safety and essential performance & EN60601-1-2:2015 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance – Collateral Standard: Electromagnetic disturbances - Requirements and tests

TECHNICAL DESCRIPTION

01	Channel	2 channels with adjustable intensity
02	Intensity	Adjustabl 0 - 80 mA with a load of 500 Ohm on each channel
03	Pulse amplitude	Asymmetrical, bi-phasic square pulse
04	Output Voltage	0 - 40 V
05	Power Source	1 pc. 9,0V battery
06	Size	10.0 cm (L) x 6.3 cm (W) x 3.2 cm (H)
07	Weight	124 g incl. battery
08	Program 1	Frequency: 5 - 125Hz Pulse width: 120µs
09	Program 2	Frequency: 80Hz Pulse width: 80µs
10	Program 3	Frequency: 2Hz Pulse width: 200µs
11	Program 4	Frequency: 80Hz Pulse width: 120µs
12	Program 5	Frequency: 100Hz Pulse width: 60 - 200µs
13	Program 6	Frequency: 15 - 150Hz Pulse width: 160 - 260µs
14	Program 7	Frequency: 24 - 80Hz Pulse width: 180µs
15	Program 8	Frequency: 80 / 2Hz Pulse width: 250µs
16	Program 9	Frequency: 8 - 80Hz Pulse width: 150µs
17	Low battery indicator	A low battery indicator will show up on the LCD when the battery is low. Change the battery.
18	Operating condition	Temperature 0° C to 40° C Relative humiditiy 30 % - 75 % Air pressure 700 hPa - 1060 hPa
19	Notice	All technical values include a tolerance of +/- 5 %

WARRANTY

All TENS models carry a warranty of 24 months from the date of delivery. The warranty applies to the stimulator only and covers both parts and labor relating thereto. The warranty does not apply to damage resulting from improper handling, the failure to follow the operating instructions, loss or dropping.

EN

Manufacturer:
axion GmbH
Mollenbachstr. 13
71229 Leonberg, Germany
www.axion.shop/en

1.  Advice for operation
2.  Degree of electrical protection Type BF
3.  Do not insert the plug into the power supply socket of 230V
4.  IP protection class
5.  Do not dispose of via residual waste
6.  Read the instruction manual
7.  Direct current (DC)
8.  Manufacturer
9.  Serial number

Manual Version: 1.2

Release Date of the Manual: December 2018

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