


nella[®] KIND

Reusable

FROM




Instructions for Use

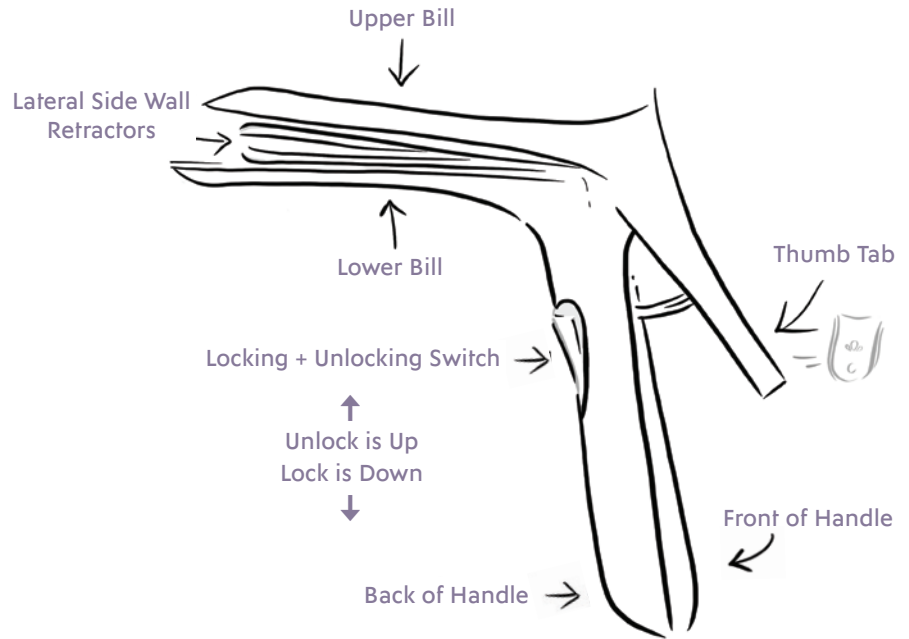

1

Start with the toggle on the back side of the handle in the locked position. This means pressing the inside button down.

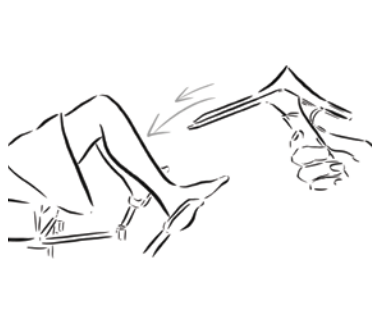

2

Insert the closed and locked speculum into the vagina.



3

Once inserted, unlock the speculum to open. Unlock the speculum by pressing the toggle (located on the inside of the handle) up to the unlock position.




4

Open Nella to desired size by pressing thumb tab on handle where the small flower icon is located.



5

Switch the toggle into the locked position by pressing the lower half of the toggle down.

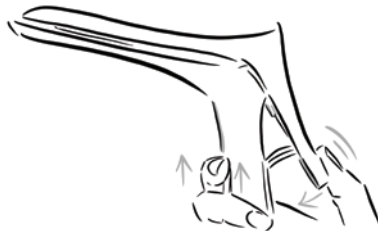

6

When ready to remove Nella, press toggle upward with your pointer finger while at same time using thumb to press thumb tab down (where the flower icon is located). Speculum will unlock and gently close.



7

Remove speculum from vagina.

99%
of women prefer Nella
over a metal speculum¹



nella®
KIND

Reusable

FROM



The 5 Key Benefits of Nella KIND

PATIENT PREFERRED

Improves comfort during GYN exams and procedures with narrow bills, quiet operation and temperature neutral material.

MODERN DESIGN

Narrow bills with integrated lateral side wall retractors and one-handed locking mechanism with an ergonomic design.

STRONG

Autoclavable polymer, twice as strong as metal and single-use specula to ensure patient safety under high stress.

PROVEN EFFICACY

Cervical visualization and access demonstrated in the VIOLET study.

VALUE-ADD EFFICIENCIES

Smooth material does not require lubrication.



NELLA KIND INSTRUCTIONS for USE

Point your smartphone camera at the QR code and click the link to access the Nella KIND training video.

Sources: 1 Ceek Enterprises (2017). VIOLET Clinical Trial. ClinicalTrials.gov Identifier: NCT03123367. Internal Report: unpublished.

Product	SKU/PN	Description	UOM	Qty	Size
Nella KIND	NSRL024	Single-Use Vaginal Speculum with integrated sidewall retractors + LED light	Box	24	OS
Nella KIND	NSRL096	Single-Use Vaginal Speculum with integrated sidewall retractors + LED light	Case	96	OS



www.ceekwomenshealth.com order@ceekwomenshealth.com
PHONE 855-559-CEEK FAX 866-680-3002