


nella[®] NEXT

Single-Use

FROM




Instructions for Use

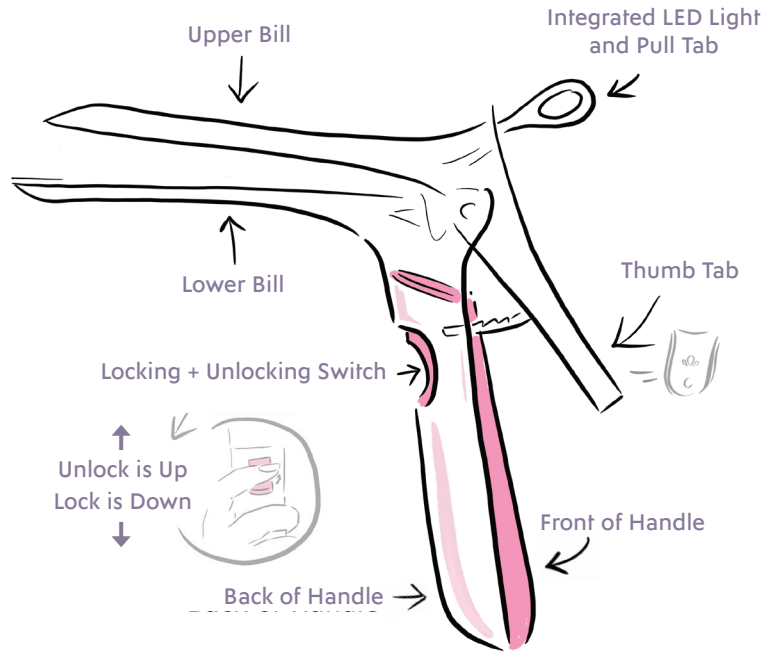

1
Open the polybag by tearing along the perforated line, above the pink arrows.



2
Remove speculum from bag by the pink handle.

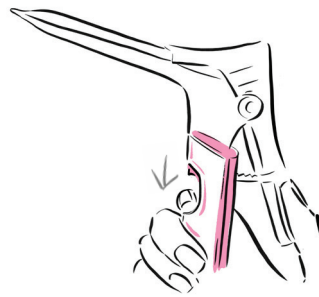


3
On the back handle of the speculum is the locking and unlocking switch. Confirm the switch is down, which is the locked position.



4
Activate the light source by removing the white circle tab on the top of the speculum.

5
Dispose of tab.

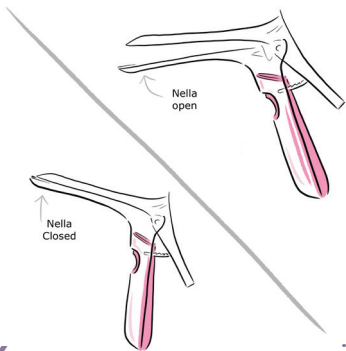


6
This is what the speculum looks like open and closed.



7
Insert the speculum into the vagina. Make sure speculum is fully closed before insertion.




8
Once inserted, open Nella by pressing the thumb tab on handle where the small clear flower icon is located. Nella will automatically stay open and locked in position.

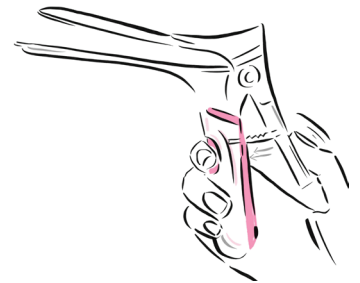

9
If necessary, open speculum by further depressing thumb tab.


INSTRUCTIONS CONT. on BACK

WHEN READY TO REMOVE

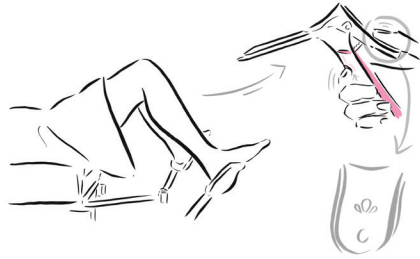

10

Slide the pink/purple switch on the back side of the handle up with your pointer finger to unlock and at the same time maintain pressure on thumb tab (where the flower icon is located).

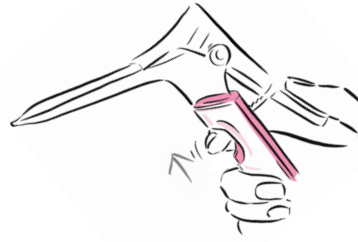

11

Speculum will unlock and gently close.
(Push switch UP to unlock.)



12

Remove speculum from vagina.



13

Dispose of speculum in accordance with local regulations.

Product	SKU/PN	Description	UOM	Qty	Size
Nella NEXT	NSL024	Single-Use Vaginal Speculum with Integrated LED Light	Box	24	M
Nella NEXT	NSL096	Single-Use Vaginal Speculum with Integrated LED Light	Case	96	M
VuSleeve	VS020	Single-Use Speculum Sheath	Box	20	OS



www.ceekwomenshealth.com order@ceekwomenshealth.com
PHONE 855-559-CEEK FAX 866-680-3002