




fiora™

WEIGHTED BLANKET / COUVERTURE LESTÉE

4042000 and / et 4042100

Safety Instructions Consignes de sécurité

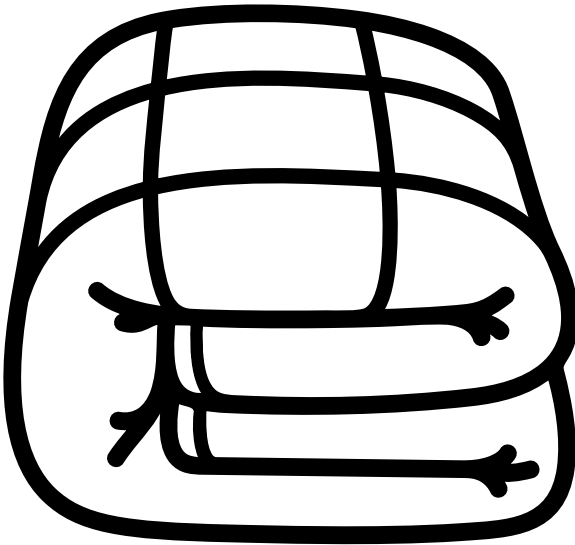

Safety instructions

- Do not use if the product is damaged.
- Do not use for any purpose other than the one mentioned

Important - caution

1. Contraindications for blanket use:
 - Respiratory, cardiac, skin (including some allergies), circulatory disorders;
 - Epilepsy;
 - Significant hypotonia.
2. The blanket is not a restraint device. The individual must be able to remove the blanket at will.
3. Not recommended for children under 3 years old. To be used under the supervision of an adult.
4. Vital signs should be observed while using the blanket.
5. The weight and size of the recommended blanket should be commensurate with the physical constitution of the user, generally a ratio of 10% of the user's weight is usually recommended, consult your occupational therapist for the exact weight at use.
6. Never roll a person in a blanket.

Care Instructions

Machine wash your blanket in cold water then tumble dry at a low temperature. Check if your washing machine and dryer can bear the weight of your blanket.

Consignes de sécurité

- Ne pas utiliser si le produit est endommagé.
- Ne pas utiliser pour un usage autre que celui mentionné.

Important - mise en garde

1. Contre-indications à l'utilisation de la couverture:
 - Problèmes respiratoires, cardiaques, cutanés (dont certaines allergies), circulatoires;
 - Épilepsie;
 - Importante hypotonie.
2. La couverture n'est pas une contention. La personne doit pouvoir sortir par elle-même de la couverture quand elle le désire.
3. Non recommandé pour les enfants de moins de 3 ans. À utiliser sous la surveillance d'un adulte.
4. Les signes vitaux doivent être observés lors de l'utilisation de la couverture.
5. Le poids et la grandeur de la couverture recommandés devraient être en rapport avec la constitution physique de l'utilisateur, de manière générale un ratio de 10% du poids de l'utilisateur est habituellement recommandé. Consulter votre ergothérapeute pour le poids exact à utiliser.
6. Ne jamais rouler une personne dans une couverture.

Entretien

Laver votre couverture à la machine à l'eau froide puis sécher à la machine à basse température. Vérifiez si votre laveuse et votre sècheuse peuvent supporter le poids de votre couverture.