

Instructions to set the time on your Alzheimer's Calendar Clock - 12 alarms

This document contains the relevant information on how to set the time on your Alzheimer's Calendar Clock with 8-inch screen, dual mode date display and 8 languages.

Important note : The buttons on the back of your clock have symbols only. It is recommended that you use the table below to identify the appropriate button to set the different functions of the clock.

HOW TO SET THE TIME

- Press the MENU button to access the configuration screen.
- Press OK then RIGHT/LEFT to choose the field to be modified and OK to select it. A small red light appears on the number to be adjusted.
- Press UP/DOWN to adjust the number.
- When you have completed the change, press OK to confirm.
- Press the MENU button to return to the main screen.

