



**WWW.OSUNGUSA.COM**

**CLEANING AND STERILIZATION INSTRUCTIONS  
FOR  
OCCLUSAL RIM PLATE – ARR1 / VAD-01**

**1) Chemical Sterilization:**

The Occlusal Rim Plate is not intended for patient contact. It is considered a non-critical item and as such can be sterilized by immersing in an EPA registered hospital disinfectant, following the disinfectant manufacturer's recommendations.

**2) Autoclave Sterilization:**

First remove any residual wax from the instrument by heating over a bunsen burner and wiping clean with a paper towel or disposable cloth.

Place the rim former by itself in a protective pouch or wrap to prevent any remaining wax on the instrument from contaminating the autoclave or other instruments.

Follow CDC recommendation for sterilization: Minimum exposure periods for sterilization of wrapped healthcare supplies are 30 minutes at 121°C (250°F) in a gravity displacement sterilizer or 4 minutes at 132°C (270°F) in a pre-vacuum sterilizer.

**PRODUCT WARRANTY:**

Replacement warranty is limited due to the degradation of plastic materials at high temperatures after repeated autoclaving.

11/2/23

15020 West Drive, Houston, TX 77053  
Email: Support@OsungUSA Tel: (844)-870-2781 Fax (847) 589-1298  
www.osungUSA.com