


Halo Ring


User Guide

Presented by
Pi Ring


1 FUNCTION INTRODUCTION



SLEEP
Scientific monitoring sleep data. Effectively help to improve sleep quality.



BODY TEMPERATURE
Body temperature monitoring and analysis. Scientific management of physical status.



BLOOD OXYGEN
Periodic monitoring and analysis of blood oxygen. One-key to detection of blood oxygen.



HEART RATE
Resting heart rate and motional heart rate. Periodic monitoring and analysis.



RECOVERY
Care of various body indicators. Provide body recovery reports.



WOMEN MENSTRUAL CYCLE
AI deep learning, scientific expectation for pregnancy.



SOS
In case of emergency. One-key SOS for rescue.




TOUCH TO CONTROL
Easy and smart touch control the 3rd Parties APPs. Convenient smart life experience.



STRICTLY WATERPROOF
Strictly waterproof performance. Enjoy the fun of sports.


2 PACKING LIST

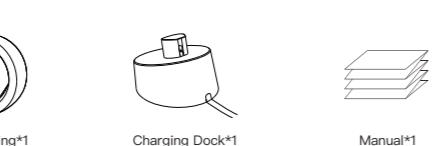

3 BASIC PARAMETERS

Model Name	Smart Health Ring
Product material	Microcrystalline Nano Ceramic and Austenitic Antibacterial Stainless Steel
Communication method	Bluetooth LE5.0
Sensor	Body temperature sensor, acceleration sensor, heart rate sensor, touch sensor
Battery life	4-6 days
Battery	Lithium polymer battery
Charging method	magnetic charging
Charging time	≤1.5hrs
Operating temperature	-20°C to 50°C
Storage temperature	-30°C to 70°C
Charging temperature	0°C to 40°C


4 HALO RING MOBILE APP

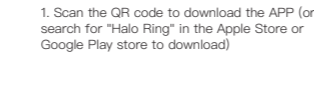

1. Scan the QR code to download the APP (or search for "Halo Ring" in the Apple Store or Google Play store to download)
2. Place the ring on the charging dock connected to the power source and the ring will be active automatically. (the 1st time you bind the ring, you need to put it on the charging dock, Keep it charged.)
3. Open "Halo Ring" APP, Click the Add Ring button on the status window or smart window to enter the ring binding interface and search for the ring.
4. When the ring appear in the Bluetooth list, choose it to start binding; If no found, check that whether the charging dock connection and the ring are charged.
5. Wait for seconds, the APP will automatically connect and bind the ring. During binding, the ring will flash green LED light for 1-2 seconds.
6. If you need to unbind it, click "Unbind Ring" on the setting window of the APP to finish unbinding. If you're using an iPhone, you'll also need to manually remove or ignore your ring from the list of paired Bluetooth devices in your phone's Bluetooth setting.


5 WEARING

When using the Smart Touch feature, please pay attention to the way the ring is worn. The ring should be worn on the index finger and the sensor area located on the inside of the finger (note: the touch experience will be more comfortable when the upper end of the touch logo is aligned with the middle line on the side of the index finger, as shown in the dotted line below)

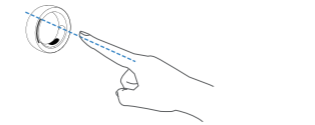


6 USE

After the smart touch function is enabled, you can touch to control the 3rd parties APPs on phone, tablet PC, computer and TV through the ring touch area (the touch area is on the outer surface of the ring, please pay attention to the direction when wearing)

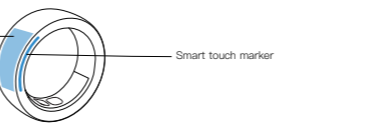

7 CHARGING

1. Connect the charging Dock with USB adapter, then put the ring on the charging Dock as shown on picture, if the ring red light flashes 1 seconds apart, it means that the ring is charging.
2. If fully charged, the ring will start flashing-green. (the charging connection is correct, the red light will continue to light for 10 seconds, if it is not connected correctly, the red light will turn off immediately, please check the charging connection. After connecting the charge for 10 seconds, flash a red light or green light every 1 seconds to indicate the charging status, flashing red light means in charging, flashing green light means full charged.)

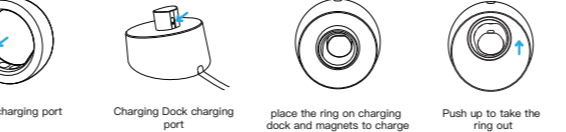

8 SAFETY AND CAUTIONS

1. The ring comes with built-in battery, and the battery is not replaceable, do not disassemble or modify the battery.
2. Do not expose the ring to high temperatures or around heat generating equipment, such as heaters, microwave ovens, etc.

Ensure optimal performance and to avoid dangerous or illegal situations. please read and observe the following use scenarios and operating environment precautions.

1. Do not allow children or pets to swallow the ring to avoid injury.
2. the product is strictly prohibited near the heat source or exposed fire, such as ovens, electric heaters, etc..
3. some people are allergic, their skin to plastic, stainless steel and other materials will be allergic, long-term contact parts will be red, swollen, inflammation and other symptoms occur, anyone with similar conditions, please stop using and consult a doctor.
4. The radio waves generated by the device may affect the normal operation of implanted medical devices or personal medical equipment, such as pacemakers, hearing aids and so on. please consult with their manufacturers for restrictions on the use of this equipment.
5. Please do not dispose of the device and its accessories as ordinary household garbage. Please observe local ordinances for the disposal of the device and its accessories and support recycling actions.

9 CONTENT OF HARMFUL SUBSTANCES

Part name	Hazardous substances					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Charging Dock	○	○	○	○	○	○
Housing	×	○	○	○	○	○
Circuit board	○	○	○	○	○	○
Battery	○	○	○	○	○	○

This form is prepared in accordance with SJ/T11364. indicates that the content of the hazardous substance in all homogeneous materials of the component is below the limit specified in GB/T26572-2011 . indicates that the content of the hazardous substance in at least one homogeneous material of the component exceeds the provisions of GB/T 26572-2011

- ⑩ The number in this sign indicates that the product (except battery) is in normal use and the environmental protection service life is 10 years
- ⑤ The number in this sign indicates that the environmental protection service life of the battery is 5 years under normal use

The environmental protection service life identification on the components takes precedence over any conflicting or different environmental protection service life identification on the products. All environmental protection service life of the product refers to the safe life of the product without leakage of toxic and harmful substances when used under the service conditions specified in the product manual.

10 WARRANTY

- Tips:
1. This product in normal use, the charging dock warranty for 1 year from the date of purchase.
 2. Failure caused by the user's personal reasons, do not provide free warranty as follows:
 - a. Crash since the disassembly, modification, etc. ;
 - b. By inadvertent drops during use.
 - c. All man-made damage or damage due to third-party negligence, external shock crack, damage to external components of the scratch, etc., are not covered by the warranty.

11 SPECIAL INSTRUCTIONS

1. This manual is made based on the existing information, and in the principle of continuous improvement and continuous development, Our company reserves the right to change the product specifications and functions and modify and improve any products described in this manual.
2. The contents in this manual are provided according to the condition of the product at the time of manufacture. Unless otherwise specified by applicable laws, no explicit or default warranty of any kind is made for the accuracy, reliability and content of the document. Any claim based on the data, drawing or text description of this manual will not be accepted.
3. This product is not a medical device. The data and information provided are for reference only.
4. If the out frame of this product is removed without authorization, the product will lose its warranty qualification.
5. The pictures in this manual are used to guide for the user and are for reference only. Please refer to the actual object for details.

12 WARRANTY CARD

1. User information

Full Name	
Contact No.	
Address	
Email	
Post code	

2. Fuselage Information

SN number of device	
---------------------	--

3. Maintenance record

Acceptance date	Problems	Treatment scheme	Signature of maintenance

13 WARRANTY CARD

You can reach our support team.

- +91 9509440149
- info@piring.co.in

Note: For more detailed info, visit on our website www.piring.co.in



Halo Ring

Presented by
Pi Ring