

工号

6765-003

文件名称

英文中性Poco X903说明书

版本号

文件编号

A0

设计时间

2023.8.14

设计师

审批人

郭侣号

王淑雅

材质工艺要求


材质要求: 尺寸: A3
材质: 105g铜版纸过油
工艺: 折页
颜色: 彩色印刷

制作工序:


工艺要求: 色相正确、印迹牢固、
套印准确、各色套印不露杂色,
套印误差 $\leq 0.1\text{mm}$
模切走位 $\leq \pm 0.5\text{mm}$

颜色及专色:

纹理要求:

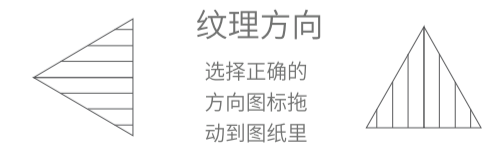


比例 1:1 单位 mm

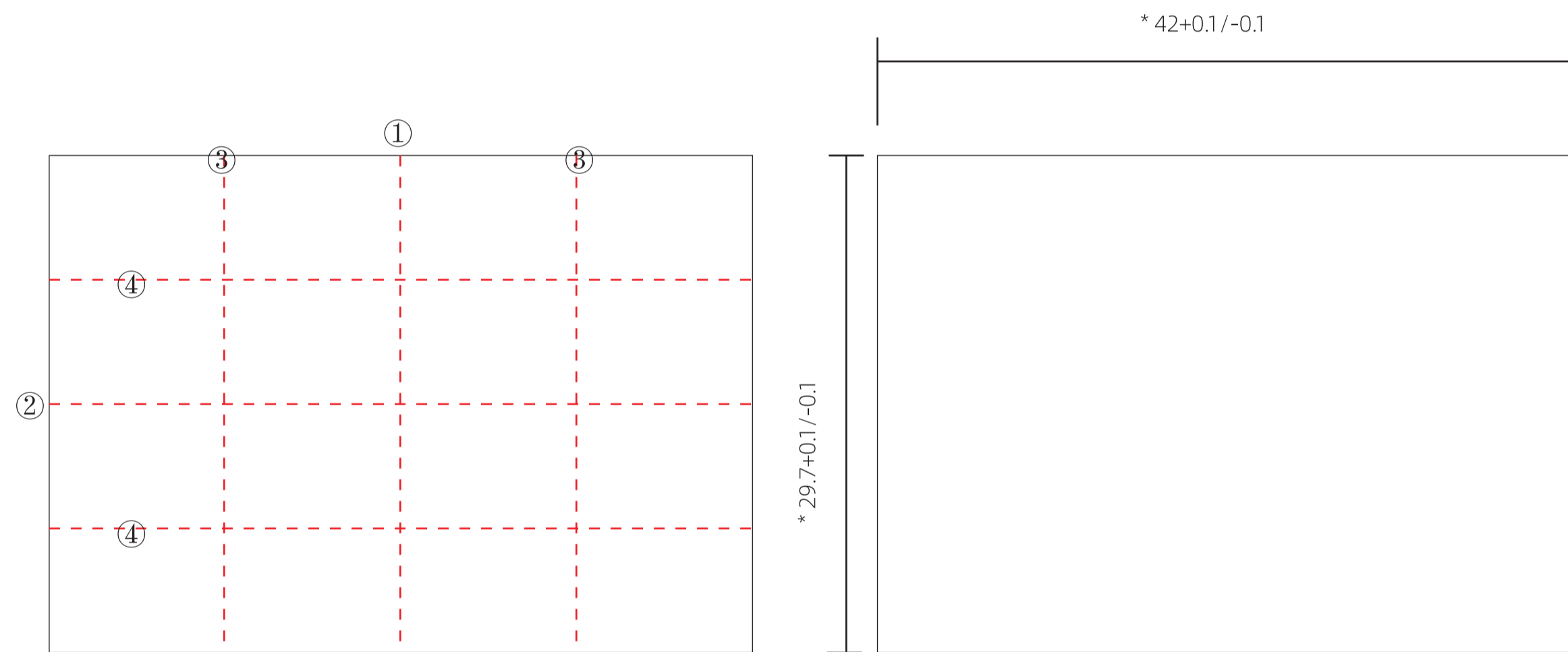



* 折页描述, 一共折四次, 按标注序号对折线依次对折

Know about the hearing devices and accessories

1. Hearing devices

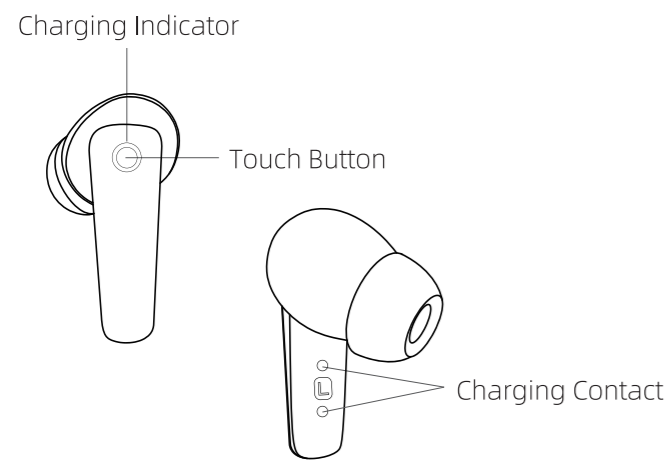

2. Charging case

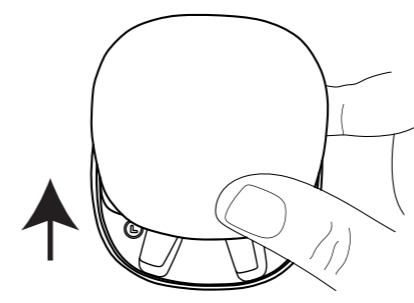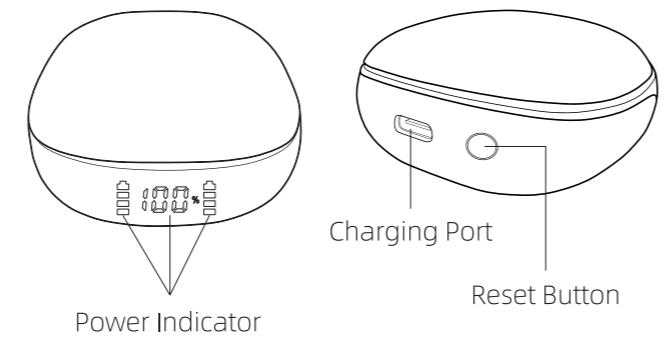

Tip:
You can open the cover of the hearing aid case by gently sliding your finger upwards as the photo shows.


3. About the dome

- Ellipse dome**
Ellipse dome effectively reduce the feedback / whistling sound.
- Semi-open dome**
Effectively reduce the feedback sound and occlusion effect.


Notice:
The number and type of accessories are subject to situation.

How to assemble and wear the devices

4. How to assemble

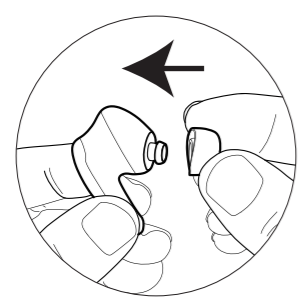

Step 1
Choose a dome.

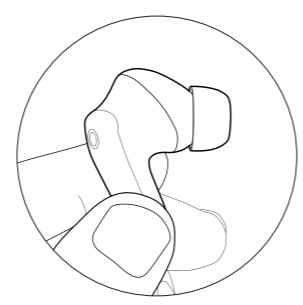

Step 2
Mount onto the connector firmly.

⚠ Notice: The devices come with pre-assembled domes. You can replace one to fit your special needs.

5. How to wear



Step 1
Hold the device.




Step 2
Push the device into best fit position.


⚠ Notice: Not to touch the microphone port during wearing then position securely in your ear to avoid whistling. Once properly inserted, whistling will stop.

How to charge the devices


6. How to charge the devices

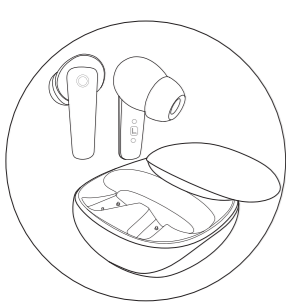


Step 1
Place the devices into charging case.

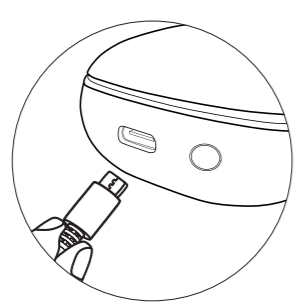

Step 2
Plug the small end of cable into charging port of charging case.

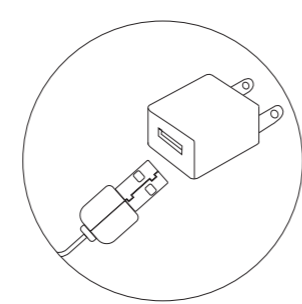

Step 3
Plug the big end of cable into adaptor.

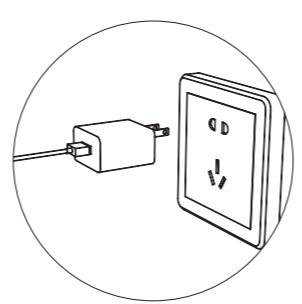

Step 4
Connect with power.

⚠ Notice: Avoid charging at high temperatures, as this can shorten the life span and capacity of the power cell.

Working time	Hearing enhancement: 6 hours Music streaming: 6 hours Phone answering: 6 hours
Time of full charge	For device: 1.5 hours For charging case: 2.5-3 hours
Charging time (per pair)	4 times

⚠ Notice:
1) Always put your device into the charging case when you do not use them.
2) We strongly recommend you fully charging the devices when you first receiving.

How to charge the devices


7. Indicator light

	Hearing devices	Charging case
Charging	The touch button blinking blue light	The power indicator will flash the corresponding power percentage.
Fully charged	The white light turn off	Indicator turn off

Low battery indicator:
The device will emit voice indicator "Battery low" every 30 seconds.

8. ON-OFF switch

	Indicator light	Switch ON-OFF
Turn ON	Blue	Long press for 3s
Turn OFF	Blue	Long press for 5s

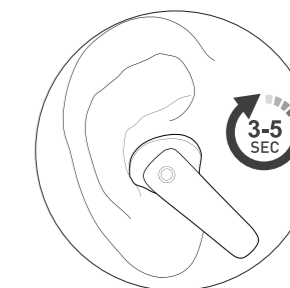

Notice:
The device will turn off when placed into charging case. If failed, please follow the operations above.

How to use the devices

9. Connect with cellphone

- Step 1: Take out of charging case and wear your devices, you will hear the voice indicator "Power on" and then "Pairing".
- Step 2: Enable the Bluetooth on your cellphone.
- Step 3: Search the device name of "Poco X903".
- Step 4: Tap on the "Poco X903" to connect your device with your cellphone. Once connected, you will hear the voice indicator "Connected".



Notice: 1. When you are asked to allow access to your phone's location information, it is solely for the purpose of establishing a connection between the phone and the device. Rest assured, the device manufacturer does not collect any personal information.
2. After the first pairing, the devices will automatically connect to the cellphone when the Bluetooth turned on. If you finds two devices ID, it means the left and right ear devices lost sync pairing. Please restart pairing refer to the next page and then follow the steps above to reconnect.

10. Hearing enhancement

If not connect to the Bluetooth, the devices will automatically in the mode of hearing enhancement. There are three modes of hearing enhancement. To change the modes, you can choose in the SOUNDWEAR App, or double tap on the touch button. Beep will indicate a mode changing. Tips: You can fine tuning the volume in the SOUNDWEAR App.

Mode	Beeps
Normal	1 beep
Noise reduction	2 beeps
Strong noise reduction	3 beeps

11. Other functions

- Phone Call**
Please double tap touch button to answer/hand up the call, and long press touch button for 2s to reject the call. Please adjust the Volume on your smartphone.
- Music Streaming**
Please adjust Volume on your smartphone or through the music App. Activate Virtual Assistant/Siri
Long press touch button for 1s.
- Restart pairing**
Put the devices into the charging case, long press the Reset Button on the charging case for 10 seconds to clear up all pairing records, and entry pairing mode. At that time, indicator lights on the charging case will flash 3 times.

How to use the APP

12. Download


Search SoundWear in



Or Scan to Install



Kindly refer to the APP manual for better using.

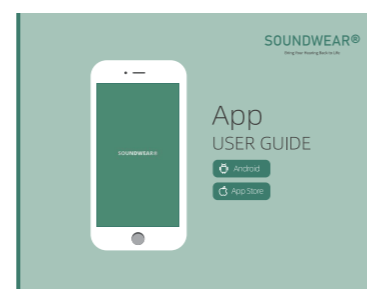

Notice:
Only Available on iOS 9 or later, iPhone 5 or above; Android 6 or later, screen size 4" or above. You need to connect the device with your smartphone first, then you can connect the devices in the SOUNDWEAR App.

Maintenance

13. Care (You should inspect and clean your hearing devices everyday)

- 1. Clean the exterior & charging contact**
To clean, gently wipe the surface with a soft, dry cloth. Use the brush to clean the receiver port. Do keep the charging contact and charging port clean and dry. It is recommended to use dry box or electronic dry box for drying every day to reduce moisture and wax buildup.
- 2. Clean the dome**
Use the brush to clean the dome. It's recommended that replace with new dome every 2-3 months.

Notice: The hearing instruments and charger must be clean and dry in any case.

