

INSTRUCTION MANUAL



Overview

Dear Customer


Thank you for choosing our product,

The manual will introduce you to some basic methods of using hearing aids and solutions to common problems to help you use hearing aids better.

Please read it carefully and regard it as a user guide.


-01-

Structure

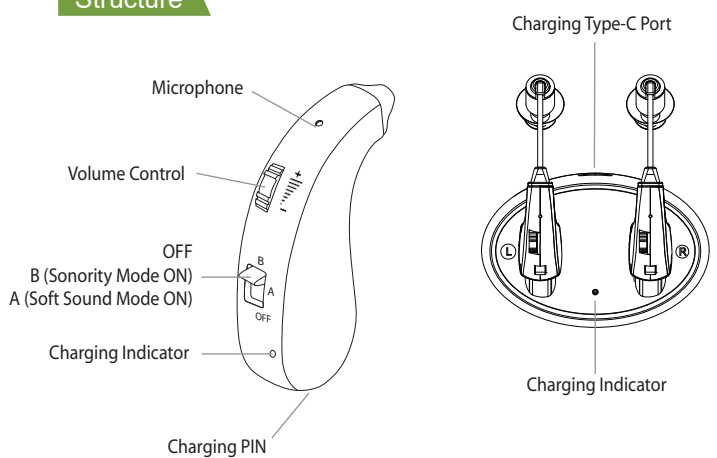

-02-

Make sure correct side

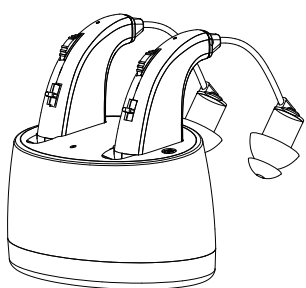

Make sure correct side

So that it's better to set a suitable volume or program on different sides.

-03-

Operating Steps


1. Turn the volume to the lowest level before use.
2. Turn the power switch to "A" or "B" position before putting it into your ear as follows
3. Regulate the volume to a proper level slowly.
4. Choose "A" or "B" mode to meet your needs.
5. Be sure to switch off after use to save power.
6. Charge it for next use.



Correct way of wearing



Put the earplug into your ear

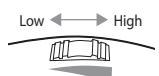


-04-

Volume Control

1) Volume Gear control from 1 to 4 level

"1" Position for Low Sound

"4" Position for High Sound

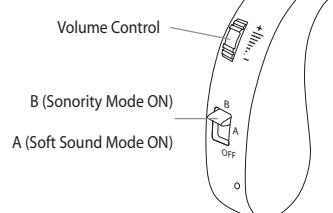

Different programs

1) There are two programs that can be selected:


"A" Position for Soft Sound Mode ON

"B" Position for Sonority Mode ON

-05-

Charging Instruction

1. Turn the power switch to the "OFF" position before charging, and put the hearing device into the charging dock
 2. Plug the adapter into a power outlet via Type-C cable
- INPUT : 100V ~ 240V, AC 50/60Hz, 0.2A
OUTPUT : 5V = 1A
3. When charging, the green indicator light turns on, and the light will be closed after the charging is finished. The whole charging process take around 3 hours.
 4. Please charge it in time to avoid affecting the battery performance and battery life.

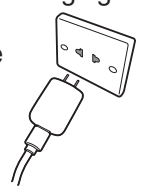

-06-

Simple Trouble Shooting

Note: Do not take your hearing amplifiers apart without authorization, so as not to cause damage.

| Problem | Possible Causes/Solution |
|---------------------------|--|
| No sound | 1. Check if the earplug is inserted tightly; 2. Check if the hearing amplifiers are out of battery; 3. Adjust the volume to make the sound louder; 4. Check whether the earwax in the earplug has been cleared. |
| Sound is weak | Adjust the volume to make the sound louder. |
| Noise or bad tone quality | 1. Check whether it is in a strong magnetic field environment; 2. Check if the earplug is inserted tightly. |
| Choppy sound | Too much earwax in the earplugs, clean with the cleaning brush. |

-07-

Safety

- 1) Warning: Keep small parts that can be swallowed out of children's reach.
- 2) Warning: Do not put the hearing aid in a microwave oven or fire, as it may cause an explosion.
- 3) Users who have undergone surgery need to use the hearing aid under the guidance of a professional doctor.
- 4) Users who have undergone surgery need to use hearing aid under the guidance of a professional doctor.
- 5) Special User:
 - 5.1) Children can't wear hearing aids without the guidance of adults..
 - 5.2) Mental patients should consult a physician before wearing a Sound Amplifier.

-08-