


INSTRUCTION MANUAL

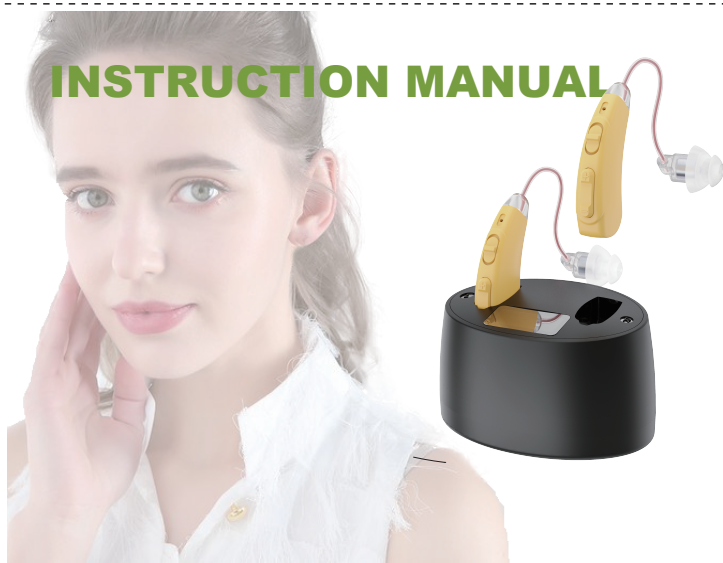

Overview

Dear Customer


Thank you for choosing our product,

The manual will introduce you some basic use methods of hearing aids and the solutions of common problems to help you use hearing aids better.

Please read it carefully and regard it as a user guide.

- 01 -


Structure

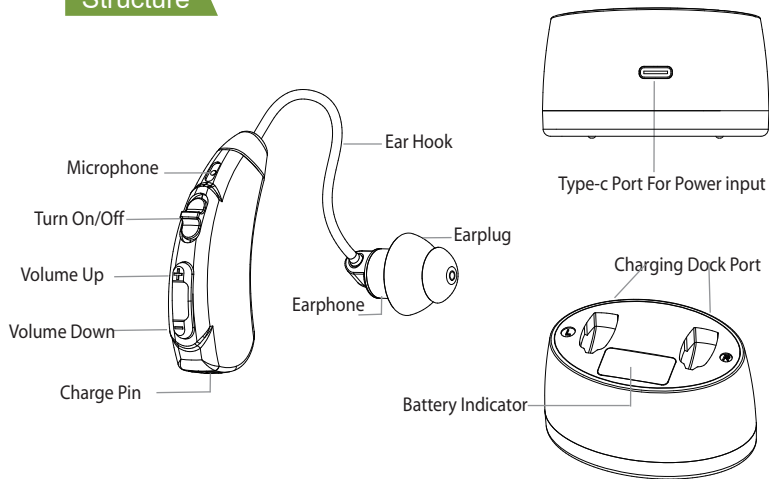

- 02 -

Operating Steps

1. Choosing the right size of the Earplug that to suit your ear.
2. Turn " ON/OFF " Key to Up before putting into your ear.
3. Change the volume to a proper level
4. Charge it for next use

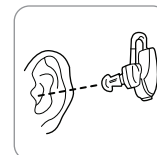

Put the earplug into your ear



Correct way of wearing

- 03 -

- 04 -

Volume Control

- 1) Press " Volume Up " button to increase volume
Big " Bi Bi " sound when Max volume
- 2) Press " Volume Down " button to Reduce volume
Weak " Du Du " sound when Min volume



Turn on / off

- 1) Here are ON/OFF Button
Up to Turn On
Down to Trun off




- 05 -

Charging Instruction

1. Put hearing device into the charging dock
2. Charging Lamp
Green lamp flashing, charging....
Green lamp steady on or turn off, charged...
It takes about 2 hours to fully charge.
3. Please charging it in time to avoid affecting the battery performance and battery life.
4. Plug the adapter into a power outlet via Type-C cable

INPUT : 100V ~ 240V, AC 50/60Hz, 0.2A
OUTPUT : 5V = 1A

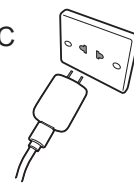

- 06 -

Simple Trouble Shooting

Note: Do not take your hearing amplifiers apart without authorization, so as not to cause damage.

Problem	Possible Causes/Solution
No sound	1. Check if the earplug is inserted tightly; 2. Check if the hearing amplifiers is out of battery; 3. Adjust the volume to make the sound louder; 4. Check whether the earwax in the earplug has been cleared.
Sound is weak	Adjust the volume to make the sound louder.
Noise or bad tone quality	1. Check whether it is in a strong magnetic field environment; 2. Check if the earplug is inserted tightly.
Choppy sound	Too much earwax in the earplugs, clean with the cleaning brush.

- 07 -


Safety

- 1) Warning: Keep small parts that can be swallowed out of children's reach.
- 2) Warning: Do not put the hearing aid in a microwave oven or fire, as it may cause an explosion.
- 3) The specific hearing aid must only be used by the intended person and not by others.
- 4) Users who have undergone surgery need to use hearing aid under the guidance of a professional doctor.
- 5) Special User :
 - 5.1) Children can't wear the hearing aid without the guidance of adult.
 - 5.2) Mental patients should consult a physician before wearing a Sound Amplifier.

- 08 -

Bluetooth Function

- 1) Long press " Volume Up " Button for 3 seconds to open bluetooth function

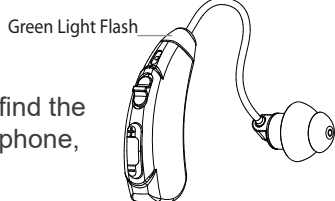

- 2) The green light is flashing, now, find the name " 108BT " from the mobile phone, and click the connection.

- 3) Just turn off if need to close bluetooth function.

Re: The Bluetooth can not work if battery low, so please charge it when the Bluetooth function is not used.

Different Programs

- 1) Long press " Volume down " button for 4 seconds to change different programs

Noise reduction mode if one "Bi" sound

Normal mode if two "Bi Bi" sound

