



# Gballs<sup>3</sup> App



- EN** Press ON/OFF to switch on  
Download Magic Kegel app from App Store or Google Play  
Connect your smartphone with Gballs<sup>3</sup>  
Follow the instructions in the manual
- DE** Drücken Sie ON / OFF um das Gerät einzuschalten  
Laden Sie die Magic Kegel App aus dem App Store oder von Google Play herunter  
Verbinden Sie Ihr Smartphone mit Gballs<sup>3</sup>  
Befolgen Sie die Anweisungen im Handbuch
- FR** Appuyez sur ON / OFF pour allumer  
Téléchargez l'application Magic Kegel depuis l'App Store ou Google Play  
Connectez votre smartphone avec Gballs<sup>3</sup>  
Suivez les instructions du manuel
- ES** Presione ON / OFF para encender  
Descargue la aplicación Magic Kegel desde la App Store o Google Play  
Conecte su teléfono inteligente con Gballs<sup>3</sup>  
Siga las instrucciones del manual
- RU** Для включения нажмите значок ON/OFF   
Скачайте приложение Magic Kegel из iOS App Store или Google Play  
Соедините смартфон с тренажёром Gballs<sup>3</sup>  
Следуйте инструкции в приложении

## Specifications

Material	Body - safe silicon / ABS
Size	30x100 mm
Battery	200 mAh
Work time	4 hours
Frequency	7200/ min
Water Resistance	100%

## Start

### Download for iOS and android

Find **Magic Kegel** for iOS at the App Store or at the Google Play for android.



## Launch Magic Kegel

Click the application to launch and follow the guidelines in the manual to connect Gballs<sup>3</sup>.

### Connect Gballs<sup>3</sup>

To switch on Gballs<sup>2</sup>, press the power button. LED lights turn on when Gballs<sup>3</sup> work.



The application will show that the device is connected when Gballs<sup>3</sup> successfully connects to the smartphone.



## Enjoy the workout


Find a comfortable position to insert the Gballs<sup>3</sup>. Many women find that it is most comfortable to insert when lying down on your back or standing up.

To make the process easier, apply a small amount of water-based lubricant to the tip of Gballs<sup>3</sup> or onto a condom.



### Workout mode

- Turn on Gballs<sup>3</sup> and connect to the **Magic Kegel** application.
- On the **Magic Kegel** application settings page, select the «Off-line mode» option.

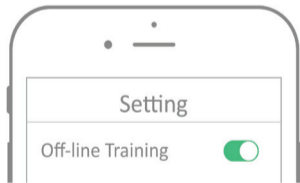


- Turn on the Gballs<sup>3</sup> device and the workout will start in 30 seconds. Squeeze the muscles of the pelvic floor when the vibration starts. Follow the instructions in the application.

**TIP:** To exit the Off-line Mode, reconnect Gballs<sup>3</sup> to Magic Kegel application.

### Independent Modes (without using a smartphone)

#### Vibration mode when squeezing

Turn on the Gballs<sup>3</sup> device. Squeeze the muscles of the pelvic floor to control the vibration of the device.

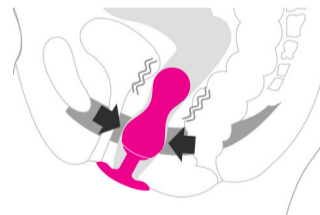

#### Constant vibration mode

Press the power button to go from the «Squeeze» mode for a constant vibration mode.

## How to charge

- Plug the USB cable into the charger slot. Connect to the USB port of the laptop, personal computer or to the charging adapter for smartphones. Plug the adapter into an outlet.
- The indicator light signals that the charging has started. The indicator turns off when the device is charged completely.

**TIP:** The period of the complete charging should be 4 hours.

### Cleaning Gballs<sup>3</sup>

Wash the device with the warm water with antibacterial soap before using for the first time. After using, wash the toy with the soap and wipe it dry with a towel.

### Switching Off

To switch off the device press and hold the power button for 3 seconds. The indicator turns off, signaling that the device is off.

### Storage

- Keep the Gballs<sup>3</sup> device in a cool, ventilated place. Do not expose to direct sunlight.
- Keep out of the reach of children.

## EC Declaration of Conformity

We: Gvibe Ltd.  
Registered at: Paneeli 2b, 11415, Tallinn, Estonia

in accordance with the following Directive(s):

2014/53/EU  
Article 3.1a) : EN 60950-1:2006+A11:2009+A1:2010+A12:2011+A2:2013  
Article 3.1b) : Draft ETSI EN 301 489-1V2.2.0 (2017-03)  
Draft ETSI EN 301 489-17 V3.2.0 (2017-03)

hereby declare that:

Equipment: Vibrating Rechargeable Kegel balls  
Branded: Gballs2 App (transmitting power: 0dBm; used frequency: 2.4G)  
Model No: FT-10127

Is in conformity with the applicable requirements of the above directives and the following standards:

EN 55032:2015  
EN 55035:2017  
Article 3.2): Draft ETSI EN 300 328 V2.2.0 (2017-11)  
EN 62479: 2010

I hereby declare that the equipment named above has been designed to comply with the relevant sections of the above referenced specifications. The unit complies with all applicable Essential Requirements of the Directives.

Designed by Gvibe, England. Made in China  
ft london llp. All Rights Reserved.  
23a, High Street Sheringham, Norfolk, NR26 8JP, UK  
DESIGN PROTECTED



## 3-YEAR WARRANTY