


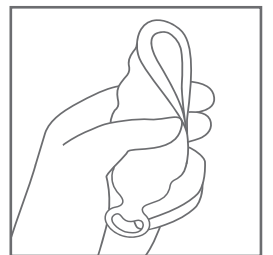


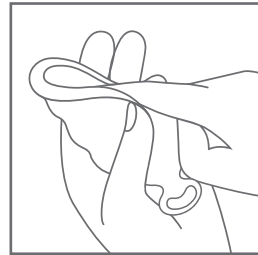


Gcup

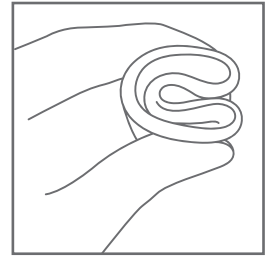


- EN** Menstrual cup
Size 6x3.5 cm
- DE** Menstruationstasse
Die Größe 6x3.5 cm
- FR** Coupe menstruelle
Taille 6x3.5 cm
- ES** Copa menstrual
Tamaño 6x3.5 cm
- RU** Чаша менструальная
Размер 6x3.5 cm

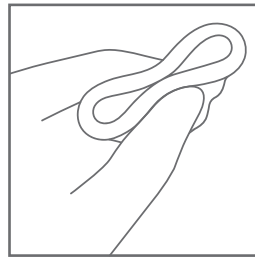

Gcup is equipped with a CHECK VALVE

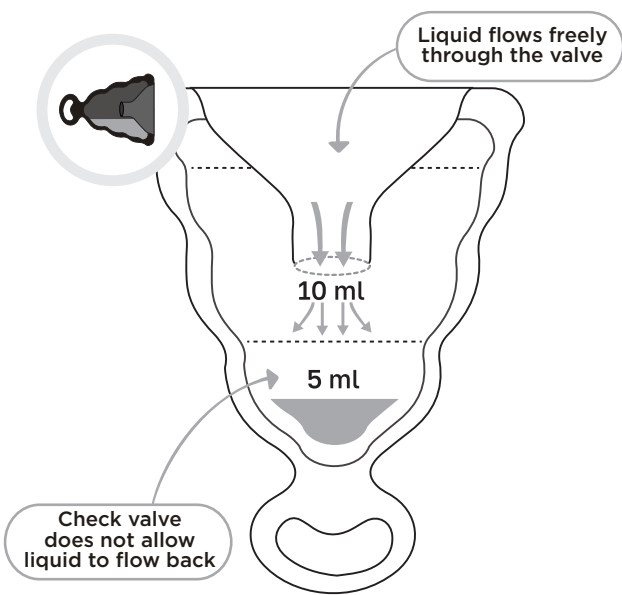

DAY TO NIGHT PROTECTION
The innovative valve protects safely against leakage

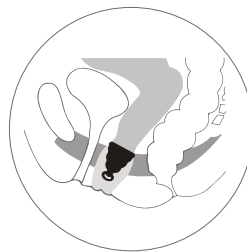

IF PLACED PROPERLY

You will not feel the cup inside.
Gcup reduces cramps during menstruation.
You will not feel irritated and dry after changing the cup, as opposed to disposable hygienic tampons.

The Gcup equipped with a check valve suits ideally for first-time menstrual cup users.

Size S is suitable for teenagers.

Size S is recommended for women with a delicate constitution or for lighter menstruation flow days.

Whether you have just started menstruating or you've been experiencing your period for years, Gcup will give you peace of mind during these days!

How to use



Sterilize the cup for 2-3 minutes before the first use and at the beginning of each period.



Wash your hands thoroughly with soap and water.



Fold the cup. Insert the folded cup into your vagina.



Remove the cup after 12 hours or as soon as it is full. Empty it.



Wash the cup with warm water and soap and use again.



At the end of the period, store the cup in a bag, away from sunlight.

3-YEAR WARRANTY