


ACETATE FRONT STYLES

Standard cellulose acetate frontplates can be hot or cold mounted. No tech in frontplate, glaze as you normally would.

Temples are injection plastic and non-adjustable. Try another style if fit is not in range of adjustment. Heating temple may damage components.

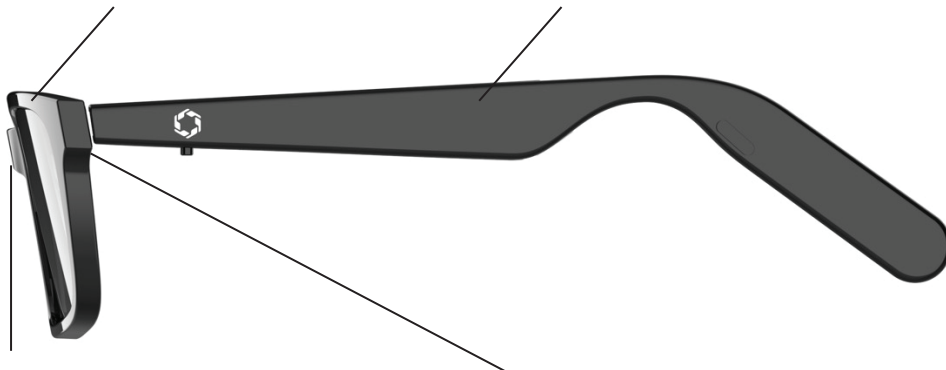


To tighten or loosen temples significantly, heat the bridge and bend it inward or outward slightly.

Corners of frontplate can be heated and adjusted by hand or with a plier to change temple angles or pantoscopic angle.

TITANIUM FRONT STYLES


To tighten or loosen temples significantly, heat the bridge or frontplate corners and bend it inward or outward slightly.

Titanium styles have a little more built-in flexibility at the hinge, and so are recommended for non-standard head shapes.

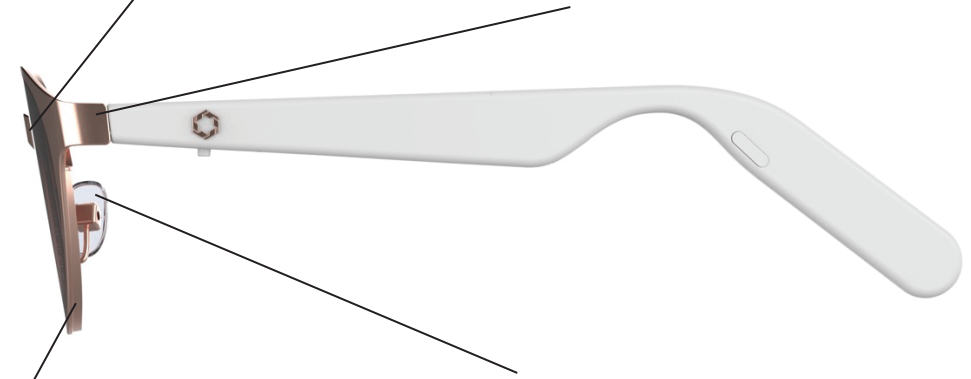

All titanium styles have a standard screw-mount rim, no heating necessary. Recommend high index lenses to customer to reduce lens overhang on the rim.

Traditional prong nose pads allow for tightening and loosening on the bridge.

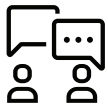

NEED SOME MORE HELP? Don't hesitate to reach out to our support team at wholesale@lucyd.co or at our support line: 1800-530-7230