SELECT

PROFCARE

1

2

3

4

Made in China
SELECT SPORT A/S
Fabriksparken 46
DK-2600 Glostrup
select-sport.com

UK

Ankle Support 2-parts - Art. 70564

Function:

- Supports and stabilizes the ankle
- Is used for sprains and instability of the ankle
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

Instruction for use:

1. Place the foot in the sock. Put the ball of your foot on the round circle of the strap. 2. Take the end of the strap A. Pull the strap from the arch of the foot around the outside of the foot, proceed around the ankle and attach the strap to the sock. 3. Take the other end B, pull around the ankle and attach to the lime green Velcro. 4. Make sure that the large lime green oval is on the outside of the foot to get the best result.

Material:

- Ankle sock: 60% Neoprene, 25% Chloroprene rubber, 6% TPU, 5% Polyester, 4% Spandex. Elastic band: 41% Polyester, 41% Nylon, 18% Rubber.

Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to heat, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

Washing instructions:

1. Close all Velcro
2. Mesh laundry bag is recommended
3. Stretch product while wet and let dry

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