SELECT

PROFCARE

Made in China
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com

UK

Ankle Support Lace-up - Art. 70562

Function:

- Provides optimal support and protection against ankle sprains
- Quick lace-up system for an incredibly easy application
- Open heel design
- Top elastic strap for a customized fit
- Padded lining for extra comfort
- Fits both left and right foot

Instructions for use:

Place the ankle support as shown on the front of the packaging.

Material:

- Outer materials: 92% nylon 8%PU
- Inner materials: 82% nylon, 18% spandex
- Inner foam: 100% PU

Important information:

- Do not use on patients with a known sensitivity to any of the product components.
- The product is not flameproof, avoid temperature above 120°C/ 248°F.
- The function of the product must not be changed or compromised.
- The product should only be used for its original, intended indication.
- Time of use should not exceed 3-4 hours.
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support.
- If you are allergic to heat, we advise you to use a product that does not contain SBR/neoprene.

To ensure correct fit, try the product on.

Washing instructions:

1. Close all Velcro
2. Remove detachable splints
3. Mesh laundry bag is recommended
4. Stretch product while wet and let dry

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